Biomnis

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;	HIV1/2 - Human Immunodeficiency Virus - screening serology - serum
.	Amino acids - chromatography
.	Anti-DNA antibodies Double stranded DNA - Serum
.	Anti-TSH receptor antibodies
.	Circulating anticoagulant - screening - plasma
.	Hepatitis B - anti-HBc antibodies - IgM - serum
.	Hepatitis B - total anti-HBc antibodies - serum
.	HIV2 - Human Immunodeficiency Virus - viral load - Plasma
.	Pro-gastrin-releasing-peptide (ProGRP)
.	Syphilis - Treponemique test - screening serology - serum
.	Syphilis - VDRL or quantitative RPR
.	West Nile virus - IgG/IgM serology - serum
10-Hydroxy-carbazepine - serum	
1,25-dihydroxycalciferol	Vitamin D - 1,25-dihydroxyvitamin D - serum
1,25(OH)2-vitamin D3	Vitamin D - 1,25-dihydroxyvitamin D - serum
1,25(OH)2D	Vitamin D - 1,25-dihydroxyvitamin D - serum
11-Deoxycorticosterone	
11-Desoxycortisol - Compound S -	
11-Desoxycortisol - Compound S - - (Tubes without separating gel)	
11-DF	11-Desoxycortisol - Compound S - - (Tubes without separating gel)
11-OH-THC	Cannabidoides - dosage - plasma
1-3 B-D-Glucan	Beta-D-Glucan antigen - screening
13C-urea breath tests - Helicobacter pylori	Helicobacter pylori - 13C urea breath test - expired gas
1691G>A mutation	Factor V Leiden - p.Arg506Gln mutation
16S rRNA	Detection and identification bacterial DNA - 16S PCR by NGS sequencing
17 beta-oestradiol	Oestradiol - man/child - serum
17 beta-oestradiol	Oestradiol - woman - serum
17 beta-oestradiol	
17-alpha-Hydroxyprogesterone	17-Hydroxyprogesterone
17-Hydroxy-delta-5-pregnenolone	17-Hydroxy-Pregnenolone - serum
17-Hydroxy-Pregnenolone - serum	
17-Hydroxyprogesterone	17-Hydroxyprogesterone
17-Hydroxyprogesterone	
17-OH progesterone	17-Hydroxyprogesterone
17-OH-P	17-Hydroxyprogesterone
17-OH-pregnenolone	17-Hydroxy-Pregnenolone - serum
2019-nCoV	COVID-19 (SARS-CoV-2) IgG serology anti-S
2019-nCoV	COVID-19 (SARS-CoV-2) serology (total Ig) anti-N
2,4'-DDE	Pesticides - screening and/or dosage - urine
2,4'-DDT	Pesticides - screening and/or dosage - urine
21-Deoxy F	21-Deoxycortisol - (Tubes without separating gel)
21-Deoxycortisol - (Tubes without separating gel)	
21-DF	21-Deoxycortisol - (Tubes without separating gel)
21-DOB	21-Deoxycortisol - (Tubes without separating gel)
21-Hydroxylase - antibodies	Anti-21-hydroxylase antibodies - serum
[-2]pro-PSA - Phi score calculation - serum	
23S	Helicobacter pylori and clarithromycin resistance gene - direct diagnosis - PCR
27dupC	MUC1 nanopore Nephropathie
3 methoxytyramine	Methoxylated derivatives of catecholamines - Free metanephries - child - Urine
3 methoxytyramine	Methoxylated derivatives of catecholamines - Free metanephries - Urine
3 ortho methyl dopamine	Methoxylated derivatives of catecholamines - Free metanephries - child - Urine
3 ortho methyl dopamine	Methoxylated derivatives of catecholamines - Free metanephries - Urine
3 Ortho Methyl dopamine	Methoxylated derivatives of catecholamines - Free metanephries
3- (B-aminoethyl) -5-hydroxyindol	Serotonin - whole blood
3-Alpha-Androstenediol Glucuronide - serum	
3-hydroxy-3-methyl-glutaryl-CoA reductase antibodies	Anti HMGCR antibodies - serum
3tc	Lamivudine - plasma

4,4' -DDT	Pesticides - screening and/or dosage - urine
4,4'-DDE	Pesticides - screening and/or dosage - urine
50% haemolysis time - alternative pathway complement - serum	
5 -pregnenolone	
5-alpha, 3-alpha-Androstenediol glucuronide	3-Alpha-Androstenediol Glucuronide - serum
5-alpha DHT	DHT - Dihydrotestosterone - serum
5-alpha-Androstenediol	3-Alpha-Androstenediol Glucuronide - serum
5-alpha-Dihydrotestosterone	DHT - Dihydrotestosterone - serum
5-FU Toxicity	Uracil - dosage(search a risk of toxicity to fluoropyrimidines)
5-HIAA	5-HIAA - 5-hydroxyindoleacetic acid - Urine
5-HIAA - 5-hydroxyindoleacetic acid - Urine	
5-HNMP	5-Hydroxy-N-methylpyrrolidone - urine
5-HT	Serotonin - whole blood
5-Hydroxy-N-methylpyrrolidone - urine	
5-Hydroxytryptamine	Serotonin - whole blood
5'-Nucleotidase - activity - serum	
5-OHT - whole blood	Serotonin - whole blood
5-Pregnenolone	Pregnenolone - serum
5̳Alpha̳reductase	DHT - Dihydrotestosterone - serum
5ème maladie éruptive de l'enfant	Parvovirus B19 - direct diagnosis - PCR
5'Nu	5'-Nucleotidase - activity - serum
5P	Pregnenolone - serum
5P	
6 mono acéthyl morphin	Opiates - dosage - plasma
6-hydroxy-(mono-propyl-heptyl) phthalate (OH-MiDP) - metabolite of DiDP	Phthalates - extended panel - urines - Phthalates - extended panel
6-monoacetylmorphine	Opiates - dosage - urines
6-sulfatoxymelatonin - urine	Melatonin - 24-hour urine collection
6-sulfatoxymelatonin - urine	Melatonin - Urine
7-Dehydrocholesterol	
9 Hydroxy-Risperidone	Risperidone
A1 Antiplasmin	Alpha-2 Antiplasmin - activity - plasma
A1 Antitrypsin	Alpha-1 Antitrypsin - dosage - Serum
A1 Antitrypsin - clearance	Alpha-1 Antitrypsin - clearance
A1-microglobulin	Alpha-1 Microglobulin - urine
A and B Influenza	COVID 19 / Influenza virus A and B - panel - direct diagnosis - PCR
A2-macroglobulin	Alpha-2 Macroglobulin - serum
A-fetoprotein	AFP - Alpha fetoprotein - fluid
A-fetoprotein	AFP - Alpha fetoprotein - serum
A-fetoprotein - amniotic fluid	AFP - Alpha-fetoprotein - amniotic fluid
A2M	Alpha-2 Macroglobulin - serum
AAC	Amino acids - chromatography - urine
Abacavir - plasma	
ABCG2	Pharmacogenetic Passport - PharmacoPass Expert
ABETA 1-42 protein	TAU - PTAU - Amyloid proteins - LCR
Abilify®	Aripiprazole
ABO blood grouping - RH1(D)	
ABO system immune antibodies - anti-A anti-B - screening and titration	
Ac anti Glial acidic fibrillary protein	anti GFAP antibodies - Repeat testing - CSF - anti GFAP antibodies
AC ANTI RACH	Anti-acetylcholine receptor antibodies - serum
Ac anti soluble liver antigen	Anti-SLA antibodies - Soluble Liver Antigen - serum
ACD	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
ACD	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
ACE - Angiotensin Converting Enzyme	
ACE - Angiotensin Converting Enzyme - BAL	
Acetoacetate	Ketone bodies - Deproteinized heparin whole blood sample or CSF
Acetoacetic acid	Ketone bodies - Deproteinized heparin whole blood sample or CSF
Acetylcholine Esterase - amniotic fluid	
Acetylcholine receptor - anti-antibodies	Anti-acetylcholine receptor antibodies - serum

ACHe - amniotic fluid	Acetylcholine Esterase - amniotic fluid
Aciclovir	Valaciclovir - Plasma
Acid Alpha-1 glycoprotein	Orosomucoid - serum
Acid phosphatases total - serum	
Acide N-methylperfluoro-1-octanesulfonamidoacetique (NMeFOSAA)	Per- and polyfluoroalkyl compounds - 7 compounds panel - Serum
aCL	Anti-cardiolipin antibodies - IgM - Serum
aCL-IgG	Anti-cardiolipin antibodies - IgG - Serum
ACOA	Exome with interpretation interface
ACPA	Constitutional Molecular Karyotype-SNP ARRAY
ACPA	Constitutional Molecular Karyotype - SNP ARRAY - Product of conception
ACPA	exome
ACPA	Nephrology exome
ACPA (Anti-Citrullinated Protein Antibodies)	Anti-cyclic citrullinated peptide antibodies - CCP2
Acquired FISH	Haematological and molecular cytogenetics - FISH
Acquired karyotypes	Blood karyotype
acquired molecular cytogenetics	Haematological and molecular cytogenetics - FISH
ACTH - Corticotrophin	
ACTH - Corticotrophin - evening	
ACTH - Corticotrophin - morning	
ActiTest	Fibrotest - Actitest - . - serum
ActiTest®	NASH-FibroTest - serum
ACTITEST	FibroMAX - . - serum
Activated Partial Thromboplastin Time - aPTT - plasma	
Activated protein C resistance - plasma	
Activated protein C resistance test	Activated protein C resistance - plasma
Active testosterone	Testosterone bioavailability - serum
Acute leukemia	Immunophenotyping of malignant hemopaties
ADA	Adenosine Deaminase - ADA
adalimumab	Adalimumab
Adalimumab	
ADAMTS13	Factor Willebrand protease activity - ADAMTS 13
Adenocarcinoma	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
Adenosine Deaminase - ADA	
adenovirus	Adenovirus - IgG serology - serum
Adenovirus	Respiratory Infection Panel - direct diagnosis - PCR
Adenovirus - IgG serology - serum	
Adenovirus - screening - Stools	
Adenovirus F 40/41	Gastrointestinal Panel - direct diagnosis - PCR - Stools
Adenylosuccinase - deficiency syndrome screening - urine	
ADH - Antidiuretic hormone	
ADH - Antidiuretic hormone - urine	
ADN bactérien	Designated bacterium - direct diagnosis - PCR
Adoport	Tacrolimus - FK-506
adrenal androgen	DHEAS - Dehydroepiandrosterone sulfate
Adrenal glands - anti-antibodies	Anti-adrenal gland antibodies - serum
Adrenaline	Catecholamines - free - Urine
Adrenaline	Free catécholamines
Adrenaline	Free catécholamines - children - Urine
Adrenocortical hormones - anti-antibodies	Anti-adrenal gland antibodies - serum
Adrenocorticotropic hormone	ACTH - Corticotrophin
ADTKD	MUC1 nanopore Nephropathie
ADV	Adenovirus - IgG serology - serum
Advagraf	Tacrolimus - FK-506
Aerobic / anaerobic bacteria - MIC: Minimum Inhibitory Concentration	
AFAP1	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Afinitor	Everolimus
Afipia felis	Bartonellosis - B. henselae - B. quintana - IgG serology - serum
AFP - Alpha fetoprotein - fluid	

AFP - Alpha fetoprotein - serum	
AFP - Alpha-fetoprotein - amniotic fluid	
African trypanosomiasis	Trypanosoma gambiense - serology - CSF
African trypanosomiasis	Trypanosoma gambiense - serology - serum
Age Check	Juvenalis - AgeCheck - man
Age Check	Juvenalis - AgeCheck - woman
AGL	Fatty acids free non-esterified - serum
AGNA	Anti- SOX1 antibodies - CSF
AGNA	Anti- SOX1 antibodies - serum
AGNE	Fatty acids free non-esterified - serum
Akineton®	Biperiden - serum
AKT1	DNA NGS Panel solid tumor Breast
AKT1 - ALK - AR - ATM - BARD1 - BRAF - BRCA1 - BRCA2 - BRIP1 - CDK12 - CDK4 -	DNA NGS Panel solid tumor Pan-organ : 64 genes
ALA	delta-Aminolevulinic acid - urine
ALA U-	delta-Aminolevulinic acid - urine
Alatop	Specific IgE : inhalant allergen testing - Phadiatop®
Albuminuria	Proteins - dosage - urine
Aldosterone	
Aldosterone - Lying down	
Aldosterone - Standing or Sitting position	
Aldosterone - urine	
Aldrin	Pesticides - screening and/or dosage - urine
Alepsal®	Phenobarbital - serum
ALK	RNA NGS panel for brain solid tumors
ALK	RNA NGS Panel for lung cancers
ALK	RNA NGS Panel for thyroid cancer
ALK	RNA NGS Panel solid tumors, Pan-organ : 65 genes
alkaline phosphatase electrophoresis	Alkaline phosphatases - isoenzymes - serum
Alkaline phosphatases - fluid	
Alkaline phosphatases - isoenzymes - serum	
Alkaline phosphatases - serum	
Alkaline phosphatases bone - serum	
Alkaptonuria	Homogentisic acid
ALL	Immunophenotyping of malignant hemopaties
Allèles HLA-C - génotypage	HLA-C genotyping (HLA de classe I) - Total EDTA blood
Allergens - food allergens - mix	Specific IgE : food allergens - allergen mix - serum
Allergens - food allergens - single	Specific IgE : Phadia - food allergens - single allergen
Allergens - hymenoptera venom	Specific IgE : hymenoptera venom - single allergen
Allergens - inhalant allergens - mix	Specific IgE : inhalant allergens - allergen mix
Allergens - inhalant allergens - single	Specific IgE inhalant allergens - single allergen - GSD Gold Standard Diagnostic -
Allergens - inhalant allergens - single	Specific IgE : Phadia - inhalant allergens - single allergen
Allergens - latex	Specific IgE : latex - single allergen
Allergens - medicines	Specific IgE medicines - single allergen - GSD Gold Standard Diagnostic - serum,
Allergens - medicines	Specific IgE medicines - single allergen-Phadia
allo platelet immunisation	Anti-platelet antibodies bound or free - screening and identification
Allo-immunization against RH3	Fetal RH3(E) - genotyping
Allo-immunization against RH4	Fetal RHc - genotyping
ALP	Alkaline phosphatases - serum
ALP - isoenzymes	Alkaline phosphatases - isoenzymes - serum
Alpha interferon	Interferon alpha - biological fluid
Alpha interferon	Interferon alpha - CSF
Alpha interferon	Interferon alpha - serum
Alpha interferon - neutralizing antibody	Interferon alpha - neutralizing antibody - serum
Alpha subunit - hCG, FSH, LH, TSH - serum	
Alpha-1	Orosomucoïd - serum
Alpha-1 Antitrypsin	Alpha-1 Antitrypsin - dosage - Serum
Alpha-1 Antitrypsin - clearance	Alpha-1 Antitrypsin - clearance
Alpha-1 Antitrypsin - clearance	

Alpha-1 Antitrypsin - dosage - Serum	
Alpha-1 Antitrypsin - PI phenotype	
Alpha-1 Antitrypsine - Testing for S and Z variants of the SERPINA1 gene	
Alpha-1 Microglobulin - urine	
alpha-1-seromuroid	Orosomuroid - serum
Alpha-2 Antiplasmin - activity - plasma	
Alpha-2 Macroglobulin - serum	
Alpha-fetoprotein	AFP - Alpha fetoprotein - fluid
Alpha-fetoprotein	AFP - Alpha fetoprotein - serum
Alpha-fetoprotein - amniotic fluid	AFP - Alpha-fetoprotein - amniotic fluid
Alzheimer	TAU - PTAU - Amyloid proteins - LCR
Amaril virus	Yellow Fever - immunity test - serum
amarile	Yellow Fever - immunity test - serum
AMDA	Myosite Markers - serum
Amgevita	Adalimumab
AMH - Anti-Müllerian hormone	AMH - Anti-Müllerian hormone
AMH - Anti-Müllerian hormone	AMH - Anti-Müllerian hormone - Child
AMH - Anti-Müllerian hormone	
AMH - Anti-Müllerian hormone - Child	
Amibes	Gastrointestinal Panel - direct diagnosis - PCR - Stools
Amidolytic protein C assay	Protein C - activity - plasma
Amikacin - serum	
Amiklin®	Amikacin - serum
Amino acids - chromatography	
Amino acids - chromatography - urine	
Aminogram	Amino acids - chromatography
Aminogram	Amino acids - chromatography - urine
Aminolevulinate	delta-Aminolevulinic acid - urine
Aminophylline	Theophylline - serum
Amiodarone + metabolite - Serum	
Amisulpride - Serum	
Amitriptyline - Serum	
AML	Immunophenotyping of malignant hemopaties
Ammonaemia - plasma	
Ammonia	Ammonaemia - plasma
Amoebae - test	Amoebiasis - serology - serum
Amoebiasis - serology	Amoebiasis - serology - serum
Amoebiasis - serology - serum	
Amoeboids - cysts screening	Parasitology - microscopy
Amoxicillin - Serum	
Amphetamines - dosage - plasma	
Amphetamines - dosage - urines	
Amplifications / Copy number gains: ALK; ERBB2; MET; NRAS; PIK3CA; FGFR3; PDGFRA;	Circulating tumor DNA NGS Panel - Peripheral blood from Cell Free DNA tube : ambiant
Amsparity	Adalimumab
Amyloid beta-protein	TAU - PTAU - Amyloid proteins - LCR
Amylose héréditaire à transthyréline - Etude moléculaire du gène TTR par séquençage	
Anaerobic and aerobic bacteria - susceptibility testing	
Anafranil®	Clomipramine - Serum
Anaplasma phagocytophylum	Human ehrlichiosis - IgG/IgM serology - serum
Anaplasmosé	Human ehrlichiosis - IgG/IgM serology - serum
Andersen-Tawil syndrome (ATS)	Cardiogenetic panel or exome
Androstenediol	3-Alpha-Androstenediol Glucuronide - serum
Androstenediol glucuronide	3-Alpha-Androstenediol Glucuronide - serum
Androstanolone	DHT - Dihydrotestosterone - serum
Androstenedione - Delta-4	D4A - Delta-4-Androstenedione - serum
Aneurin	Vitamin B1 - Thiamin
Angelman / Prader-Willi Syndrome - SNRPN methylation	
Angelman / Prader-Willi Syndrome - uniparental disomy - mechanism identification - amniotic	

Angelman / Prader-Willi Syndrome - uniparental disomy - mechanism identification - EDTA	
Angioconvertase	ACE - Angiotensin Converting Enzyme
Angioconvertase	ECA - Enzyme de Conversion de l'Angiotensine (activité Buhlmann Alinity® Abbott) -
Angiostrongyliasis	Angiostrongylus cantonensis - serology - serum
Angiostrongylus cantonensis - serology - serum	
Angiotensin II - plasma	
Angiotensin-converting enzyme	ACE - Angiotensin Converting Enzyme
Angiotensin-converting enzyme	ACE - Angiotensin Converting Enzyme - BAL
Angiotensin-converting enzyme	ECA - Enzyme de Conversion de l'Angiotensine (activité Buhlmann Alinity® Abbott) -
Anion hole	Electrolytes stool - Stool
anisakiasis - Specific IgE and IgG4 - serum	
anisakis	anisakiasis - Specific IgE and IgG4 - serum
Anisakis	anisakiasis - Specific IgE and IgG4 - serum
Annexin V - IgG - anti-antibodies	Anti-annexin V antibodies - IgG - serum
Anti RNAt synthetase antibodies	Myosite Markers - serum
Anti - GD1a antibodies	Anti-glycolipid antibodies - CSF
Anti - GD1a antibodies	Anti-glycolipid antibodies - serum
Anti - GD2 antibodies	Anti-glycolipid antibodies - CSF
Anti - GD2 antibodies	Anti-glycolipid antibodies - serum
Anti - GD3 antibodies	Anti-glycolipid antibodies - CSF
Anti - GD3 antibodies	Anti-glycolipid antibodies - serum
anti 2 glycoproteine 1 antibodies IgG	Anti phospholipids antibodies - Serum
anti 2 glycoproteine 1 antibodies IgM	Anti phospholipids antibodies - Serum
Anti - GM2 antibodies	Anti-glycolipid antibodies - CSF
Anti - GM2 antibodies	Anti-glycolipid antibodies - serum
Anti - GM4 antibodies	Anti-glycolipid antibodies - CSF
Anti - GM4 antibodies	Anti-glycolipid antibodies - serum
Anti - GT1a antibodies	Anti-glycolipid antibodies - CSF
Anti - GT1a antibodies	Anti-glycolipid antibodies - serum
Anti - GT1b antibodies	Anti-glycolipid antibodies - CSF
Anti - GT1b antibodies	Anti-glycolipid antibodies - serum
Anti - PM1 antibodies	Myosite Markers - serum
Anti -GM3 antibodies	Anti-glycolipid antibodies - CSF
Anti -GM3 antibodies	Anti-glycolipid antibodies - serum
Anti actin antibodies - screening	Anti-smooth muscle antibodies - screening and titration - serum
Anti alanyl RNAt synthetase antibodies (PL12)	Myosite Markers - serum
Anti Aquaporin 4 antibodies	Anti-neuromyelitis optica antibodies - serum
Anti Calcium sensor antibodies	Anti-Calcium Sensing Receptor antibodies - serum
Anti Cardiolipin IgG antibodies	Anti phospholipids antibodies - Serum
anti cardiolipin IgM antibodies	Anti phospholipids antibodies - Serum
Anti CASPR1	Paranodes antibodies (anti- NF155, NF186, CNTN1, CASPR1) - serum
Anti cerebellum antibodies/ anti PCA2 antibodies	Anti-neuron antibodies - CSF
Anti cerebellum antibodies/ anti PCA2 antibodies	Anti-neuron antibodies - serum
Anti cholinesterase antibodies	Anti-acetylcholine receptor antibodies - serum
Anti CNTN1	Paranodes antibodies (anti- NF155, NF186, CNTN1, CASPR1) - serum
Anti CRPM5/CV2 antibodies	Anti-neuron antibodies - serum
Anti deamidated gliadin peptides antibodies IgA - IgA - serum	
Anti deamidated gliadin peptides antibodies IgA - IgG - serum	
Anti DPG	Anti deamidated gliadin peptides antibodies IgA - IgA - serum
Anti endocrin pancreas antibodies	Anti-islet cell antibodies - ICA - serum
anti GFAP antibodies - Repeat testing - CSF - anti GFAP antibodies	
Anti gluten IgA antibodies	Anti-endomysium antibodies - IgA - serum
Anti gluten IgA antibodies	Human tissue transglutaminase antibody - IgA - serum
Anti gluten IgG antibodies	Anti-endomysium antibodies - IgG - serum
Anti gluten IgG antibodies	Human tissue transglutaminase antibody - IgG - serum
Anti heterophyll mononucleose antibodies	Epstein-Barr Virus - EBV - MNI test - serum
Anti Histidyl RNAt synthetase (JO1)	Myosite Markers - serum
Anti HMGCR antibodies - serum	

Anti Ila activity measurement - Dabigatran Etexilate	Dabigatran - Measurement of the Dabigatran anti Ila activity - plasma
Anti inteferon alpha antibodies	Interferon alpha - neutralizing antibody - serum
Anti isoleucyl RNAt synthetase antibodies(OJ)	Myosite Markers - serum
Anti MDA-5/CADM 140 antibodies	Myosite Markers - serum
Anti native DNA antibodies	Anti-DNA antibodies Double stranded DNA - Serum
Anti nervous synaptics antibodies (anti- NMDAR, AMPAR1, AMPAR 2, Lgi1, Caspr2,	
Anti NF155	Paranodes antibodies (anti- NF155, NF186, CNTN1, CASPR1) - serum
Anti NF186	Paranodes antibodies (anti- NF155, NF186, CNTN1, CASPR1) - serum
Anti PDH antibodies	Anti-mitochondrial antibodies - typing - serum
Anti phospholipids antibodies - Serum	
Anti pyruvate dehydrogenase antibodies	Anti-mitochondrial antibodies - typing - serum
Anti receptor tyrosine kinase antibodies	Anti-MuSK antibodies - serum
Anti ribonucleoproteine antibodies	Anti-ENA antibodies (SSA, SSB, Sm, Rnp, Jo1, Scl-70) - screening and titration - Serum
anti spike antibodies	COVID-19 (SARS-CoV-2) IgG serology anti-S
Anti tetanus antibodies	Tetanus - immunity test - IgG serology - serum
Anti threonyl RNAt synthetase antibodies (PL7)	Myosite Markers - serum
anti THSD7A antibodies	
Anti TIF p155/1400 antibodies	Myosite Markers - serum
Anti tubular basement membrane antibodies	Anti-glomerular basement membrane antibody - Serum
Anti Type II bovin collagen antibodies	Anti-type II collagen antibodies - serum
Anti ZNT8 antibodies - serum	
anti- AMPAR	Anti nervous synaptics antibodies (anti- NMDAR, AMPAR1, AMPAR 2, Lgi1, Caspr2,
anti- GABAB	Anti nervous synaptics antibodies (anti- NMDAR, AMPAR1, AMPAR 2, Lgi1, Caspr2,
anti- GlycR1	Anti nervous synaptics antibodies (anti- NMDAR, AMPAR1, AMPAR 2, Lgi1, Caspr2,
anti- mGluR1	Anti nervous synaptics antibodies (anti- NMDAR, AMPAR1, AMPAR 2, Lgi1, Caspr2,
anti- mGluR5	Anti nervous synaptics antibodies (anti- NMDAR, AMPAR1, AMPAR 2, Lgi1, Caspr2,
anti- protein 7A of thrombospondine antibodies	anti THSD7A antibodies
Anti- SOX1 antibodies - CSF	
Anti- SOX1 antibodies - serum	
Anti-21-hydroxylase antibodies - serum	
Anti-acetylcholine receptor antibodies - serum	
Anti-actin antibodies -	Anti-smooth muscle antibodies - screening and titration - serum
Anti-adrenal gland antibodies	Anti-adrenal gland antibodies - serum
Anti-adrenal gland antibodies - serum	
Anti-AKT	DNA NGS Panel for prostate cancer
Anti-ALK	RNA NGS Panel for lung cancers
Anti-AMPAR (CSF)	CSF
Anti-amphiphysin antibodies	Anti-neuron antibodies - CSF
Anti-amphiphysin antibodies	Anti-neuron antibodies - serum
Anti-annexin V antibodies - IgG - serum	
Anti-AQP4 antibodies	Anti-neuromyelitis optica antibodies - serum
Anti-ASGPR: asialoglycoprotein receptor	Anti-asialoglycoprotein receptor antibodies - serum
Anti-asialoglycoprotein receptor antibodies - serum	
Anti-B2GP1 antibodies - IgG	Anti-beta 2 glycoprotein 1 antibodies - B2GP1 - IgG - Serum
Anti-B2GP1 antibodies - IgM	Anti-beta 2 glycoprotein 1 antibodies - B2GP1 - IgM - Serum
Anti-beta 2 glycoprotein 1 antibodies - B2GP1 - IgG - Serum	
Anti-beta 2 glycoprotein 1 antibodies - B2GP1 - IgM - Serum	
Anti-BPAG1 antibodies	Anti-epidermal basement membrane and intracellular substance antibodies Typing
Anti-BPAG2 antibodies	Anti-epidermal basement membrane and intracellular substance antibodies Typing
Anti-BRAF	DNA NGS Panel for cholangiocarcinoma
Anti-BRAF	DNA NGS Panel for thyroid cancer
Anti-C1q antibodies - serum	
Anti-calcium channel antibodies - serum	
Anti-Calcium Sensing Receptor antibodies - serum	
Anti-Calcium-Sensing Receptor antibodies	Anti-Calcium Sensing Receptor antibodies - serum
Anti-cardiolipin antibodies - IgG - Serum	
Anti-cardiolipin antibodies - IgM - Serum	
Anti-cartilage antibodies	Anti-type II collagen antibodies - serum

anti-Caspr2 antibodies	Anti-potassium channel antibodies / Lgi1 Caspr2 - CSF
anti-Caspr2 antibodies	Anti-potassium channel antibodies / Lgi1 Caspr2 - serum
anti-CaSR antibodies	Anti-Calcium Sensing Receptor antibodies - serum
Anti-CCP antibodies	Anti-cyclic citrullinated peptide antibodies - CCP2
Anti-Centromere antibodies (subunits A and B)	Markers in progressive systemic sclerosis(Scleroderma) - serum
Anti-citrulline antibodies	Anti-cyclic citrullinated peptide antibodies - CCP2
Anti-connectin antibodies	Anti-titin antibodies - serum
Anti-contactine	Paranodes antibodies (anti- NF155, NF186, CNTN1, CASPR1) - serum
Anti-CV2 antibodies	Anti-neuron antibodies - CSF
Anti-CV2 antibodies	Anti-neuron antibodies - serum
Anti-cyclic citrullinated peptide antibodies	Anti-cyclic citrullinated peptide antibodies - CCP2
Anti-cyclic citrullinated peptide antibodies - CCP2	
Anti-cytochrome P450 II D6 antibodies - screening	Anti-LKM antibodies - endoplasmic reticulum - screening - serum
Anti-cytochrome P450 II D6 antibodies - typing	Anti-LKM antibodies - endoplasmic reticulum - typing - serum
Anti-deamidated gliadin antibodies	Anti deamidated gliadin peptides antibodies IgA - IgA - serum
Anti-deamidated gliadin antibodies	Anti deamidated gliadin peptides antibodies IgA - IgG - serum
Anti-deamidated gliadin peptide antibodies	Anti deamidated gliadin peptides antibodies IgA - IgA - serum
Anti-deamidated gliadin peptide antibodies	Anti deamidated gliadin peptides antibodies IgA - IgG - serum
Anti-deamidated Gliadin Peptide Antibodies	Anti deamidated gliadin peptides antibodies IgA - IgA - serum
Anti-deaminated gliadin antibodies	Anti deamidated gliadin peptides antibodies IgA - IgG - serum
Anti-desmoglein 1 antibodies	Anti-epidermal basement membrane and intracellular substance antibodies Typing
Anti-desmoglein 3 antibodies	Anti-epidermal basement membrane and intracellular substance antibodies Typing
Anti-desmosome antibodies	Anti-epidermal basement membrane and intracellular substance antibodies - serum
Anti-diphtheria toxin antibodies	Diphtheria - immunity test - IgG serology - serum
Anti-DNA antibodies double stranded - Farr test - serum	
Anti-DNA antibodies Double stranded DNA - Serum	
anti-DNER	Anti-Tr antibodies - CSF
anti-DNER	Anti-Tr antibodies - serum
Anti-double strand DNA antibodies	Anti-DNA antibodies double stranded - Farr test - serum
Anti-DPG antibodies	Anti deamidated gliadin peptides antibodies IgA - IgG - serum
anti-DPPX (CSF)	CSF
Anti-dsDNA antibodies	Anti-DNA antibodies Double stranded DNA - Serum
Anti-E1 protective antibodies	Rubella - Western blot - serum
anti-EGFR	DNA NGS Panel solid tumor Colon
Anti-EGFR	DNA NGS Panel solid tumor Colon and MSI status determination
Anti-EGFR	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
Anti-EJ-antibodies	Myosite Markers - serum
Anti-ENA antibodies (SSA, SSB, Sm, Rnp, Jo1, Scl-70) - screening and titration - Serum	
Anti-endomysium antibodies - IgA - serum	
Anti-endomysium antibodies - IgG - serum	
Anti-endoplasmic reticulum antibodies - screening	Anti-LKM antibodies - endoplasmic reticulum - screening - serum
Anti-endoplasmic reticulum antibodies - typing	Anti-LKM antibodies - endoplasmic reticulum - typing - serum
Anti-epidermal basement membrane and intracellular substance antibodies - serum	
Anti-epidermal basement membrane and intracellular substance antibodies Typing (BPAg2/	
Anti-erythrocyte antibodies - screening/identification	Indirect antiglobulin test - IAT - screening/identification
Anti-factor II antibodies - IgG	Anti-prothrombin antibodies - IgG - serum
Anti-factor IX antibodies - screening - plasma	
Anti-Factor V antibodies - screening - plasma	
Anti-factor VII antibodies - screening - plasma	
Anti-factor VIII antibodies - screening - plasma	
Anti-factor XI antibodies - screening - plasma	
Anti-factor XII antibodies - screening - plasma	
Anti-FGFR2	RNA NGS Panel for cholangiocarcinoma
Anti-FI antibodies	Anti-intrinsic factor antibodies - serum
Anti-Fibrillarin antibodies	Markers in progressive systemic sclerosis(Scleroderma) - serum
Anti-filagrin antibodies	Anti-cyclic citrullinated peptide antibodies - CCP2
Anti-Francisella tularensis antibodies	Tularemia - Francisella tularensis - serology total antibodies - serum
Anti-GAD antibodies - CSF	

Anti-GAD antibodies - serum		
Anti-GAD65 antibodies		Anti-GAD antibodies - CSF
Anti-GAD65 antibodies		Anti-GAD antibodies - serum
Anti-ganglioside antibodies		Anti-glycolipid antibodies - CSF
Anti-ganglioside antibodies		Anti-glycolipid antibodies - serum
Anti-gastric cell antibodies		Anti-parietal cell antibodies - stomach - serum
Anti-GBM antibodies		Anti-glomerular basement membrane antibody - Serum
Anti-GD1b antibodies		Anti-glycolipid antibodies - CSF
Anti-GD1b antibodies		Anti-glycolipid antibodies - serum
anti-glial nuclear antibodies		Anti- SOX1 antibodies - CSF
anti-glial nuclear antibodies		Anti- SOX1 antibodies - serum
Anti-glomerular basement membrane antibody		Anti-epidermal basement membrane and intracellular substance antibodies - serum
Anti-glomerular basement membrane antibody - Serum		
Anti-glycolipid antibodies - CSF		
Anti-glycolipid antibodies - serum		
Anti-GM1 antibodies		Anti-glycolipid antibodies - CSF
Anti-GM1 antibodies		Anti-glycolipid antibodies - serum
Anti-GP210 antibodies - serum		
Anti-GQ1b antibodies		Anti-glycolipid antibodies - CSF
Anti-GQ1b antibodies		Anti-glycolipid antibodies - serum
Anti-H+ K+ ATPase antibodies		Anti-proton pump antibodies - serum
Anti-HA abs		Influenza A and B - IgG serology - serum
Anti-HBc IgG antibodies		Hepatitis B - anti-HBc antibodies - IgM - serum
Anti-HDV IgM antibodies		Hepatitis Delta - IgM antibody - serum
Anti-heparin antibodies		Anti-PF4 antibodies - heparin - plasma
Anti-hepatitis C antibodies - screening test		Hepatitis C - screening serology - serum
Anti-HER2		DNA NGS Panel for cholangiocarcinoma
Anti-HER2		DNA NGS Panel solid tumor Breast
Anti-HEV antibodies - IgG		Hepatitis E - IgG antibody - serum
Anti-HEV IgM antibodies		Hepatitis E - IgM antibody - serum
Anti-Hib antibodies		Haemophilus influenzae B - IgG serology - serum
Anti-histone antibodies - serum		
anti-HLA antibodies - classes I and II - transplant - serum		
Anti-HMG-CoA reductase Antibody		Anti HMGCR antibodies - serum
Anti-Hu antibodies		Anti-neuron antibodies - CSF
Anti-Hu antibodies		Anti-neuron antibodies - serum
Anti-IA-2 antibodies		Anti-tyrosine phosphatase antibodies - IA-2 - serum
Anti-IDH1		DNA NGS Panel for cholangiocarcinoma
Anti-IgA antibodies - serum		
Anti-iGluR2		Anti-NMDA receptor antibodies - CSF
Anti-immunoglobulin A antibodies		Anti-IgA antibodies - serum
anti-Infliximab antibodies		Infliximab
Anti-insulin antibodies - serum		
Anti-integrase agents		HIV - Human Immunodeficiency Virus - genotyping (reverse transcriptase, protease and
Anti-intrinsic factor antibodies - serum		
Anti-islet cell antibodies - ICA - serum		
Anti-Jo1 antibodies		Anti-ENA antibodies (SSA, SSB, Sm, Rnp, Jo1, Scl-70) - screening and titration - Serum
Anti-keratin antibodies		Anti-cyclic citrullinated peptide antibodies - CCP2
Anti-Ku antibodies		Markers in progressive systemic sclerosis(Scleroderma) - serum
Anti-Ku antibodies		Myosite Markers - serum
Anti-lactoferrin antibodies and anti carbonic anhydrase antibodies - serum		
Anti-leukoplatelet antibodies		anti-HLA antibodies - classes I and II - transplant - serum
Anti-Leydig cell antibodies		Anti-testicular antibodies - serum
anti-Lg1 antibodies		Anti-potassium channel antibodies / Lgi1 Caspr2 - CSF
anti-Lgi1 antibodies		Anti-potassium channel antibodies / Lgi1 Caspr2 - serum
Anti-liver cytosol type 1 antibodies - LC1 - confirmation - serum		
Anti-liver cytosol type 1 antibodies - LC1 - screening - serum		
Anti-liver-kidney microsomal antibodies - LKM1 - screening		Anti-LKM antibodies - endoplasmic reticulum - screening - serum

Anti-liver-kidney microsomal antibodies - LKM1 - typing	Anti-LKM antibodies - endoplasmic reticulum - typing - serum
Anti-LKM1 antibodies - screening	Anti-LKM antibodies - endoplasmic reticulum - screening - serum
Anti-LKM1 antibodies - typing	Anti-LKM antibodies - endoplasmic reticulum - typing - serum
Anti-LKM antibodies - endoplasmic reticulum - screening - serum	
Anti-LKM antibodies - endoplasmic reticulum - typing - serum	
Anti-LP/SLA antibodies	Anti-SLA antibodies - Soluble Liver Antigen - serum
Anti-M2 - screening and titration	Anti-mitochondrial antibodies - screening and titration - serum
Anti-M2 antibodies	Anti-mitochondrial antibodies - typing - serum
Anti-M2 antibodies - typing	Anti-mitochondrial antibodies - typing - serum
Anti-Ma2 antibodies	Anti-neuron antibodies - CSF
Anti-Ma2 antibodies	Anti-neuron antibodies - serum
Anti-MAG antibodies - serum	
Anti-MEK	DNA NGS Panel for cholangiocarcinoma
Anti-MET	RNA NGS Panel for lung cancers
Anti-MGT30 antibodies	Anti-titin antibodies - serum
Anti-Mi2 antibodies	Myosite Markers - serum
Anti-microcosmal thyroidian antibodies	Anti-thyroid peroxidase antibodies
Anti-mitochondrial antibodies - screening and titration - serum	
Anti-mitochondrial antibodies - typing - serum	
Anti-Morbillivirus antibodies - IgG/IgM	Measles - IgG/IgM serology - serum
Anti-MPO antibodies - myeloperoxidase - Serum	
Anti-Müllerian hormone	AMH - Anti-Müllerian hormone
Anti-Müllerian hormone	AMH - Anti-Müllerian hormone - Child
Anti-MuSK antibodies - serum	
Anti-myelin antibodies - serum	
Anti-myelin-associated glycoprotein antibodies	Anti-MAG antibodies - serum
Anti-myeloperoxidase antibodies	Anti-MPO antibodies - myeloperoxidase - Serum
Anti-Myosin antibodies	Anti-striated muscle antibodies - serum
anti-N	COVID-19 (SARS-CoV-2) serology (total Ig) anti-N
Anti-N-methyl-D-aspartate receptor antibodies	Anti-NMDA receptor antibodies - serum
Anti-Natalizumab antibodies - serum	
Anti-NC16A antibodies	Anti-epidermal basement membrane and intracellular substance antibodies Typing
Anti-neurofascine	Paranodes antibodies (anti- NF155, NF186, CNTN1, CASPR1) - serum
Anti-neuromuscular junction antibodies	Anti-acetylcholine receptor antibodies - serum
Anti-neuromyelitis optica antibodies - CSF	
Anti-neuromyelitis optica antibodies - serum	
Anti-neuron antibodies - CSF	
Anti-neuron antibodies - serum	
Anti-NMDA receptor antibodies - CSF	
Anti-NMDA receptor antibodies - serum	
Anti-NMO antibodies	Anti-neuromyelitis optica antibodies - serum
Anti-node of Ranvier glycoproteins	Paranodes antibodies (anti- NF155, NF186, CNTN1, CASPR1) - serum
Anti-NOR 90 antibodies	Markers in progressive systemic sclerosis(Scleroderma) - serum
Anti-NR2 glutamate receptors antibodies	Anti-NMDA receptor antibodies - CSF
Anti-NR2 (NMDA receptor)	Anti-NMDA receptor antibodies - CSF
anti-nucleocapsid	COVID-19 (SARS-CoV-2) serology (total Ig) anti-N
Anti-nucleosome antibodies - serum	
Anti-NXP2 (p140/MJ) antibodies	Myosite Markers - serum
Anti-OJ-antibodies	Myosite Markers - serum
Anti-ovarian antibodies - serum	
Anti-parietal cell antibodies	Anti-parietal cell antibodies - stomach - serum
Anti-parietal cell antibodies - stomach - serum	
Anti-PARP	DNA NGS Panel for pancreatic cancer
Anti-PARP	DNA NGS Panel for prostate cancer
Anti-PDGFR antibodies	Markers in progressive systemic sclerosis(Scleroderma) - serum
Anti-PE antibodies - IgG	Anti-phosphatidylethanolamine antibodies - IgG - serum
Anti-PE antibodies - IgM	Anti-phosphatidylethanolamine antibodies - IgM - serum
Anti-perinuclear antibodies	Anti-cyclic citrullinated peptide antibodies - CCP2

Anti-PF4 antibodies - heparin - plasma	
Anti-phosphatidylethanolamine antibodies - IgG - serum	
Anti-phosphatidylethanolamine antibodies - IgM - serum	
Anti-phospholipase A2	anti-PLA2R antibodies
anti-phospholipase A2 receptor antibodies	anti-PLA2R antibodies
Anti-phospholipides antibodies	Anti-cardiolipin antibodies - IgG - Serum
Anti-phospholipides antibodies	Anti-cardiolipin antibodies - IgM - Serum
Anti-PL12 antibodies	Myosite Markers - serum
Anti-PL7 antibodies	Myosite Markers - serum
anti-PLA2R antibodies	
Anti-platelet antibodies bound or free - screening and identification	
Anti-platelet factor 4 antibodies	Anti-PF4 antibodies - heparin - plasma
Anti-PM100-antibodies	Myosite Markers - serum
Anti-PM-Scl antibodies	Myosite Markers - serum
Anti-PM75-antibodies	Myosite Markers - serum
Anti-PM/Scl antibodies (subunits 75 and 100)	Markers in progressive systemic sclerosis(Scleroderma) - serum
Anti-potassium channel antibodies / Lgi1 Caspr2 - CSF	
Anti-potassium channel antibodies / Lgi1 Caspr2 - serum	
Anti-PR3 antibodies - proteinase 3 - Serum	
Anti-profilaggrin antibodies	Anti-cyclic citrullinated peptide antibodies - CCP2
Anti-proteinase 3 antibodies	Anti-PR3 antibodies - proteinase 3 - Serum
Anti-prothrombin antibodies - IgG - serum	
Anti-proton pump antibodies - serum	
Anti-rabies antibodies	Rabies - IgG serology - serum
Anti-recoverin antibodies	Anti-retina antibodies - serum
Anti-RET	RNA NGS Panel for lung cancers
Anti-RET	RNA NGS Panel for thyroid cancer
Anti-retina antibodies - serum	
anti-rGABAb (CSF)	CSF
Anti-Ri antibodies	Anti-neuron antibodies - CSF
Anti-Ri antibodies	Anti-neuron antibodies - serum
Anti-ribosome antibodies - serum	
Anti-Rickettsia conorii, typhi antibodies - IgG	Rickettsioses - R. conorii, R. typhi - serology - serum
Anti-RNA polymerase III antibodies (subunits RP 11 and 155)	Markers in progressive systemic sclerosis(Scleroderma) - serum
Anti-Rnp antibodies	Anti-ENA antibodies (SSA, SSB, Sm, Rnp, Jo1, Scl-70) - screening and titration - Serum
Anti-Ro antibodies	Anti-ENA antibodies (SSA, SSB, Sm, Rnp, Jo1, Scl-70) - screening and titration - Serum
Anti-ROS1	RNA NGS Panel for lung cancers
anti-S	COVID-19 (SARS-CoV-2) IgG serology anti-S
Anti-Saccharomyces cerevisiae antibodies - IgG/IgA - serum	
Anti-SAE1 antibodies	Myosite Markers - serum
Anti-SAE2 antibodies	Myosite Markers - serum
anti-SARS-CoV-2 spike IgG antibodies	COVID-19 (SARS-CoV-2) IgG serology anti-S
[Anti-Scl-70 antibodies	Markers in progressive systemic sclerosis(Scleroderma) - serum
Anti-Scl-70 antibodies	Anti-ENA antibodies (SSA, SSB, Sm, Rnp, Jo1, Scl-70) - screening and titration - Serum
Anti-SLA antibodies - Soluble Liver Antigen - serum	
Anti-SLA/LP antibodies	Anti-SLA antibodies - Soluble Liver Antigen - serum
Anti-Sm antibodies	Anti-ENA antibodies (SSA, SSB, Sm, Rnp, Jo1, Scl-70) - screening and titration - Serum
Anti-smooth muscle antibodies - screening and titration - serum	
Anti-SNA antibodies	Anti-ENA antibodies (SSA, SSB, Sm, Rnp, Jo1, Scl-70) - screening and titration - Serum
Anti-sperm antibodies - indirect screening	
Anti-sphingolipides antibodies	Anti-glycolipid antibodies - serum
Anti-SRP antibodies	Myosite Markers - serum
Anti-SSA, anti-SSB antibodies	Anti-ENA antibodies (SSA, SSB, Sm, Rnp, Jo1, Scl-70) - screening and titration - Serum
Anti-streptococcal enzyme antibodies - ASLO	Streptococcus A - Serology- Ac anti streptolysin O (ASLO) - serum
Anti-streptodornase antibodies	Streptococcus A- Serology-Abs to streptodornase B (ASD) - serum
Anti-streptolysin-O antibodies	Streptococcus A - Serology- Ac anti streptolysin O (ASLO) - serum
Anti-striated muscle antibodies - serum	
Anti-sulfatides antibodies	Anti-glycolipid antibodies - CSF

Anti-sulfatides antibodies	Anti-glycolipid antibodies - serum
Anti-T3 antibodies - triiodothyronine - serum	
Anti-T4 antibodies - thyroxine - serum	
Anti-testicular antibodies - serum	
Anti-tetanus antibodies - vaccination control	Tetanus - immunity test - IgG serology - serum
Anti-Tg antibodies	Anti-thyroglobulin antibodies
Anti-TH/TO antibodies	Markers in progressive systemic sclerosis(Scleroderma) - serum
Anti-thyroglobulin antibodies	
Anti-thyroid peroxidase antibodies	
Anti-thyroidean antibodies	Anti-thyroglobulin antibodies
Anti-thyroidean antibodies	Anti-thyroid peroxidase antibodies
Anti-thyroxine antibodies	Anti-T4 antibodies - thyroxine - serum
Anti-titin antibodies - CSF	
Anti-titin antibodies - serum	
Anti-topoisomerase antibodies - screening	Anti-ENA antibodies (SSA, SSB, Sm, Rnp, Jo1, Scl-70) - screening and titration - Serum
Anti-Toxoplasma gondii antibodies - IgG/IgM	Toxoplasmosis - IgG/IgM serology - serum
Anti-TPO antibodies	Anti-thyroid peroxidase antibodies
Anti-Tr antibodies - CSF	
Anti-Tr antibodies - serum	
Anti-Trichinella spiralis antibodies - confirmation serology	Trichinosis - confirmation serology - serum
Anti-Trichinella spiralis antibodies - IgG serology	Trichinosis - IgG serology - serum
Anti-triiodothyronine antibodies	Anti-T3 antibodies - triiodothyronine - serum
Anti-TRK	RNA NGS Panel for cholangiocarcinoma
Anti-TRK	RNA NGS Panel for pancreatic cancer
Anti-TRK	RNA NGS Panel for thyroid cancer
Anti-IRNA synthetase antibodies	Myosite Markers - serum
Anti-Trypanosoma brucei gambiense antibodies	Trypanosoma gambiense - serology - CSF
Anti-Trypanosoma brucei gambiense antibodies	Trypanosoma gambiense - serology - serum
Anti-Trypanosoma cruzi antibodies	Trypanosoma cruzi - serology - serum
Anti-TSH receptor antibodies	
Anti-type II collagen antibodies - serum	
Anti-tyrosine phosphatase antibodies - IA-2 - serum	
Anti-U1RNP antibodies	Anti-ENA antibodies (SSA, SSB, Sm, Rnp, Jo1, Scl-70) - screening and titration - Serum
anti-Védolizumab	Vedolizumab - serum
Anti-VGCC antibodies	Anti-calcium channel antibodies - serum
anti-VGKC antibodies	Anti-potassium channel antibodies / Lgi1 Caspr2 - CSF
Anti-VGKC antibodies	Anti-potassium channel antibodies / Lgi1 Caspr2 - serum
Anti-VZV antibodies - IgM	Varicella zoster virus - IgM Serology - serum
Anti-Xa	Anti-Xa activity : Standard heparin - NFH - plasma
Anti-Xa - Arixtra®	Anti-Xa activity : Fondaparinux - Arixtra - plasma
Anti-Xa - Danaparoid	Anti-Xa activity : Danaparoid - Orgaran - plasma
Anti-Xa - LMWH	Anti-Xa activity : low molecular weight heparin - LMWH - plasma
Anti-Xa - Orgaran®	Anti-Xa activity : Danaparoid - Orgaran - plasma
Anti-Xa activity : Danaparoid - Orgaran - plasma	
Anti-Xa activity : Fondaparinux - Arixtra - plasma	
Anti-Xa activity : low molecular weight heparin - LMWH - plasma	
Anti-Xa activity : Standard heparin - NFH - plasma	
Anti-Xa activity - Arixtra®	Anti-Xa activity : Fondaparinux - Arixtra - plasma
Anti-Xa activity - low molecular weight heparin	Anti-Xa activity : low molecular weight heparin - LMWH - plasma
Anti-Xa activity - Orgaran®	Anti-Xa activity : Danaparoid - Orgaran - plasma
Anti-Yersinia Outer Protein antibodies	Anti-YOP antibodies - IgG - serum
Anti-Yo antibodies	Anti-neuron antibodies - CSF
Anti-Yo antibodies	Anti-neuron antibodies - serum
Anti-YOP antibodies - IgG - serum	
Anti-ZC4 antibodies - CSF	
Anti-ZC4 antibodies - serum	
Anti-zinc transporter protein antibodies	Anti ZNT8 antibodies - serum
Anti-zona pellucida antibodies	Anti-ovarian antibodies - serum

Antibiogramme	Anaerobic and aerobic bacteria - susceptibility testing
Antibiotic multi-resistant bacteria (MRB) - screening	
Antibody	DNA NGS Panel solid tumor Breast
antibody avidity - rubella	Rubella - antibody avidity - IgG - serum
anticorps anti-adalimumab	Adalimumab
Anticorps anti-Ustekinumab	Ustekinumab - serum
Antidepressants - screening - identification - dosage - urine	
Antidepressants screening- identification - without dosage	
Antidiuretic hormone	ADH - Antidiuretic hormone
Antidiuretic hormone	ADH - Antidiuretic hormone - urine
Antigen - Helicobacter pylori	Helicobacter pylori - antigen - Stools
Antigène P24	HIV-1: Qualitative RNA detection, suspected primary infection - serum
Antigènes leucoplaquettaires HLA de classe 1	HLA-C genotyping (HLA de classe I) - Total EDTA blood
Antihaemophilic A - factor	Factor VIII - Antihaemophilic A - plasma
Antihaemophilic B - factor	Factor IX - Antihaemophilic B - plasma
Antihaemophilic factor A	Factor VIII - Antihaemophilic A - plasma
Antihaemophilic factor B	Factor IX - Antihaemophilic B - plasma
Antihaemorrhagic factor	Vitamin K1 - Phyloquinone
Antiplasmin - Alpha 2	Alpha-2 Antiplasmin - activity - plasma
Antithrombin - activity	Antithrombin - functional activity - plasma
Antithrombin - antigen - plasma	
Antithrombin - antigen or quantitation	Antithrombin - antigen - plasma
Antithrombin - functional activity - plasma	
AP50	50% haemolysis time - alternative pathway complement - serum
Aparoxal	Phenobarbital - serum
APC	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
APC	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
APC virus	Adenovirus - IgG serology - serum
APC virus	Adenovirus - screening - Stools
APCAM	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
Apixaban(Eliquis ®) - Anti Xa activity measurement - plasma	
Apo A1	Apolipoprotein A1 - serum
Apo C3	Apolipoprotein C3 - serum
ApoB	Apolipoprotein B - serum
Apolipoprotein A1 - serum	
Apolipoprotein B - serum	
Apolipoprotein C3 - serum	
APTT	Activated Partial Thromboplastin Time - aPTT - plasma
APTT , activated partial thromboplastin time	Activated Partial Thromboplastin Time - aPTT - plasma
AR	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Arbovirosis	Tick-borne encephalitis - TBE - IgG/IgM serology - serum
Arbovirus	Dengue fever - direct diagnosis - PCR
Arbovirus	Dengue fever - IgG/IgM serology - serum
Arginine - Vasopressin	ADH - Antidiuretic hormone
Arginine - Vasopressin	ADH - Antidiuretic hormone - urine
Aripiprazole	
Arixtra® - anti-Xa	Anti-Xa activity : Fondaparinux - Arixtra - plasma
ARN ribosomal 16S	Detection and identification bacterial DNA - 16S PCR by NGS sequencing
Arrhythmogenic cardiomyopathy	Cardiogenetic panel or exome
Arrhythmogenic Right Ventricular Dysplasia	Cardiogenetic panel or exome
Arsenic - inorganic an metabolites derivatives - urine	
ASCA	Anti-Saccharomyces cerevisiae antibodies - IgG/IgA - serum
Ascariasis - serology - serum	
Ascaris	Ascariasis - serology - serum
ASD	Streptococcus A- Serology-Abs to streptodornase B (ASD) - serum
ASDB	Streptococcus A- Serology-Abs to streptodornase B (ASD) - serum
ASDOR	Streptococcus A- Serology-Abs to streptodornase B (ASD) - serum
AshTest	FibroMAX - . - serum

AshTest®	NASH-FibroTest - serum
Asialotransferrin	Carbohydrate-deficient transferrin - serum
ASLO	Streptococcus A - Serology- Ac anti streptolysin O (ASLO) - serum
ASMA	Anti-smooth muscle antibodies - screening and titration - serum
Aspergillose invasiv	Aspergillosis - soluble antigen - testing - serum
Aspergillosis	Fungus - Yeasts - MIC: Minimum Inhibitory Concentration
Aspergillosis	Fungus - Yeasts - screening
Aspergillosis - confirmation serology - serum	
Aspergillosis - soluble antigen - testing - BAL	
Aspergillosis - soluble antigen - testing - serum	
Aspergillosis IgG serology screening - serum	
Aspergillosis precipitins	Aspergillosis IgG serology screening - serum
Aspergillus	Fungus - Yeasts - MIC: Minimum Inhibitory Concentration
Aspergillus	Fungus - Yeasts - screening
Aspergillus flavus	Aspergillosis - confirmation serology - serum
Aspergillus flavus	Aspergillosis IgG serology screening - serum
Aspergillus fumigatus	Aspergillosis - confirmation serology - serum
Aspergillus fumigatus	Aspergillosis IgG serology screening - serum
Aspergillus nidulans	Aspergillosis - confirmation serology - serum
Aspergillus nidulans	Aspergillosis IgG serology screening - serum
Aspergillus niger	Aspergillosis - confirmation serology - serum
Aspergillus niger	Aspergillosis IgG serology screening - serum
Aspergillus precipitins	Aspergillosis - confirmation serology - serum
Aspergillus terreus	Aspergillosis IgG serology screening - serum
Aspergillus versicolor	Aspergillosis - confirmation serology - serum
Aspergillus versicolor	Aspergillosis IgG serology screening - serum
Astrocytoma	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
Astrovirus	Gastrointestinal Panel - direct diagnosis - PCR - Stools
Astrovirus/Sapovirus	Gastrointestinal Panel - direct diagnosis - PCR - Stools
AT	Antithrombin - functional activity - plasma
AT - antigen or quantitation	Antithrombin - antigen - plasma
ATF1	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Athymil®	Mianserin - Serum
ATM	DNA NGS Panel for prostate cancer
ATM	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
ATM	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
ATM	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
ATM	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
ATP1B1	RNA NGS Panel solid tumors, Pan-organ : 65 genes
attachment inhibitor	HIV - Human Immunodeficiency Virus - genotyping (envelope, tropism): antiretroviral
atypic	Mycobactéria - identification
Australia Antigen	Hepatitis B - Qualitative HBs antigen - serum
Autoimmune pancreatitis	Anti-lactoferrin antibodies and anti carbonic anhydrase antibodies - serum
Autosomal dominant tubulointerstitial nephropathy	MUC1 nanopore Nephropathie
Avidity index of anti-Toxoplasma gondii IgG	Toxoplasmosis - antibody avidity - IgG - serum
Avidity of anti-Toxoplasma gondii IgG	Toxoplasmosis - antibody avidity - IgG - serum
Avlocardyl®	Propranolol - Serum
Axepim®	Cefepime - Serum
Axerophtol	Vitamin A - Retinol - Plasma
B19 - IgG serology	Parvovirus B19 - immunity test - serum
B19 - IgG/IgM serology	Parvovirus B19 - IgG/IgM serology - serum
B factor	Factor B - complement fraction - serum
B-ALP	Alkaline phosphatases bone - serum
B-AP	Alkaline phosphatases bone - serum
B2M	Beta-2 microglobulin - CSF
B2M	Beta-2 microglobulin - serum
B2M	Beta-2 microglobulin - urine
B-type natriuretic peptide	NT-proBNP - N-Terminal Pro-Brain Natriuretic Peptide - serum

Babesia	Human babesiosis - serology - serum
Bacillus cereus	Cereulide toxin - stool
Bacteria - screening	
Bactérie aérobie	Anaerobic and aerobic bacteria - susceptibility testing
Bactérie anaérobie	Anaerobic and aerobic bacteria - susceptibility testing
BAP1	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
BAP1	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
BAP1	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
BAP1	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
Baraclude	Entecavir - plasma
Barbiturate	Phenobarbital - serum
Barbituremia (barbiturate levels).	Phenobarbital - serum
Bartonella	Bartonellosis - B. henselae - B. quintana - IgG serology - serum
Bartonella	Bartonellosis - direct diagnosis - PCR
Bartonelle	Bartonellosis - direct diagnosis - PCR
Bartonellosis - B. henselae - B. quintana - IgG serology - serum	
Bartonellosis - direct diagnosis - PCR	
basedow disease	Anti-TSH receptor antibodies
Batten disease	Lipofuscinosis Neuronal Ceroide enzymatic deficiency - EDTA whole blood
BCHE	Pharmacogenetic Passport - PharmacPass Expert
BCOR	RNA NGS Panel for soft tissue tumors : 20 genes
BCOR	RNA NGS Panel solid tumors, Pan-organ : 65 genes
BCR-ABL mutation	tyrosine kinase ABL mutation
Béagyne®	Fluconazole - Serum
Behçet's disease B51 (B5) retinitis birdshot A29	Class I HLA - HLA-A and HLA-B genotyping - EDTA whole blood
Bence Jones - screening	
Bence Jones protein	Bence Jones - screening
Bence Jones proteinuria	Bence Jones - screening
Benepali ®	Etanercept - serum
Benzo	Benzodiazepines - screening without identification - Serum
Benzodiazepines - identification - urine	
Benzodiazepines - screening and identification	
Benzodiazepines - screening without identification - Serum	
Benzodiazepines - screening without identification - urine	
Benzoylecgonine	Cocaine - dosage - plasma
Benzoylecgonine	Cocaine - dosage - urines
Beta 2 GP1 - anti-antibodies	Anti-beta 2 glycoprotein 1 antibodies - B2GP1 - IgG - Serum
Beta 2 GP1 - anti-antibodies	Anti-beta 2 glycoprotein 1 antibodies - B2GP1 - IgM - Serum
Beta Cross-Laps	CrossLaps - Serum
Beta-2 microglobulin - CSF	
Beta-2 microglobulin - serum	
Beta-2 microglobulin - urine	
Beta-carotene	
Beta-D-Glucan antigen - screening	
Beta-hCG - free chain	hCG - free beta chain - subunit - serum
Beta-hCG - holo-hCG	hCG - free beta chain - subunit - serum
beta-Hydroxybutyric acid	Ketone bodies - Deproteinized heparin whole blood sample or CSF
bêta-lactamase à spectre étendu	Antibiotic multi-resistant bacteria (MRB) - screening
Beta-subunit - Free hCG	hCG - free beta chain - subunit - serum
Betahydroxybutyrate	Ketone bodies - Deproteinized heparin whole blood sample or CSF
Bevacizumab - serum	
BGP	Osteocalcin - Serum
BICC1	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Bictegravir - Serum	
Big Big prolactin	Prolactin - molecular forms - serum
Big prolactin	Prolactin - molecular forms - serum
Biktarvy	Bictegravir - Serum
Bile acids - chromatography - serum	
Eurofins Biomnis – International Division – Test guide Specialised medical pathology – Valid from : 01/08/2026	

Bile acids - serum	
Bile salts	Bile acids - chromatography - serum
Bile salts	Bile acids - serum
bilharzia	Schistosomiasis - direct diagnosis - PCR
biliarys lithiasis	Gallstones/Saliva/Other - morphologic and infrared analysis
biliarys stone	Gallstones/Saliva/Other - morphologic and infrared analysis
Bilirubin - total - free - combined - serum	
Bioactive testosterone	Testosterone bioavailability - serum
Biotin	Vitamin B8 or H - Biotin
Biperiden - serum	
Bispecific anti-HER2/HER3 antibodies	RNA NGS Panel for pancreatic cancer
Bisphenol	Bisphenols - urines - Bisphenols
Bisphenol A -BPA	Bisphenols - urines - Bisphenols
Bisphenol F -BPF	Bisphenols - urines - Bisphenols
Bisphenol S -BPS	Bisphenols - urines - Bisphenols
Bisphenol Z -BPZ	Bisphenols - urines - Bisphenols
Bisphenols - urines - Bisphenols	
BK	Mycobacteria - PCR M. tuberculosis complex (BKBM)
BK - culture	Mycobacteria - culture
BK - direct test	Mycobacteria - direct examination
BK - identification	Mycobactéria - identification
BK - susceptibility testing	Mycobacteria - susceptibility testing
BK polyomavirus - molecular diagnosis	Polyomavirus BK - viral load
BK polyomavirus - PCR	Polyomavirus BK - viral load
BKV polyomavirus	Polyomavirus BK - viral load
Black fever	Leishmaniasis - direct diagnosis - PCR
Blast cell phenotype	Immunophenotyping of malignant hemopaties
Blasts	Immunophenotyping of malignant hemopaties
Blitzima®	Rituximab - serum
Blood alcohol content	Ethyl alcohol - serum
Blood group determination	ABO blood grouping - RH1(D)
blood grouping - genotyping - EDTA whole blood	
Blood karyotype	
blot test	Full Neonatal screen - . - blotting paper
BMPR1A	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
BMPR1A	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
BNP - Brain Natriuretic Peptide - Plasma	
BNP precursor	NT-proBNP - N-Terminal Pro-Brain Natriuretic Peptide - serum
Bocavirus	Respiratory Infection Panel - direct diagnosis - PCR
Bone ALPs	Alkaline phosphatases bone - serum
Bone Gla Protein	Osteocalcin - Serum
Bone GLA protein	Osteocalcin - Serum
Bone-specific alkaline phosphatase	Alkaline phosphatases bone - serum
Bordet-Gengou Bacillus	Whooping cough - Bordetella pertussis and parapertussis - direct diagnosis - PCR
Bordetella pertussis	Pertussis - serology
Bordetella pertussis	Respiratory Infection Panel - direct diagnosis - PCR
Bordetella pertussis	Whooping cough - Bordetella pertussis and parapertussis - direct diagnosis - PCR
Borrelia	Borreliosis - direct diagnosis - PCR
Borrelia	Borreliosis - IgG confirmation serology
Borrelia	Borreliosis - IgM confirmation serology - serum
Borrelia	Lyme disease - IgG/IgM screening serology - serum
Borrelia	Lyme disease - IgG intrathecal index - CSF + serum
Borrelia	Lyme disease - screening serology - CSF
Borreliosis - direct diagnosis - PCR	
Borreliosis - IgG confirmation serology	
Borreliosis - IgM confirmation serology - serum	
bound testosterone	
Bound testosterone	Testosterone - man
Eurofins Biomnis – International Division – Test guide Specialised medical pathology – Valid from : 01/08/2026	

Bound testosterone	Testosterone - woman/child - serum
bowel	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
BP180	Anti-epidermal basement membrane and intracellular substance antibodies Typing
BP180 - anti-antibodies	Anti-epidermal basement membrane and intracellular substance antibodies Typing
bp180 - antibodies	Anti-epidermal basement membrane and intracellular substance antibodies Typing
BP230	Anti-epidermal basement membrane and intracellular substance antibodies Typing
BP230 - anti-antibodies	Anti-epidermal basement membrane and intracellular substance antibodies Typing
BPAg1	Anti-epidermal basement membrane and intracellular substance antibodies Typing
BPAG1 - anti-antibodies	Anti-epidermal basement membrane and intracellular substance antibodies Typing
BPAG2 - anti-antibodies	Anti-epidermal basement membrane and intracellular substance antibodies Typing
BRAF	DNA NGS Panel for brain solid tumors
BRAF	DNA NGS Panel for cholangiocarcinoma
BRAF	DNA NGS Panel for thyroid cancer
BRAF	DNA NGS Panel solid tumor Colon
BRAF	RNA NGS panel for brain solid tumors
BRAF	RNA NGS Panel for thyroid cancer
BRAF	RNA NGS Panel solid tumors, Pan-organ : 65 genes
BRAF, KRAS, NRAS, POLE, ERBB2, TP53, PIK3CA, MLH1, GNAS, PTEN, TERT, CDKN2A,	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
Brain Natriuretic Peptide	BNP - Brain Natriuretic Peptide - Plasma
Brain RNA-seq	RNA NGS panel for brain solid tumors
Brain tumor NGS panel	DNA NGS Panel for brain solid tumors
Brain tumors	RNA NGS panel for brain solid tumors
Brain tumors	RNA NGS Panel solid tumors, Pan-organ : 65 genes
BRB	Bilirubin - total - free - combined - serum
BRCA1	DNA NGS Panel for pancreatic cancer
BRCA1	DNA NGS Panel for prostate cancer
BRCA1	DNA NGS Panel solid tumor Breast
BRCA1	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
BRCA1	Oncogenetic exome
BRCA1	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
BRCA1	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
BRCA1	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
BRCA1, BRCA2	Séquencage des gènes BRCA1 et BRCA2
BRCA1 - BRCA2 - BRCA1/2 - POLE - TP53	DNA NGS Panel solid tumor Endometrium
BRCA1 - BRCA2 - BRCA1/2 - POLE - TP53 - MSI - MSS	DNA NGS Panel solid tumor Endometrium and MSI status determination
BRCA2	DNA NGS Panel for pancreatic cancer
BRCA2	DNA NGS Panel for prostate cancer
BRCA2	DNA NGS Panel solid tumor Breast
BRCA2	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
BRCA2	Oncogenetic exome
BRCA2	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
BRCA2	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
BRCA2	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
BRD3	RNA NGS Panel solid tumors, Pan-organ : 65 genes
BRD4	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Breast	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
BREAST	Oncogenetic exome
Breast cancer	DNA NGS Panel solid tumor Breast
Breast Cancer Prognostic Gene Signature Assay - PROSIGNA TEST	
Breast NGS panel	DNA NGS Panel solid tumor Breast
Breasy	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
Breath test	Helicobacter pylori - 13C urea breath test - expired gas
Brintellix®	Vortioxétine - Serum
Bristopen®	Oxacillin - Serum
Brivaracetam - Serum	
Briviact	Brivaracetam - Serum
BRK1	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
Brucella	Brucellosis - Serology (Rose bengale + brucellacapt) - serum

Brucella abortus	Brucellosis - Serology (Rose bengale + brucellacapt) - serum
Brucella melitensis	Brucellosis - Serology (Rose bengale + brucellacapt) - serum
Brucella suis	Brucellosis - Serology (Rose bengale + brucellacapt) - serum
Brucellacapt	Brucellosis - Serology (Rose bengale + brucellacapt) - serum
Brucellosis - Serology (Rose bengale + brucellacapt) - serum	
Brucellosis-Screening	Bacteria - screening
Brugada syndrome	Cardiogenetic panel or exome
Bullous pemphigoid	Anti-epidermal basement membrane and intracellular substance antibodies - serum
Buprenorphine - Serum	
Buprenorphine and metabolites - urine	
burgdorferi	Borreliosis - direct diagnosis - PCR
burgdorferi	Borreliosis - IgG confirmation serology
burgdorferi	Borreliosis - IgM confirmation serology - serum
burgdorferi	Lyme disease - IgG/IgM screening serology - serum
burgdorferi	Lyme disease - screening serology - CSF
Burgdorferi	Lyme disease - IgG intrathecal index - CSF + serum
Burkitt	Epstein-Barr Virus - EBV - IgA EA serology - serum
Burkitt	Epstein-Barr Virus - EBV - IgG EA serology - serum
Butoxyacetic acid - urine	
Butyl Cellosolve	Butoxyacetic acid - urine
Butyrylcholinesterase	Cholinesterase - Enzymatic activity - serum
BW - screening	Syphilis - Treponemique test - screening serology - serum
BZD	Benzodiazepines - identification - urine
BZD	Benzodiazepines - screening and identification
BZD	Benzodiazepines - screening without identification - Serum
BZD	Benzodiazepines - screening without identification - urine
C1 - Inh	C1-inhibitor - complement fraction - functional + quantitation
C1 - Inh	C1-inhibitor - complement fraction - quantitation - serum
C1 esterase inhibitor	C1-inhibitor - complement fraction - functional + quantitation
C1 esterase inhibitor	C1-inhibitor - complement fraction - quantitation - serum
C1 inhibitor activity	C1-inhibitor - complement fraction - functional + quantitation
C1 inhibitor functional	C1-inhibitor - complement fraction - functional + quantitation
C1-inhibitor - complement fraction - functional + quantitation	
C1-inhibitor - complement fraction - quantitation - serum	
c.187C>G mutation	Haemochromatosis - HFE gene - p.His63Asp mutation
c.193A>T mutation	Haemochromatosis - HFE gene - p.Ser65Cys mutation
C1q - anti-antibodies	Anti-C1q antibodies - serum
C1q - complement fraction - serum	
C1q fraction - complement	C1q - complement fraction - serum
C2 - complement fraction - serum	
C2 fraction - complement	C2 - complement fraction - serum
C282Y Mutation	Haemochromatosis - HFE gene - p.Cys282Tyr mutation
c-erbB-2	HER2 - Human Epidermal growth factor Receptor 2 - c-erbB-2 protein - serum
C-peptide	C-peptide - serum
C-peptide	C-peptide - Urines
C-peptide 12 time	
C-peptide 13 time	
C-peptide - serum	
C-peptide - Urines	
C-reactive protein	CRP - C-reactive protein - serum
C-Reactive Protein	CRP - C-reactive protein - serum
C-reactive protein - ultrasensitive	CRP - C-reactive protein - ultrasensitive - serum
C-Reactive Protein - ultrasensitive	CRP - C-reactive protein - ultrasensitive - serum
C-telopeptide of type I procollagen	CrossLaps - Serum
C-telopeptide of type I procollagen	CrossLaps - urines
C3 - complement fraction - serum	
C3 fraction - complement	C3 - complement fraction - serum
C3NeF - Nephritic factor - serum	

C3PA	Factor B - complement fraction - serum
C4 - complement fraction - serum	
C4 fraction - complement	C4 - complement fraction - serum
C5 - complement fraction - serum	
C5 fraction - complement	C5 - complement fraction - serum
c.845G>A mutation	Haemochromatosis - HFE gene - p.Cys282Tyr mutation
C9 - complement fraction - serum	
C9 fraction - complement	C9 - complement fraction - serum
CA 125	CA 125 antigen - aspiration biopsy
CA 125	CA 125 antigen - serum
CA 125 antigen - aspiration biopsy	
CA 125 antigen - serum	
CA 15-3	CA 15-3 antigen - aspiration biopsy
CA 15-3	CA 15-3 antigen - serum
CA 15-3 antigen - aspiration biopsy	
CA 15-3 antigen - serum	
CA 19-9	CA 19-9 antigen - aspiration biopsy
CA 19-9	CA 19-9 antigen - serum
CA 19-9 antigen - aspiration biopsy	
CA 19-9 antigen - serum	
CA 50	CA 50 antigen - serum
CA 50 antigen - serum	
CA 72-4	CA 72-4 antigen - aspiration biopsy
CA 72-4	CA 72-4 antigen - serum
CA 72-4 antigen - aspiration biopsy	
CA 72-4 antigen - serum	
Cachectin	TNF-alpha - Tumour Necrosis Factor
CACNA1S	Pharmacogenetic Passport - PharmacoPass Expert
Calcitonin - aspiration biopsy	
Calcitonin - Serum	
Calcitriol	Vitamin D - 1,25-dihydroxyvitamin D - serum
Calcium channels - anti-antibodies	Anti-calcium channel antibodies - serum
Calcium Sensing Receptor - anti-antibodies	Anti-Calcium Sensing Receptor antibodies - serum
Calicivirus	Norovirus - direct diagnosis - PCR - Stools
Calicivirus	Norovirus - screening - Stools
Campylobacter	Bacteria - screening
Campylobacter (C. jejuni/coli/upsaliensis)	Gastrointestinal Panel - direct diagnosis - PCR - Stools
Cancer	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
Cancer	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
Cancer	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
Cancer	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
Cancer	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
Cancer	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
Cancer	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
Cancer	Retinoblastoma oncogenetic panel Target gene: RB1
Cancer	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
CANCER	Oncogenetic exome
Cancer of the ovary	Homologous recombination deficiency (HRD) and somatic mutations of the BRCA1 and
cancer ORL HPV induit	Papillomavirus - HPV Détection and Typing of 19 high-risk HPV and 18 low-risk HPV -
Candida	Candida - Mannane antigen - serum
Candida	Fungus - Yeasts - MIC: Minimum Inhibitory Concentration
Candida	Fungus - Yeasts - screening
Candida	Yeasts - antifungal susceptibility testing
Candida - Mannane antigen - serum	
Candida albicans - confirmation serology - serum	
Candida albicans - screening serology - serum	
Candidemia	Beta-D-Glucan antigen - screening
Candidiasis	Beta-D-Glucan antigen - screening

Candidiasis	Candida albicans - confirmation serology - serum
Candidiasis	Candida albicans - screening serology - serum
Candidiasis	Candida - Mannane antigen - serum
Candidiasis	Fungus - Yeasts - MIC: Minimum Inhibitory Concentration
Candidiasis	Fungus - Yeasts - screening
Candidiasis	Yeasts - antifungal susceptibility testing
Cannabidoïdes - dosage - plasma	
Cannabidoïdes - dosage - urines	
Cannabis	Cannabidoïdes - dosage - plasma
Cannabis	Cannabidoïdes - dosage - urines
Cannabis	
capecitabine toxicity	Uracile - dosage(search a risk of toxicity to fluoropyrimidines)
CAR - index - saliva	
CAR antigen	Anti-retina antibodies - serum
Carbamazepine - serum	
Carbapenemase-producing Enterobacteriaceae	Antibiotic multi-resistant bacteria (MRB) - screening
Carbohydrate-deficient transferrin	Carbohydrate-deficient transferrin - serum
Carbohydrate-Deficient Transferrin	Carbohydrate-deficient transferrin - serum
Carbohydrate-deficient transferrin - nephelometry - serum	
Carbohydrate-deficient transferrin - serum	
Carbon monoxide	Carbon monoxide
Carbon monoxide	
Carbonic anhydrase - anti-antibodies	Anti-lactoferrin antibodies and anti carbonic anhydrase antibodies - serum
carboxy-(mono-methyl-heptyl) phthalate (cx-MiNP) - metabolite of DiNP	Phthalates - extended panel - urines - Phthalates - extended panel
Carboxyhemoglobin	Carbon monoxide
Carbutamide	Sulphonylureas - serum
Carcinoembryonic antigen	CEA - Carcinoembryonic antigen - aspiration biopsy
Carcinoembryonic antigen	CEA - Carcinoembryonic antigen - serum
Carcinoma of unknown primary	DNA NGS Panel solid tumor Pan-organ : 64 genes
Cardiac arrhythmia	Cardiogenetic panel or exome
Cardiac conduction disorders	Cardiogenetic panel or exome
Cardiogenetic panel or exome	
Cardiolipin - antibodies	Anti-cardiolipin antibodies - IgG - Serum
Cardiolipin antibodies	Anti-cardiolipin antibodies - IgM - Serum
Cardiomyopathy	Cardiogenetic panel or exome
Carotene	Beta-carotene
CaSR - anti-antibodies	Anti-Calcium Sensing Receptor antibodies - serum
Castle's intrinsic factor	Anti-intrinsic factor antibodies - serum
Cat scratch (disease)	Bartonellosis - B. henselae - B. quintana - IgG serology - serum
Cat scratch (disease)	Bartonellosis - direct diagnosis - PCR
Cat scratch disease	Bartonellosis - B. henselae - B. quintana - IgG serology - serum
Cat scratch disease	Bartonellosis - direct diagnosis - PCR
Catecholamines - free - Urine	
Catecholamines - methoxylated derivatives	Methoxylated derivatives of catecholamines - Free metanephrines
Catecholamines - methoxylated derivatives	Methoxylated derivatives of catecholamines - Free metanephrines - child - Urine
Catecholamines - methoxylated derivatives	Methoxylated derivatives of catecholamines - Free metanephrines - Urine
CBG	CBG - Cortisol-Binding Globulin - Transcortin - serum
CBG - Cortisol-Binding Globulin - Transcortin - serum	
CBNPC	DNA NGS Panel solid tumor Lung
CCDC6	RNA NGS Panel solid tumors, Pan-organ : 65 genes
CCDC88A	RNA NGS Panel solid tumors, Pan-organ : 65 genes
CCNB3	RNA NGS Panel for soft tissue tumors : 20 genes
CCNB3	RNA NGS Panel solid tumors, Pan-organ : 65 genes
CCP - anti-antibodies	Anti-cyclic citrullinated peptide antibodies - CCP2
CD19	Immunophenotyping: Lymphocytes B - CD19 - EDTA whole blood
CD Test	Carbohydrate-deficient transferrin - serum
CD20	Immunophenotyping: Lymphocytes B - CD19 - EDTA whole blood
CD3+	Immunophenotyping: Lymphocytes CD3-CD4-CD8 - EDTA whole blood

CD3 lymphocytes	Immunophenotyping: Lymphocytes CD3-CD4-CD8 - EDTA whole blood
CD3/CD4/CD8	Immunophenotyping: Lymphocytes CD3-CD4-CD8 - EDTA whole blood
CD56	Immunophenotyping: Lymphocytes CD56+ / CD16+ - Natural Killer Cells - EDTA whole
CD74	RNA NGS Panel solid tumors, Pan-organ : 65 genes
CDH1	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
CDH1	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
CDH1	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
CDH1	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
CDK4	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
CDK4	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
CDKN2A	DNA NGS Panel for brain solid tumors
CDKN2A	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
CDKN2A	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
CDKN2A	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
CDKN2B	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
CDKN2B	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
CDT	Carbohydrate-deficient transferrin - serum
CDT-IFC	Carbohydrate-deficient transferrin - serum
CEA - Carcinoembryonic antigen - aspiration biopsy	
CEA - Carcinoembryonic antigen - serum	
Cefepime - Serum	
Cefotaxime - Serum	
Ceftazidime - Serum	
Ceftriaxone - Serum	
Celectol®	Celiprolol - serum
Celiprolol - serum	
Cellcept®	Mycophenolate - plasma
Cereulide toxin	Cereulide toxin - stool
Cereulide toxin - stool	
Certican®	Everolimus
Certolizumab - serum	
Cervical cancer screening;	Detection of high-risk HPV nucleic acids using a molecular technique as part of primary
CgA	Chromogranin A - Serum
CGH	Constitutional Molecular Karyotype-SNP ARRAY
CGH	Constitutional Molecular Karyotype - SNP ARRAY - Product of conception
CGH -array	Constitutional Molecular Karyotype-SNP ARRAY
CGH array	Constitutional Molecular Karyotype-SNP ARRAY
CGH-array	Constitutional Molecular Karyotype - SNP ARRAY - Product of conception
CH50 - Total haemolytic complement	
Chagas disease - confirmation	Trypanosoma cruzi - confirmation - serum
Chagas disease - screening for parasite	Trypanosoma gambiense - screening - CSF
Chagas disease - serology	Trypanosoma cruzi - serology - serum
Chancre	Genital ulcérations panel - direct diagnosis - PCR
Chancre mou	Haemophilus ducreyi - direct diagnosis - PCR
Chancroid	Haemophilus ducreyi - screening
Channelopathy	Cardiogenetic panel or exome
CHEK2	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
CHEK2	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
Chenodeoxycholic acid	Bile acids - chromatography - serum
Chenodeoxycholic acid	Bile acids - serum
CHIP	exome
CHIP	Exome with interpretation interface
CHIP	Nephrology exome
Chitotriosidase - activity	
Chlamydia	Chlamydia pneumoniae - direct diagnosis - PCR
Chlamydia pneumoniae	Chlamydia pneumoniae - IgG/IgM serology - serum
Chlamydia pneumoniae	Respiratory Infection Panel - direct diagnosis - PCR
Chlamydia pneumoniae - direct diagnosis - PCR	

Chlamydia pneumoniae - IgG/IgM serology - serum	
chlamydia psittaci	Chlamydia psittaci - IgG serology (1st intention) - serum
Chlamydia psittaci - IgA serology - serum	
Chlamydia psittaci - IgG serology (1st intention) - serum	
Chlamydia trachomatis	Chlamydia trachomatis, Neisseria gonorrhoeae and Trichomonas vaginalis - multiplex
Chlamydia trachomatis	IST PMA Expert & Multiplex PCR detection & Urogenital mycoplasma, C. trachomatis, N.
Chlamydia Trachomatis	Chlamydia trachomatis, Neisseria gonorrhoeae, Mycoplasma genitalium and
Chlamydia trachomatis, Neisseria gonorrhoeae, Mycoplasma genitalium and Trichomonas	
Chlamydia trachomatis, Neisseria gonorrhoeae and Trichomonas vaginalis - multiplex gene	
Chlamydia trachomatis - IgA serology - serum	
Chlamydia trachomatis - IgG serology - serum	
Chlamydia trachomatis - IgM serology - serum	
Chlordecone	Pesticides - screening and/or dosage - urine
chlore	Electrolytes stool - Stool
Chlorpropamide	Sulphonylureas - serum
Cholangiocarcinoma	RNA NGS Panel for cholangiocarcinoma
Cholangiocarcinoma	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Cholangiocarcinoma NGS panel	DNA NGS Panel for cholangiocarcinoma
Cholangiocarcinoma NGS panel	RNA NGS Panel for cholangiocarcinoma
Cholera screening	Bacteria - screening
Cholic acid	Bile acids - chromatography - serum
Cholic acid	Bile acids - serum
Cholinesterase - Enzymatic activity - serum	
Chromatin	Anti-nucleosome antibodies - serum
Chromatography of erythrocyte fatty acids	Fatty acids of erythrocyte menbrane - chromatography - Total EDTA blood - Fatty free
Chromatography of prolactin	Prolactin - molecular forms - serum
Chromogranin A - Serum	
CIA	Exome Congenital heart defects
Ciflox®	Ciprofloxacin - Serum
Cimzia ®	Certolizumab - serum
Ciprofloxacin - Serum	
Circulating anticoagulant - screening - plasma	
Circulating forms of prolactin	Prolactin - molecular forms - serum
Circulating immune complexes	Circulating immune complexes - CIC - serum
Circulating immune complexes - CIC - serum	
circulating tumor DNA	Circulating tumor DNA NGS Panel - Peripheral blood from Cell Free DNA tube : ambiant
Circulating tumor DNA NGS Panel - Peripheral blood from Cell Free DNA tube : ambiant	
Cis-Chlordane	Pesticides - screening and/or dosage - urine
Citrate	Citric acid - seminal plasma
Citrate	Citric acid - serum
Citrate	Citric acid - urine
Citric acid - seminal plasma	
Citric acid - serum	
Citric acid - urine	
Citrullinated peptides - anti-antibodies	Anti-cyclic citrullinated peptide antibodies - CCP2
CIV	Exome Congenital heart defects
CK MB	CPK - Creatine phosphokinase MB - serum
CK-BB - isoenzymes	CPK - Creatine phosphokinase - isoenzymes - serum
CK-MM - isoenzymes	CPK - Creatine phosphokinase - isoenzymes - serum
CI	Electrolytes stool - Stool
CLA30 - food allergens	IgE against multiple individual allergens - food allergens - Serum
CLA30 - inhalant allergens	IgE against multiple individual allergens - inhalant allergens - Serum
CLA30 - mixed allergens	IgE against multiple individual allergens - Food and Inhalant panels - Serum
Claforan®	Cefotaxime - Serum
Clamoxil®	Amoxicillin - Serum
Clarithromycin	Helicobacter pylori and clarithromycin resistance gene - direct diagnosis - PCR
Class I HLA - HLA-A and HLA-B genotyping	Class I HLA - HLA-A and HLA-B genotyping - EDTA whole blood
Class I HLA - HLA-A and HLA-B genotyping - EDTA whole blood	

Class I HLA leukocyte and platelet antigens	Class I HLA - HLA-A and HLA-B genotyping - EDTA whole blood
Class II HLA - HLA-DQ and HLA-DR genotyping	Class II HLA - HLA-DQ and HLA-DR genotyping - EDTA whole blood
Class II HLA - HLA-DQ and HLA-DR genotyping - EDTA whole blood	
Class II HLA leukocyte and platelet antigens	Class II HLA - HLA-DQ and HLA-DR genotyping - EDTA whole blood
Cleaving protease - Willebrand factor	Factor Willebrand protease activity - ADAMTS 13
Clindamycin - Serum	
CLL	Immunophenotyping of malignant hemopaties
Clobazam - Serum	
Clomipramine - Serum	
Clonazepam - Serum	
Clopidogrel - resistance test	VASP - VAsodilatator Stimulated Phosphoprotein - whole blood
Clopixol®	Zuclopenthixol - Serum
Clorazepate - Serum	
Clostridium difficile : toxines A/B	Gastrointestinal Panel - direct diagnosis - PCR - Stools
Clostridium difficile - Glutamate dehydrogenase (GDH) screening (+toxins)-stool	
Clostridium difficile - toxins-screening by PCR	
Clostridium tetani - immunity test - IgG antibodies	Tetanus - immunity test - IgG serology - serum
Cloxacillin - Serum	
Clozapine	
CLTC	RNA NGS Panel solid tumors, Pan-organ : 65 genes
CLU	Free cortisol - Compound F - urine
CMV	Cytomegalovirus - CMV - - direct diagnosis - PCR
CMV	Cytomegalovirus - CMV - - direct diagnosis - PCR - amniotic fluid
CMV	Cytomegalovirus - CMV - - IgG/IgM serology - serum
CMV	Cytomegalovirus - CMV - - IgG serology - serum
CMV	viral meningoencephalitis panel - direct diagnosis - PCR
CMV - antibody avidity	Cytomegalovirus - CMV - IgG avidity test - serum
CNV	exome
CNV	Exome Congenital heart defects
CNV	Exome Isolated or syndromic intellectual disability and syndromic neurodevelopmental
CNV	Exome Panel for chronic and hereditary pancreatitis
CNV	Exome with interpretation interface
CNV	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
CNV	Nephrology exome
CNV	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
CNV	Oncogenetic exome
CNV	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
CNV	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
CNV	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
CNV	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
CNV	Prenatal exome
CNV	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
CNV	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
Cobalamin	Vitamin B12 - serum
Cobalt - urines	
Cobalt - Whole blood	
Cobalt concentration	Cobalt - Whole blood
Cobalt overload	Cobalt - urines
Cocaéthylène	Cocaine - dosage - plasma
Cocaéthylène	Cocaine - dosage - urines
Cocaine - dosage - plasma	
Cocaine - dosage - urines	
Codeine	Opiates - dosage - plasma
Codeine	Opiates - dosage - urines
Codon 129 prion protein - genotype	Protein 14.3.3
Coenzyme Q10	Coenzyme Q10 - CoQ10 - Serum
Coenzyme Q10 - CoQ10 - Serum	
CoHb	Carbon monoxide

Colimycin	Colistin - Serum
Colistin - Serum	
Collagen N-telopeptides	NTx - N-telopeptides of type I collagen - urines
Collagene de type 7	Anti-epidermal basement membrane and intracellular substance antibodies Typing
Colon	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
Colon cancer	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
Colorectal	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
Colorectal adenocarcinoma	DNA NGS Panel solid tumor Colon
Colorectal adenocarcinoma	DNA NGS Panel solid tumor Colon and MSI status determination
Colorectal adenocarcinoma	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
Colorectal cancer	DNA NGS Panel solid tumor Colon
Colorectal cancer	DNA NGS Panel solid tumor Colon and MSI status determination
Colorectal cancer	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
COLORECTAL CANCER	Oncogenetic exome
Colorectal NGS panel	DNA NGS Panel solid tumor Colon
Colorectal NGS panel	DNA NGS Panel solid tumor Colon and MSI status determination
Colorectal NGS panel	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
Compagnon test	Circulating tumor DNA NGS Panel - Peripheral blood from Cell Free DNA tube : ambiant
Companion diagnostic	DNA NGS Panel for cholangiocarcinoma
Companion diagnostic	DNA NGS Panel for pancreatic cancer
Companion diagnostic	DNA NGS Panel for prostate cancer
Companion diagnostic	DNA NGS Panel for thyroid cancer
Companion diagnostic	DNA NGS Panel solid tumor Colon
Companion diagnostic	RNA NGS Panel for cholangiocarcinoma
Companion diagnostic	RNA NGS Panel for lung cancers
Companion diagnostic	RNA NGS Panel for pancreatic cancer
Companion diagnostic	RNA NGS Panel for thyroid cancer
Companion test	DNA NGS Panel solid tumor Breast
Companion test	DNA NGS Panel solid tumor Colon and MSI status determination
Companion test	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
Companion test	DNA NGS Panel solid tumor Endometrium
Companion test	DNA NGS Panel solid tumor Endometrium and MSI status determination
Companion test	DNA NGS Panel solid tumor GIST
Companion test	DNA NGS Panel solid tumor Lung
Companion test	DNA NGS Panel solid tumor Lung extended
Companion test	DNA NGS Panel solid tumor Melanoma
Companion test	DNA NGS Panel solid tumor Pan-organ : 64 genes
Companion test	Homologous recombination deficiency (HRD) and somatic mutations of the BRCA1 and
Complement - C1q fraction	C1q - complement fraction - serum
Complement - C2 fraction	C2 - complement fraction - serum
Complement - C3 fraction	C3 - complement fraction - serum
Complement - C3 proactivator	Factor B - complement fraction - serum
Complement - C4 fraction	C4 - complement fraction - serum
Complement - Factor B	Factor B - complement fraction - serum
Complement C1 inhibitor	C1-inhibitor - complement fraction - functional + quantitation
Complement C1 inhibitor	C1-inhibitor - complement fraction - quantitation - serum
Complement: C5 fraction	C5 - complement fraction - serum
Complement: C9 fraction	C9 - complement fraction - serum
Compound B	Corticosterone - Compound B
Compound E	Cortisone - Compound E - plasma
Compound E	Cortisone - Compound E - urines
Compound F	Cortisol - Compound F - saliva
Compound F	Cortisol - Compound F - serum
Compound F	Free cortisol - Compound F - urine
Compound F	
Compound S	11-Desoxycortisol - Compound S - - (Tubes without separating gel)
Condylomes	Papillomavirus - HPV Détection and Typing of 19 high-risk HPV and 18 low-risk HPV -
Conferoport	Tacrolimus - FK-506

confirmation	HTLV I/II - confirmation serology - serum
Connectin - anti-antibodies	Anti-titin antibodies - serum
Constitutional Molecular Karyotype - SNP ARRAY - Product of conception	
Constitutional Molecular Karyotype-SNP ARRAY	
CONSTITUTIONAL ONCOGENETICS	Oncogenetic exome
control - CDT	Carbohydrate-deficient transferrin - nephelometry - serum
Conventional acquired cytogenetics	Blood karyotype
Copper	
Copper - exchangeable / free	
Copper - exchangeable/free	Copper - exchangeable / free
Copper - urine	
Copper concentration	Copper
Copper overload	Copper - urine
Coproporphyrins	Erythrocyte porphyrins
Coproporphyrins	Porphyrins - stool
Coproporphyrins	Porphyrins - Urines
Cordarone®	Amiodarone + metabolite - Serum
Coronavirus	COVID-19 (SARS-CoV-2) IgG serology anti-S
CoronaVirus Disease	COVID-19 (SARS-CoV-2) IgG serology anti-S
CoronaVirus Disease	COVID-19 (SARS-CoV-2) serology (total Ig) anti-N
Coronavirus HKU1, NL63, 229E, OC43	Respiratory Infection Panel - direct diagnosis - PCR
Coronavirus SARS-CoV-2	COVID-19 (SARS-CoV-2) serology (total Ig) anti-N
Cortexolone	11-Desoxycortisol - Compound S - - (Tubes without separating gel)
Corticosterone - 11-Deoxy	11-Deoxycorticosterone
Corticosterone - 11-Deoxy	
Corticosterone - Compound B	
Corticotstimulin	ACTH - Corticotrophin
Corticotrophin	ACTH - Corticotrophin
Corticotropin	ACTH - Corticotrophin
Cortisol - 21-Deoxy	21-Deoxycortisol - (Tubes without separating gel)
Cortisol - Compound F	
Cortisol - Compound F - evening - saliva	
Cortisol - Compound F - evening - Serum	
Cortisol - Compound F - morning - saliva	
Cortisol - Compound F - saliva	
Cortisol - Compound F - Saliva	
Cortisol - Compound F - serum	
Cortisol Awakening Response	CAR - index - saliva
Cortisol-binding globulin	CBG - Cortisol-Binding Globulin - Transcortin - serum
Cortisone - Compound E - plasma	
Cortisone - Compound E - urines	
Corynebacterium diptheriae	Diphtheria - immunity test - IgG serology - serum
Cotinine - urine	
COVID	COVID 19 / Influenza virus A and B - panel - direct diagnosis - PCR
COVID 19 / Influenza virus A and B - panel - direct diagnosis - PCR	
Covid-19	Respiratory Infection Panel - direct diagnosis - PCR
COVID-19	COVID 19 / Influenza virus A and B - panel - direct diagnosis - PCR
COVID-19	COVID-19 (SARS-CoV-2) IgG serology anti-S
COVID-19	COVID-19 (SARS-CoV-2) serology (total Ig) anti-N
COVID-19 (SARS-CoV-2) IgG serology anti-S	
COVID-19 (SARS-CoV-2) serology (total Ig) anti-N	
Coxiella burnetii (Q fever) - IgG and IgM serology, Phases I and II - serum	
Coxiella burnetii (Q fever) - Screening serology Total Ac Phase II - serum	
Coxsackie virus groupes A et B	Enterovirus - direct diagnosis - PCR
CPE	Antibiotic multi-resistant bacteria (MRB) - screening
CPK - Creatine phosphokinase - isoenzymes - serum	
CPK - Creatine phosphokinase MB - serum	
CPK - Creatine phosphokinase total - serum	

CPK isoenzymes	CPK - Creatine phosphokinase - isoenzymes - serum
CPK MB	CPK - Creatine phosphokinase MB - serum
CRC	DNA NGS Panel solid tumor Colon
CRC	DNA NGS Panel solid tumor Colon and MSI status determination
CRC	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
Creatine deficiency	Guanidinoacetate and creatine - plasma
Creatine deficiency	Guanidinoacetate and creatine - urine
Creatine phosphokinase - isoenzymes	CPK - Creatine phosphokinase - isoenzymes - serum
Creatine phosphokinase MB	CPK - Creatine phosphokinase MB - serum
Creutzfeldt-Jakob	Protein 14.3.3
Cross link	NTx - N-telopeptides of type I collagen - urines
CrossLaps - Serum	
CrossLaps - urines	
CRP - C-reactive protein - serum	
CRP - C-reactive protein - ultrasensitive - serum	
CRP - Mycobacteria	Mycobacteria - PCR M. tuberculosis complex (BKBM)
CRP full range	CRP - C-reactive protein - ultrasensitive - serum
Cryoglobulins - identification - serum	
Cryptococcosis	Fungus - Yeasts - screening
Cryptococcus	Fungus - Yeasts - screening
Cryptococcus - soluble antigen	Cryptococcus neoformans - soluble antigen - testing
Cryptococcus neoformans - soluble antigen - testing	
Cryptosporidies - Microsporidies - direct diagnosis - PCR	
Cryptosporidiose	Cryptosporidies - Microsporidies - direct diagnosis - PCR
Cryptosporidium spp.	Cryptosporidies - Microsporidies - direct diagnosis - PCR
Cryptosporidium spp.	Gastrointestinal Panel - direct diagnosis - PCR - Stools
CSF	
CSF electrophoresis	Isoelectric focusing - CSF + serum
CST	Iron - saturation percentage - serum
ctDNA	Circulating tumor DNA NGS Panel - Peripheral blood from Cell Free DNA tube : ambiant
CTF	Iron - total binding capacity - serum
CTX	CrossLaps - Serum
CTX	CrossLaps - urines
Cuivre libre	Copper - exchangeable / free
CUP	DNA NGS Panel solid tumor Pan-organ : 64 genes
Cutaneous	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
Cyamemazine - Serum	
Cyanide ions	Thiocyanates - ion - Serum
Cyanide ions	Thiocyanates - ion - urine
Cyclohexane-1,2-dicarboxylate-mono-(7-carboxylate-4- methyl)heptyl ester (cx-MINCH) -	Phtalates - extended panel - urines - Phtalates - extended panel
Cyclohexane-1,2-dicarboxylate-mono-(7-hydroxy-4- methyl)octyl ester (OH-MINCH) -	Phtalates - extended panel - urines - Phtalates - extended panel
Cyclospora cayetanensis	Gastrointestinal Panel - direct diagnosis - PCR - Stools
Cyclosporin	
Cyclosporin A	Cyclosporin
Cyfra 21-1	Cyfra 21-1 antigen - aspiration biopsy
Cyfra 21-1	Cyfra 21-1 antigen - serum
Cyfra 21-1 antigen - aspiration biopsy	
Cyfra 21-1 antigen - serum	
Cymbalta	Duloxetine - serum
Cymevan®	Ganciclovir - plasma
CYP1A2	Pharmacogenetic Passport - PharmacoPass Expert
CYP2B6	Pharmacogenetic Passport - PharmacoPass Expert
CYP2C19	Pharmacogenetic Passport - PharmacoPass Expert
CYP2C9	Pharmacogenetic Passport - PharmacoPass Expert
CYP2D6	Pharmacogenetic Passport - PharmacoPass Expert
CYP3A4	Pharmacogenetic Passport - PharmacoPass Expert
CYP3A5	Pharmacogenetic Passport - PharmacoPass Expert
CYP4F2	Pharmacogenetic Passport - PharmacoPass Expert

Cystic fibrosis Detection of variations within the CFTR gene after massive	
Cystic Fibrosis Detection of variations within the CFTR gene after massive	
Cysticercosis - confirmation serology	
Cysticercosis - Taenia Solium - IgG serology - serum	
cysts	Parasitology - microscopy
Cytokin	Interleukin (IL)-10 - .
Cytokin	Interleukin (IL)-6
Cytokin	Interleukin (IL)-6 - .
Cytokine	Interleukin (IL)-10
Cytokine	Interleukin (IL)-1 beta
Cytokine	Interleukin (IL)-2 - receptor
Cytokine	Interleukin (IL)-5
Cytokine	Interleukin (IL)-8
Cytokine	sérum and plasma EDTA
Cytokine	TNF-alpha - Tumour Necrosis Factor
Cytokines	Interleukin (IL)-10
Cytokines	Interleukin (IL)-1 beta
Cytokines	Interleukin (IL)-6
Cytokines	Interleukin (IL)-8
Cytokines	sérum and plasma EDTA
Cytokines	TNF-alpha - Tumour Necrosis Factor
Cytomegalovirus	Cytomegalovirus - CMV - - direct diagnosis - PCR - amniotic fluid
CytomégaloVirus	Cytomegalovirus - CMV - - direct diagnosis - PCR
CytomégaloVirus - CMV - PCR - liquide amniotique	Cytomegalovirus - CMV - - direct diagnosis - PCR - amniotic fluid
Cytomegalovirus - CMV - - direct diagnosis - PCR	
Cytomegalovirus - CMV - - direct diagnosis - PCR - amniotic fluid	
Cytomegalovirus - CMV - - IgG serology - serum	
Cytomegalovirus - CMV - - IgG/IgM serology - serum	
Cytomegalovirus - CMV - IgG avidity test - serum	
Cytometry	Immunophenotyping: Lymphocytes B - CD19 - EDTA whole blood
Cytometry	Immunophenotyping: Lymphocytes CD3-CD4-CD8 - EDTA whole blood
Cytometry	Immunophenotyping: Lymphocytes CD56+ / CD16+ - Natural Killer Cells - EDTA whole
Cytometry	Immunophenotyping: Lymphocytes NK/CD57 - EDTA whole blood
Cytosol - LC1 - anti-antibodies	Anti-liver cytosol type 1 antibodies - LC1 - confirmation - serum
Cytosol- LC1 - Abs anti	Anti-liver cytosol type 1 antibodies - LC1 - screening - serum
Cytotoxic antibodies	anti-HLA antibodies - classes I and II - transplant - serum
D hepatitis	Hepatitis Delta - antigen - serum
D hepatitis	Hepatitis Delta - IgM antibody - serum
D hepatitis	Hepatitis Delta - total antibodies - serum
D-Di	D-Dimers - plasma
D-Dimers - plasma	
D4A - Delta-4-Androstenedione	
D4A - Delta-4-Androstenedione - serum	
Dabigatran - Measurement of the Dabigatran anti lia activity - plasma	
Dacryocytes	Erythrocytes morphological aspect
Dahlback factor	Activated protein C resistance - plasma
Dahlback test	Activated protein C resistance - plasma
Dalacine®	Clindamycin - Serum
Danaparoid - anti-Xa	Anti-Xa activity : Danaparoid - Orgaran - plasma
Daonil	Glibenclamide - serum
Darunavir - Serum	
Datamcine	Gentamicine - serum
DAZ gene - deleted in azoospermia	Y chromosome - (regions AZFs, AZFb, AZFc)
Decarboxytransferrin	Carbohydrate-deficient transferrin - serum
Dehydro-epiandrosterone	DHEA - Dehydroepiandrosterone
Dehydroepiandrosterone sulfate	DHEAS - Dehydroepiandrosterone sulfate
Delta 5 - pregnenolone	
delta 5-pregnenolone sulfate	Pregnenolone sulfate - serum

Delta antigen	Hepatitis Delta - antigen - serum
delta hepatitis	Hepatitis Delta - antigen - serum
delta hepatitis	Hepatitis Delta - total antibodies - serum
Delta hepatitis	Hepatitis Delta - IgM antibody - serum
Delta virus - antigen	Hepatitis Delta - antigen - serum
Delta-4-androstenedione	D4A - Delta-4-Androstenedione - serum
Delta-5-pregnenolone	17-Hydroxy-Pregnenolone - serum
Delta-5-pregnenolone	Pregnenolone - serum
delta-Aminolevulinic acid - urine	
Delta5 - pregnenolone	
dengue	Dengue fever - IgG/IgM serology - serum
Dengue fever - direct diagnosis - PCR	
Dengue fever - IgG/IgM serology - serum	
Deoxycholic acid	Bile acids - chromatography - serum
Deoxycholic acid	Bile acids - serum
Deoxycorticosterone - 11	11-Deoxycorticosterone
Deoxycorticosterone - 11	
deoxycortisol - 11	11-Desoxycortisol - Compound S - - (Tubes without separating gel)
Depakine®	Valproic acid - serum
Depakote	Valproic acid - serum
Depamide®	Valproic acid - serum
dermatomyositis	Myosite Markers - serum
Dermatophyte	Fungus - Yeasts - identification
Dermatophytes	Fungus - Yeasts - screening
Deroxat®	Paroxetine - serum
Designated bacterium - direct diagnosis - PCR	
Desmethyclomipramine	Clomipramine - Serum
Desmethylozapine	Clozapine
desmoglein 1	Anti-epidermal basement membrane and intracellular substance antibodies Typing
desmoglein 3	Anti-epidermal basement membrane and intracellular substance antibodies Typing
Detection and identification bacterial DNA - 16S PCR by NGS sequencing	
Detection of free light chains	Bence Jones - screening
Detection of high-risk HPV nucleic acids using a molecular technique as part of primary CCU	
Detection of the HPV16 genome in the context of anal cancer screening	
détection qualitative	HIV-1: Qualitative RNA detection, suspected primary infection - serum
Determination of HIV-1 viral tropism (CCR5 - V3 loop, gp120)	HIV - Human Immunodeficiency Virus - genotyping (envelope, tropism): antiretroviral
DEVELOPMENTAL DISORDER	Exome Isolated or syndromic intellectual disability and syndromic neurodevelopmental
Dextropropoxyphene - urine	
Dextroref®	Dextropropoxyphene - urine
DFI	DNA fragmentation test - DNA decondensation - SCSA - Sperm
DHA	DHEA - Dehydroepiandrosterone
DHA	
DHA sulfate	DHEAS - Dehydroepiandrosterone sulfate
DHAS	DHEAS - Dehydroepiandrosterone sulfate
DHEA - Dehydroepiandrosterone	
DHEAS - Dehydroepiandrosterone sulfate	
DHT - Dihydrotestosterone	
DHT - Dihydrotestosterone - serum	
Di Dolko®	Dextropropoxyphene - urine
Di-Antalvic®	Dextropropoxyphene - urine
Di-Hydan®	Phenytoin - serum
Diacomit®	Stiripentol - Serum
Dialgirex®	Dextropropoxyphene - urine
Diamine oxydase	Diamine oxydase (DAO) - activity - serum
Diamine oxydase (DAO) - activity - serum	
Diazepam + desmethyldiazepam - Serum	
Dicofol	Pesticides - screening and/or dosage - urine
Dieldrin	Pesticides - screening and/or dosage - urine

Digest Check	Juvenalis - DigestCheck - Stool
Digestion	Faecal test -Functional coprology - Stool
Digestive	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
Digestive	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
Digestive cancer	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
DIGESTIVE CANCER	Oncogenetic exome
Digestive parasitosis	Helminthes - Protozoaires - direct diagnosis - PCR - Stool
Digestive tumors	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Digoxine par immunoturbidimétrie (Abbott) - Sérum	
dihydropyrimidine dehydrogenase deficiency	Uracile - dosage(search a risk of toxicity to fluoropyrimidines)
dihydropyrimidine dehydrogenase phenotype	Uracile - dosage(search a risk of toxicity to fluoropyrimidines)
Dihydrotestosterone	DHT - Dihydrotestosterone - serum
Dihydroxyvitamin D3 - 1,25	Vitamin D - 1,25-dihydroxyvitamin D - serum
Dilantin	Phenytoin - serum
Dilated cardiomyopathy	Cardiogenetic panel or exome
Dilatrane®	Theophylline - serum
Dioalgo®	Dextropropoxyphene - urine
Diphantoin	Phenytoin - serum
Diphenylhydantoin	Phenytoin - serum
diphtheria	Diphtheria - immunity test - IgG serology - serum
Diphtheria - immunity test - IgG serology - serum	
Diphtheria - screening	Bacteria - screening
Diphtheria toxin - anti-antibodies	Diphtheria - immunity test - IgG serology - serum
Dipropylacetamide	Valproic acid - serum
Dipyridamole - Serum	
Direct antiglobulin exam	Erythrocyte direct Coombs
Direct antiglobulin test	Erythrocyte direct Coombs
Direct Coombs test	Erythrocyte direct Coombs
Direct LDL cholesterol - serum	
Disopyramide - serum	
Distomatosis - confirmation serology - serum	
Distomatosis - screening serology - serum	
Divalproate de sodium	Valproic acid - serum
Dixon test	Anti-platelet antibodies bound or free - screening and identification
DLST	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
DLST	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
DNA bacterien	Detection and identification bacterial DNA - 16S PCR by NGS sequencing
DNA chip	Constitutional Molecular Karyotype-SNP ARRAY
DNA CHIP	Prenatal exome
DNA Fragmentation Index	DNA fragmentation test - DNA decondensation - SCSA - Sperm
DNA fragmentation test - DNA decondensation - SCSA - Sperm	
DNA NGS Panel for brain solid tumors	
DNA NGS Panel for cholangiocarcinoma	
DNA NGS Panel for pancreatic cancer	
DNA NGS Panel for prostate cancer	
DNA NGS Panel for thyroid cancer	
DNA NGS Panel solid tumor Breast	
DNA NGS Panel solid tumor colo-rectal extended : 22 genes	
DNA NGS Panel solid tumor Colon	
DNA NGS Panel solid tumor Colon and MSI status determination	
DNA NGS Panel solid tumor Endometrium	
DNA NGS Panel solid tumor Endometrium and MSI status determination	
DNA NGS Panel solid tumor GIST	
DNA NGS Panel solid tumor Lung	
DNA NGS Panel solid tumor Lung extended	
DNA NGS Panel solid tumor Melanoma	
DNA NGS Panel solid tumor Pan-organ : 64 genes	
DNAJB1	RNA NGS Panel solid tumors, Pan-organ : 65 genes

DNMT3A	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
DNMT3A	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
DOC	11-Deoxycorticosterone
DOC	
Dolutegravir - Serum	
Dopamine	Catecholamines - free - Urine
Dopamine	Free catécholamines
Dopamine	Free catécholamines - children - Urine
Down syndrome	Trisomy 21 - 1st trimester combined screening
Down syndrome	Trisomy 21 - second-trimester maternal serum markers - 2 parameters
Down syndrome	Trisomy 21 - second-trimester maternal serum markers - 3 parameters
Doxy®	Doxycycline - Serum
Doxycycline - Serum	
Doxylis®	Doxycycline - Serum
Doxypalu®	Doxycycline - Serum
DP, DQ, DR alleles - genotyping	Class II HLA - HLA-DQ and HLA-DR genotyping - EDTA whole blood
DPYD	Pharmacogenetic Passport - PharmacoPass Expert
dsDNA - anti-antibodies	Anti-DNA antibodies Double stranded DNA - Serum
dsDNA - anti-antibodies	Anti-DNA antibodies double stranded - Farr test - serum
Duloxetine - serum	
Dysgammaglobulinemia	Immunobinding - serum
Dysgammaglobulinemia	Protein electrophoresis and Immunobinding - serum
Dysmorphic fungi	Fungus - Yeasts - screening
E1	Oestrone - serum
E. coli O157:H7	Gastrointestinal Panel - direct diagnosis - PCR - Stools
E. granulosus - E. multilocularis	Echinococcosis - confirmation serology - serum
E. granulosus - E. multilocularis	Echinococcosis - screening serology - serum
E hepatitis	Hepatitis E - IgG antibody - serum
E hepatitis	Hepatitis E - IgM antibody - serum
E2	Oestradiol - man/child - serum
E2	Oestradiol - woman - serum
E2	
EA - IgA serology	Epstein-Barr Virus - EBV - IgA EA serology - serum
EA - IgG serology	Epstein-Barr Virus - EBV - IgG EA serology - serum
Early access	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
Early access	DNA NGS Panel solid tumor Pan-organ : 64 genes
early antigen	Epstein-Barr Virus - EBV - IgA EA serology - serum
Early antigen	Epstein-Barr Virus - EBV - IgG EA serology - serum
Eberth bacillus	Salmonellosis - serology - serum
EBNA - IgG serology	Epstein-Barr Virus - EBV - VCA IgG, EBNA, IgG serology - serum
EBV	Epstein-Barr Virus - EBV - direct diagnosis - PCR
EBV	Epstein-Barr Virus - EBV - direct diagnosis - PCR - amniotic fluid
EBV	Epstein-Barr Virus - EBV - IgA EA serology - serum
EBV	Epstein-Barr Virus - EBV - IgG EA serology - serum
EBV	Epstein-Barr Virus - EBV - VCA IgG, EBNA, IgG serology - serum
EBV	Epstein-Barr Virus - EBV - VCA IgG/IgM and EBNA IgG serology - serum
EBV	Epstein-Barr Virus - EBV - VCA IgM serology - serum
EBV	Epstein-Barr Virus - EBV - viral load - PCR
EBV	viral meningoencephalitis panel - direct diagnosis - PCR
ECA - Enzyme de Conversion de l'Angiotensine (activité Buhlmann Alinity® Abbott) - serum	
Ecgonine méthyl ester	Cocaine - dosage - plasma
Echinococcosis - confirmation serology - serum	
Echinococcosis - screening serology - serum	
echinocytes	Erythrocytes morphological aspect
Echovirus	Enterovirus - direct diagnosis - PCR
ECP - Eosinophil Cationic Protein	
Ecstasy	Amphetamines - dosage - plasma
Ecstasy	Amphetamines - dosage - urines

ECT - anti-antibodies	Anti-ENA antibodies (SSA, SSB, Sm, Rnp, Jo1, Scl-70) - screening and titration - Serum
EDA	Erythrocyte direct Coombs
EDDP	Methadone and EDDP - Serum
Edurant	Rilpivirine - Serum
Effexor®	Venlafaxine - serum
Effient	VASP - VAsodilatator Stimulated Phosphoprotein - whole blood
Efient	VASP - VAsodilatator Stimulated Phosphoprotein - whole blood
EGBE	Butoxyacetic acid - urine
EGFR	DNA NGS Panel for brain solid tumors
EGFR	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
EGFR	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
EGFR	RNA NGS Panel solid tumors, Pan-organ : 65 genes
EGFR - ALK - ROS1 - RET - MET - KRAS - BRAF - ERBB2 - NTRK1 - TP53 - STK11 -	DNA NGS Panel solid tumor Lung extended
EGFR - KRAS - BRAF - ERBB2 - MET - PIK3CA	DNA NGS Panel solid tumor Lung
EGFRvIII	RNA NGS panel for brain solid tumors
eggs	Parasitology - microscopy
EGLN1	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
EGLN2	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
EGLN2	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
Eicosapentaenoic acid	Fatty acids of erythrocyte membrane - chromatography - Total EDTA blood - Fatty free
EJ - anti-antibodies	Myosite Markers - serum
Elavil®	Amitriptyline - Serum
electrolytes	Electrolytes stool - Stool
Electrolytes stool - Stool	
Eliquis	Apixaban(Eliquis ®) - Anti Xa activity measurement - plasma
elliptocytes	Erythrocytes morphological aspect
EML4	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Emtricitabine - plasma	
Emtriva®	Emtricitabine - plasma
ENA - anti-antibodies	Anti-ENA antibodies (SSA, SSB, Sm, Rnp, Jo1, Scl-70) - screening and titration - Serum
Enbrel®	Etanercept - serum
Encephalitozoon intestinalis	Cryptosporidies - Microsporidies - direct diagnosis - PCR
Endemic typhus	Rickettsioses - R. conorii, R. typhi - serology - serum
Endocrine tumors	Serotonin - whole blood
Endocrinologie	ACTH - Corticotrophin
Endocrinologie	Inhibin B - man
Endocrinologie	Oestradiol - woman - serum
Endocrinologie	Total HCG (HCG dimer + beta fraction) - serum
Endocrinologie	
Endocrinologu	
Endocrinology	11-Deoxycorticosterone
Endocrinology	11-Desoxycortisol - Compound S - - (Tubes without separating gel)
Endocrinology	17-Hydroxyprogesterone
Endocrinology	21-Deoxycortisol - (Tubes without separating gel)
Endocrinology	3-Alpha-Androstenediol Glucuronide - serum
Endocrinology	5-HIAA - 5-hydroxyindoleacetic acid - Urine
Endocrinology	ACTH - Corticotrophin - evening
Endocrinology	ACTH - Corticotrophin - morning
Endocrinology	ADH - Antidiuretic hormone - urine
Endocrinology	Aldosterone
Endocrinology	Aldosterone - Lying down
Endocrinology	Aldosterone - Standing or Sitting position
Endocrinology	Aldosterone - urine
Endocrinology	Alpha subunit - hCG, FSH, LH, TSH - serum
Endocrinology	AMH - Anti-Müllerian hormone
Endocrinology	AMH - Anti-Müllerian hormone - Child
Endocrinology	Angiotensin II - plasma
Endocrinology	CA 50 antigen - serum

Endocrinology	CAR - index - saliva
Endocrinology	Catecholamines - free - Urine
Endocrinology	Corticosterone - Compound B
Endocrinology	Cortisol - Compound F - evening - saliva
Endocrinology	Cortisol - Compound F - evening - Serum
Endocrinology	Cortisol - Compound F - morning - saliva
Endocrinology	Cortisol - Compound F - saliva
Endocrinology	Cortisol - Compound F - serum
Endocrinology	Cortisone - Compound E - plasma
Endocrinology	Cortisone - Compound E - urines
Endocrinology	C-peptide - serum
Endocrinology	C-peptide - Urines
Endocrinology	D4A - Delta-4-Androstenedione - serum
Endocrinology	DHEAS - Dehydroepiandrosterone sulfate
Endocrinology	DHT - Dihydrotestosterone - serum
Endocrinology	Free catécholamines
Endocrinology	Free catécholamines - children - Urine
Endocrinology	Free cortisol - Compound F - urine
Endocrinology	Free T3 - Free triiodothyronine - serum
Endocrinology	Free T4 - Free thyroxine - serum
Endocrinology	FSH - Follicle-stimulating hormone - serum
Endocrinology	FSH - Follicle-stimulating hormone - urines
Endocrinology	Gastrin - serum
Endocrinology	GH - Growth hormone - Serum
Endocrinology	Ghrelin - acylated and non acylated - plasma
Endocrinology	GHRH - Growth-Hormone-Releasing-Hormone - plasma
Endocrinology	Glucagon - plasma
Endocrinology	hCG - free beta chain - subunit - serum
Endocrinology	HOMA + QUICKI - index - serum
Endocrinology	Homovanillic acid - Urine
Endocrinology	IGF-1 - Somatomedin C - Serum
Endocrinology	IGF-2 - serum
Endocrinology	IGFBP-3 - Insulin-like Growth Factor Binding protein-3 - serum
Endocrinology	Inhibin A - serum
Endocrinology	Inhibin A - ultrasensitive - serum
Endocrinology	Inhibin B - Child - serum
Endocrinology	Inhibin B - woman
Endocrinology	Leptin - serum
Endocrinology	LH - Luteinising hormone - serum
Endocrinology	LH - Luteinising hormone - urines
Endocrinology	Methoxylated derivatives of catecholamines - Free metanephrines
Endocrinology	Methoxylated derivatives of catecholamines - Free metanephrines - child - Urine
Endocrinology	Methoxylated derivatives of catecholamines - Free metanephrines - Urine
Endocrinology	Oestradiol - man/child - serum
Endocrinology	Oestrone - serum
Endocrinology	Pregnenolone - serum
Endocrinology	Proinsulin - serum
Endocrinology	Prolactin - molecular forms - serum
Endocrinology	Prolactin - serum
Endocrinology	PTH-RP - Parathormone related protein - plasma
Endocrinology	Renin active
Endocrinology	Renin Active - lying
Endocrinology	Renin - Standing or Sitting position
Endocrinology	rT3 - Reverse triiodothyronine
Endocrinology	Serotonin - whole blood
Endocrinology	Sex-Binding Globulin - SHBG
Endocrinology	Somatostatin - plasma
Endocrinology	Steroid profile - urine

Endocrinology	Steroids panel - Examination of gonad function
Endocrinology	Steroids panel - Exploration of hirsutism
Endocrinology	Steroids panel - Testing for a disorder in steroid biosynthesis
Endocrinology	T3 - Triiodothyronine - urine
Endocrinology	T4 - Thyroxine - urine
Endocrinology	Testosterone bioavailability - serum
Endocrinology	Testosterone Bioavailable - woman/ child - serum
Endocrinology	Testosterone free - Serum
Endocrinology	Testosterone - man
Endocrinology	Testosterone - woman/child - serum
Endocrinology	Thyroxine-binding globulin - TBG - serum
Endocrinology	Total T3 - Total triiodothyronine - serum
Endocrinology	Total T4 - Total thyroxine - serum
Endocrinology	TSH - Thyroid-stimulating hormone - Third generation - serum
Endocrinology	Type III procollagen - serum
Endocrinology	Vanilmandelic acid - Urine
Endocrinology	VIP - Vasoactive Intestinal Polypeptide - plasma
Endocrinology	
EndocrinologyY	DHEA - Dehydroepiandrosterone
EndocrinologyY	Pregnenolone sulfate - serum
Endometrial	DNA NGS Panel solid tumor Endometrium
Endometrial	DNA NGS Panel solid tumor Endometrium and MSI status determination
Endometrial adenocarcinoma	DNA NGS Panel solid tumor Endometrium
Endometrial adenocarcinoma	DNA NGS Panel solid tumor Endometrium and MSI status determination
Endometrial cancer	DNA NGS Panel solid tumor Endometrium
Endometrial cancer	DNA NGS Panel solid tumor Endometrium and MSI status determination
Endometrial NGS panel	DNA NGS Panel solid tumor Endometrium
Endometrial NGS panel	DNA NGS Panel solid tumor Endometrium and MSI status determination
Endomysium - anti-IgA antibodies	Anti-endomysium antibodies - IgA - serum
Endomysium - anti-IgG antibodies	Anti-endomysium antibodies - IgG - serum
Endoplasmic reticulum - anti-antibodies - screening	Anti-LKM antibodies - endoplasmic reticulum - screening - serum
Endoplasmic reticulum - anti-antibodies - typing	Anti-LKM antibodies - endoplasmic reticulum - typing - serum
Endosulfan: alpha, beta, ether, lactone, sulfate	Pesticides - screening and/or dosage - urine
Endrin	Pesticides - screening and/or dosage - urine
Enolemie	Ethyl alcohol - serum
Entamoeba histolytica	Amoebiasis - serology - serum
Entamoeba histolytica	Gastrointestinal Panel - direct diagnosis - PCR - Stools
Entecavir - plasma	
enteral transmission	Hepatitis E - IgG antibody - serum
enteral transmission	Hepatitis E - IgM antibody - serum
Enteramin	Serotonin - whole blood
Enteraggregative E. coli (EAEC)	Gastrointestinal Panel - direct diagnosis - PCR - Stools
Enterocytozoon bienersi	Cryptosporidies - Microsporidies - direct diagnosis - PCR
Enteropathogenic E. coli (EPEC)	Gastrointestinal Panel - direct diagnosis - PCR - Stools
Enterotoxigenic E. coli (ETEC) lt/st	Gastrointestinal Panel - direct diagnosis - PCR - Stools
Enterovirus	Enterovirus - direct diagnosis - PCR
Enterovirus	Enterovirus - direct diagnosis - PCR - amniotic fluid
Enterovirus	viral meningoencephalitis panel - direct diagnosis - PCR
Entérovirus	Enterovirus - direct diagnosis - PCR
Entérovirus	Enterovirus - direct diagnosis - PCR - amniotic fluid
Entérovirus	Respiratory Infection Panel - direct diagnosis - PCR
Enterovirus - direct diagnosis - PCR	
Enterovirus - direct diagnosis - PCR - amniotic fluid	
entry inhibitors	HIV - Human Immunodeficiency Virus - genotyping (envelope, tropism): antiretroviral
Entyvio ®	Vedolizumab - serum
Envarsus	Tacrolimus - FK-506
Envoplaquine	Anti-epidermal basement membrane and intracellular substance antibodies Typing
Eosinophil Cationic Protein	ECP - Eosinophil Cationic Protein

Epanutin	Phenytoin - serum
EPAS1	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
EPAS1	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
EPCAM	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
EPCAM	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
EPCAM	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
EPEC	Escherichia coli - serotyping by agglutination (e.g. E. coli K1; EPEC; O157, etc.)
EPGX	Pharmacogenetic Passport - PharmacoPass Expert
Epidermal - anti-antibodies	Anti-epidermal basement membrane and intracellular substance antibodies - serum
Epidermal basement membrane + intercellular substance - anti-antibodies	Anti-epidermal basement membrane and intracellular substance antibodies - serum
Epididymal protein 4	HE4- ROMA calculation - serum
Epinephrine	Catecholamines - free - Urine
Epinephrine	Free catécholamines
Epinephrine	Free catécholamines - children - Urine
Epitomax®	Topiramate - serum
Epivir®	Lamivudine - plasma
EPO	Erythropoietin - serum
Epstein-Barr virus	viral meningoencephalitis panel - direct diagnosis - PCR
Epstein-Barr Virus - EBV - direct diagnosis - PCR	
Epstein-Barr Virus - EBV - direct diagnosis - PCR - amniotic fluid	
Epstein-Barr Virus - EBV - IgA EA serology - serum	
Epstein-Barr Virus - EBV - IgG EA serology - serum	
Epstein-Barr Virus - EBV - MNI test - serum	
Epstein-Barr Virus - EBV - VCA IgG, EBNA, IgG serology - serum	
Epstein-Barr Virus - EBV - VCA IgG/IgM and EBNA IgG serology - serum	
Epstein-Barr Virus - EBV - VCA IgM serology - serum	
Epstein-Barr Virus - EBV - viral load - PCR	
ERBB2	DNA NGS Panel for cholangiocarcinoma
ERBB2	DNA NGS Panel solid tumor Breast
ERBB2	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
ERBB2	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
Erelzi ®	Etanercept - serum
ERG	RNA NGS Panel solid tumors, Pan-organ : 65 genes
erm	Non tuberculosis mycobacteria - PCR detection of macrolide and aminoglycoside
ERVE serology	Tick-borne encephalitis - TBE - IgG/IgM serology - serum
Erythema infectiosum	Parvovirus B19 - IgG/IgM serology - serum
Erythrocyte direct Coombs	
Erythrocyte folic acid	
Erythrocyte porphyrins	
Erythrocyte pyruvate kinase	Erythrocyte pyruvate kinase
Erythrocyte pyruvate kinase	
Erythrocytes folates	Erythrocyte folic acid
Erythrocytes morphological aspect	
Erythrocytes vitamin B9	Erythrocyte folic acid
Erythropoietin - serum	
Erythrovirus B19 - IgG serology	Parvovirus B19 - immunity test - serum
Erythrovirus B19 - IgG/IgM serology	Parvovirus B19 - IgG/IgM serology - serum
ESBL	Antibiotic multi-resistant bacteria (MRB) - screening
Escherichia coli - serotyping by agglutination (e.g. E. coli K1; EPEC; O157, etc.)	
Escherichia coli K1	Escherichia coli - serotyping by agglutination (e.g. E. coli K1; EPEC; O157, etc.)
Escitalopram - Serum	
Estazolam - Serum	
Etanercept - serum	
Ethanol	Ethyl alcohol
Ethanol	Ethyl alcohol - serum
Ethionamide resistance	Mycobacteria - PCR detection of mutations associated with resistance to isoniazid,
Ethosuximide - serum	
Ethyl alcohol	

Ethyl alcohol - serum	
Ethyl loflazepate - serum	
ETV6	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Euphylline	Theophylline - serum
EUROIMMUN EUROLINE Dot	IgE against multiple individual allergens - food allergens - Serum
EUROIMMUN EUROLINE Dot	IgE against multiple individual allergens - Food and Inhalant panels - Serum
EUROIMMUN EUROLINE Dot	IgE against multiple individual allergens - inhalant allergens - Serum
everolimus	Everolimus
Everolimus	
Eviplera	Emtricitabine - plasma
Eviplera	Rilpivirine - Serum
Eviplera	Tenofovir - plasma
EWSR1	RNA NGS Panel for soft tissue tumors : 20 genes
EWSR1	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Exanthem subitum - serology	Herpes virus type 6 (HHV6) - IgG serology
Excess fat in stools	Steatorrhoea - fat dosage - Stool
exchangeable copper / serum copper ratio.	Copper - exchangeable / free
exome	
Exome	Exome Isolated or syndromic intellectual disability and syndromic neurodevelopmental
Exome	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
Exome	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
Exome	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
Exome	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
Exome	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
Exome	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
Exome	Prenatal exome
Exome	réinterprétation d'exome
Exome	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
Exome	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
EXOME	exome
EXOME	Exome Congenital heart defects
EXOME	Exome Panel for chronic and hereditary pancreatitis
EXOME	Exome with interpretation interface
EXOME	Nephrology exome
EXOME	Oncogenetic exome
Exome Congenital heart defects	
Exome Isolated or syndromic intellectual disability and syndromic neurodevelopmental	
Exome Panel for chronic and hereditary pancreatitis	
Exome sequencing - Male infertility - Screening for mutations involved in male infertility	
Exome with interpretation interface	
Exon 14 skipping	RNA NGS Panel for lung cancers
Exons and flanking intronic regions	Cardiogenetic panel or exome
Exons and flanking intronic regions	exome
Exons and flanking intronic regions	Exome Congenital heart defects
Exons and flanking intronic regions	Exome Isolated or syndromic intellectual disability and syndromic neurodevelopmental
Exons and flanking intronic regions	Exome Panel for chronic and hereditary pancreatitis
Exons and flanking intronic regions	Exome with interpretation interface
Exons and flanking intronic regions	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
Exons and flanking intronic regions	Nephrology exome
Exons and flanking intronic regions	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
Exons and flanking intronic regions	Oncogenetic exome
Exons and flanking intronic regions	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
Exons and flanking intronic regions	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
Exons and flanking intronic regions	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
Exons and flanking intronic regions	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
Exons and flanking intronic regions	Prenatal exome
Exons and flanking intronic regions	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
Exons and flanking intronic regions	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,

Exotic mycoses	Fungus - Yeasts - screening
Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro- Extended phenotype	
EZR	RNA NGS Panel solid tumors, Pan-organ : 65 genes
F 1+2	Fragment 1+2 - prothrombin - plasma
Fabry disease	Lyso-Gb3
Factor 13	Factor XIII - plasma
Factor B - complement fraction - serum	
Factor I - complement fraction - serum	
Factor I - Fibrinogen - activity - plasma	
Factor I - Fibrinogen - antigen - plasma	
Factor II - anti-IgG antibodies	Anti-prothrombin antibodies - IgG - serum
Factor II - Prothrombin - activity - plasma	
Factor II - Prothrombin - G20210G>A mutation	
Factor IX - Antihæmophilic B - plasma	
Factor IX inhibitors	Anti-factor IX antibodies - screening - plasma
Factor V - Proaccelerin - plasma	
Factor V inhibitors	Anti-Factor V antibodies - screening - plasma
Factor V Leiden - p.Arg506Gln mutation	
Factor VII - anti-antibodies	Anti-factor VII antibodies - screening - plasma
Factor VII - Proconvertin - plasma	
Factor VII inhibitors	Anti-factor VII antibodies - screening - plasma
Factor VIII - Antihæmophilic A - plasma	
Factor VIII inhibitors	Anti-factor VIII antibodies - screening - plasma
Factor VIII-related antigen	Von Willebrand factor - plasma
Factor Willebrand protease activity - ADAMTS 13	
Factor X - Stuart-Prower - plasma	
Factor XI - Rosenthal - plasma	
Factor XI inhibitors	Anti-factor XI antibodies - screening - plasma
Factor XII - Hageman - plasma	
Factor XII inhibitors	Anti-factor XII antibodies - screening - plasma
Factor XIII - plasma	
Faecal test -Functional coprology - Stool	
Familial Mediterranean fever - MEFV gene - new generation sequencing	
Familial restrictive cardiomyopathy	Cardiogenetic panel or exome
Farr test	Anti-DNA antibodies double stranded - Farr test - serum
Fasciola Hepatica - serology	Distomatosis - screening serology - serum
Fast track	Séquencage des gènes BRCA1 et BRCA2
Fatty acids free non-esterified - serum	
Fatty acids of erythrocyte membrane - chromatography - Total EDTA blood - Fatty free acide	
FDP	FDP - fibrinogen and fibrin degradation products - plasma
FDP - fibrinogen and fibrin degradation products - plasma	
Felbamate - Serum	
Ferti Check	Juvenalis - Panel - FertiCheck - man
Ferti Check	Juvenalis - Panel - FertiCheck - woman
fetal red blood cells in maternal blood	Fetal-maternal hæmorrhage - EDTA whole blood
FETAL RH3 (RHE)	Fetal RH3(E) - genotyping
Fetal RH3(E) - genotyping	
FETAL RH4 (RHc)	Fetal RHc - genotyping
Fetal RHc - genotyping	
Fetal-maternal hæmorrhage - EDTA whole blood	
FGFR	RNA NGS panel for brain solid tumors
FGFR1	DNA NGS Panel for brain solid tumors
FGFR1	RNA NGS Panel solid tumors, Pan-organ : 65 genes
FGFR2	RNA NGS Panel for cholangiocarcinoma
FGFR2	RNA NGS Panel solid tumors, Pan-organ : 65 genes
FGFR3	RNA NGS Panel solid tumors, Pan-organ : 65 genes
FH	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-

FH	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
FH	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
FHSR	Hantavirus - IgG/IgM serology - serum
Fibrin monomers	Soluble complexes - plasma
Fibrin stabilising factor	Factor XIII - plasma
Fibrinogen - antigen	Factor I - Fibrinogen - antigen - plasma
Fibrinogen - functional	Factor I - Fibrinogen - activity - plasma
Fibrinogen - quantitation	Factor I - Fibrinogen - antigen - plasma
Fibrinogen and fibrin degradation products	FDP - fibrinogen and fibrin degradation products - plasma
Fibroma	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
FibroMAX - . - serum	
FibroMeter	
Fibrose kystique	Cystic fibrosis Detection of variations within the CFTR gene after massive
FibroTest®	NASH-FibroTest - serum
FIBROTEST	FibroMAX - . - serum
Fibrotest - Actitest - . - serum	
Fièvre de Malte (GB : Malta fever)	Brucellosis - Serology (Rose bengale + brucellacapt) - serum
Fifth disease - IgG serology	Parvovirus B19 - immunity test - serum
Fifth disease - IgG/IgM serology	Parvovirus B19 - IgG/IgM serology - serum
Filamentous fungus	Fungus - Yeasts - screening
Filaria	Filariasis - serology - serum
Filariasis - serology - serum	
FIN - anti-antibodies	Anti-intrinsic factor antibodies - serum
Fit-test	Haemoglobin - screening - Stool
Flavivirus	Dengue fever - IgG/IgM serology - serum
Flavivirus - dengue fever	Dengue fever - IgG/IgM serology - serum
FLCN	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
FLCN	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
Fleaborne typhus	Rickettsioses - R. conorii, R. typhi - serology - serum
Flecaine®	Flecainide - Serum
Flecainide - Serum	
Fletcher factor	Prekallikrein - activity - plasma
FLI1	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Flixabi	Infliximab
FLNC	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
Flow cytometry or circulating lymphocytes	Immunophenotyping of malignant hemopaties
Floxyfral®	Fluvoxamine - serum
FLU	Free cortisol - Compound F - urine
Fluconazole - Serum	
Fluke	Distomatosis - confirmation serology - serum
Fluke	Distomatosis - screening serology - serum
Fluoropyrimidines toxicity	Uracile - dosage(search a risk of toxicity to fluoropyrimidines)
Fluoroquinolones resistance	Mycobacteria - PCR detection of mutations associated with resistance to isoniazid,
Fluoxetine - serum	
Fluvoxamine - serum	
FMF	Familial Mediterranean fever - MEFV gene - new generation sequencing
FMR1	Fragile X - syndrome
foetal rhesus	
Folates	Folic acid - serum
Folic acid - serum	
Follicle-stimulating hormone	FSH - Follicle-stimulating hormone - serum
Follicle-stimulating hormone	FSH - Follicle-stimulating hormone - urines
Follicle-stimulating hormone	
folliculin	
Folliculin	Oestradiol - man/child - serum
Folliculin	Oestradiol - woman - serum
Folliculin	
Food intolerance	Nutritol - IgG food panel - 100 foods - serum

Food intolerance	Nutritol - IgG food panel - 25 foods - serum
Food intolerance	Nutritol - IgG food panel 287 foods - serum
Food intolerance	Nutritol - IgG food panel - 50 foods VEGAN - serum
Food intolerance	Nutritol - IgG food panel - 50 foods - serum
Fortum®	Ceftazidime - Serum
Fosfocine®	Fosfomycin - Serum
Fosfomycin - Serum	
Fosphenytoin	Phenytoin - serum
FOXO1	RNA NGS Panel for soft tissue tumors : 20 genes
FOXO1	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Fragile X - syndrome	
Fragment 1+2 - prothrombin - plasma	
Francisella tularensis - serology	Tularemia - Francisella tularensis - serology total antibodies - serum
Free beta-hCG chain	hCG - free beta chain - subunit - serum
Free calcium	Ionised calcium
Free catécholamines	
Free catécholamines - children - Urine	
Free cortisol - Compound F - urine	
Free lite	Kappa free light chains - serum
Free lite	Lambda free light chains - serum
Free methoxyamines	Methoxylated derivatives of catecholamines - Free metanephrines
Free prostate-specific antigen	PSA - Prostate-specific antigen free - serum
Free protein S - antigen - plasma	
Free protein S antigen	Free protein S - antigen - plasma
Free PS Ag	Free protein S - antigen - plasma
Free T3 - Free triiodothyronine - serum	
Free T4 - Free thyroxine - serum	
Free tetraiodothyronine	Free T4 - Free thyroxine - serum
Free thyroxine	Free T4 - Free thyroxine - serum
Free triiodothyronine	Free T3 - Free triiodothyronine - serum
Free water clearance	Water - free - clearance
Freezing point depression measurement	Osmolarity - serum
Freezing point depression measurement	Osmolarity - urines
Fructosamine - serum	
FSF	Factor XIII - plasma
FSH - Follicle-stimulating hormone	
FSH - Follicle-stimulating hormone - serum	
FSH - Follicle-stimulating hormone - urines	
FSH - Follicle-stimulating hormone 7 time	
FSH - Follicle-stimulating hormone 8 time	
FSH - Follicle-stimulating hormone 9 time	
FT3	Free T3 - Free triiodothyronine - serum
FT4	Free T4 - Free thyroxine - serum
FTA	Syphilis - Treponemique test - screening serology - serum
Full blood count	Leukocyte Count Detailed
Full Neonatal screen - . - blotting paper	
Functional coprology	Faecal test -Functional coprology - Stool
Functional investigation	Faecal test -Functional coprology - Stool
Fungal infection	Candida - Mannane antigen - serum
Fungus - Yeasts - identification	
Fungus - Yeasts - MIC: Minimum Inhibitory Concentration	
Fungus - Yeasts - screening	
Furosemide	
Furosemide - urine	
Fusion	RNA NGS panel for brain solid tumors
Fusion	RNA NGS Panel for cholangiocarcinoma
Fusion	RNA NGS Panel for pancreatic cancer
Fusion	RNA NGS Panel for soft tissue tumors : 20 genes

Fusion	RNA NGS Panel for thyroid cancer
Fusions	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Fycompa®	Perampanel - Serum - PERAMPANEL
G12C	DNA NGS Panel for cholangiocarcinoma
G12C	DNA NGS Panel for pancreatic cancer
G20210A mutation	Factor II - Prothrombin - G20210G>A mutation
G6PD	G6PD - Glucose-6- phosphate dehydrogenase
G6PD	Pharmacogenetic Passport - PharmacoPass Expert
G6PD - Glucose-6- phosphate dehydrogenase	
GAA	Guanidinoacetate and creatine - plasma
GAA	Guanidinoacetate and creatine - urine
Gabapentin - serum	
GAD65 - anti-antibodies	Anti-GAD antibodies - serum
GAG	Mucopolysaccharides - urines
Galactomannan	Aspergillosis - soluble antigen - testing - BAL
Galactomannan	Aspergillosis - soluble antigen - testing - serum
Galactose-1-Phosphate Uridyl transferase - EDTA whole blood	
Galactosemia	Galactose-1-Phosphate Uridyl transferase - EDTA whole blood
Gallstones/Saliva/Other - morphologic and infrared analysis	
GAM	IgA/G/M - Immunoglobulins A/G/M - serum
Gamma linoleic acid	Fatty acids of erythrocyte membrane - chromatography - Total EDTA blood - Fatty free
Gamma-hydroxybutyric acid	GHB - Gamma-hydroxybutyric acid - urines
Gamma-interferon	Interferon gamma - CSF
Ganciclovir - plasma	
Ganglioma	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
Gangliosides - anti-antibodies	Anti-glycolipid antibodies - CSF
Gangliosides - anti-antibodies	Anti-glycolipid antibodies - serum
Gastric	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
Gastrin - serum	
Gastroenteritis	Norovirus - direct diagnosis - PCR - Stools
Gastroenteritis	Norovirus - screening - Stools
Gastroenteritis	Rotavirus - rapid screening - Stools
Gastrointestinal Panel - direct diagnosis - PCR - Stools	
Gaucher - disease	Chitotriosidase - activity
Gaucher disease	Chitotriosidase - activity
GBM - anti-antibodies	Anti-glomerular basement membrane antibody - Serum
gène de résistance à la clarithromycine-PCR	Helicobacter pylori and clarithromycin resistance gene - direct diagnosis - PCR
Gene rearrangements (fusions) : ALK; FGFR2; FGFR3; RET; ROS1; EML4; KIF5B;	Circulating tumor DNA NGS Panel - Peripheral blood from Cell Free DNA tube : ambiant
Genetics of infertility	Exome sequencing - Male infertility - Screening for mutations involved in male infertility
Génétique	Séquencage des gènes BRCA1 et BRCA2
Genital ulcérations panel - direct diagnosis - PCR	
Génome	Genome by long-read sequencing (Nanopore) - EDTA whole blood
Genome by long-read sequencing (Nanopore) - EDTA whole blood	
Genotyping - HCV	Hepatitis C - genotyping
Genotyping of dihydropyrimidine dehydrogenase	
Genotyping of S and Z alpha-1-antitrypsin variants	Alpha-1 Antitrypsine - Testing for S and Z variants of the SERPINA1 gene
Gentalline	Gentamicine - serum
Gentalline®	Gentamicine - serum
Gentamicine - serum	
GH - Growth hormone	
GH - Growth hormone 10 time	
GH - Growth hormone 11 time	
GH - Growth hormone - Serum	
GH - Growth hormone 7 time	
GH - Growth hormone 8 time	
GH - Growth hormone 9 time	
GHRH	GHRH - Growth-Hormone-Releasing-Hormone - plasma
GHB - Gamma-hydroxybutyric acid - urines	

Ghrelin - acylated and non acylated - plasma	
GHRH - Growth-Hormone-Releasing-Hormone - plasma	
Giardia intestinalis	Gastrointestinal Panel - direct diagnosis - PCR - Stools
Giardia intestinalis	Parasitology - microscopy
Gilbert Syndrome - Screening for UGT1A1*28 (rs8175347) polymorphism	
GIST	PDGFR alpha - exons 12, 14, 18
Gliadin - anti-IgA antibodies	Anti deamidated gliadin peptides antibodies IgA - IgA - serum
Gliadin - anti-IgG antibodies	Anti deamidated gliadin peptides antibodies IgA - IgG - serum
Glibenclamide	Sulphonylureas - serum
Glibenclamide - serum	
Glibornuride	Sulphonylureas - serum
Glicazide	Sulphonylureas - serum
Glimepiride	Sulphonylureas - serum
Glipizide	Sulphonylureas - serum
Glivec	Imatinib
Global Check	Juvenalis - Panel - GlobalCheck
Globin - Screening for alpha or beta gene - postnatal	
Glomerular basement membrane - anti-antibodies	Anti-glomerular basement membrane antibody - Serum
Glucagon - plasma	
Glucocorticoid	Cortisol - Compound F - saliva
Glucocorticoid	Cortisol - Compound F - serum
Glucocorticoid	
Glucocorticoïde	
Glucophage®	Metformin
Glucose-6 phosphate dehydrogenase	G6PD - Glucose-6- phosphate dehydrogenase
Glutamate decarboxylase - anti-antibodies	Anti-GAD antibodies - CSF
Glutamate decarboxylase - anti-antibodies	Anti-GAD antibodies - serum
Glutamic acid decarboxylase - anti-antibodies	Anti-GAD antibodies - CSF
Glutamic acid decarboxylase - anti-antibodies	Anti-GAD antibodies - serum
Glutathione peroxidase - plasma	
Glycolipids - anti-antibodies	Anti-glycolipid antibodies - CSF
Glycolipids - anti-antibodies	Anti-glycolipid antibodies - serum
Glycopeptide-resistant enterococci	Antibiotic multi-resistant bacteria (MRB) - screening
Glycosaminoglycans	Mucopolysaccharides - urines
Glycosylated haemoglobin	Haemoglobin glycosylated - HbA1c - EDTA whole blood
Glycosylated proteins	Fructosamine - serum
GM1 - anti - antibodies	Anti-glycolipid antibodies - CSF
GM1 - anti-antibodies	Anti-glycolipid antibodies - serum
Goiffon	Faecal test -Functional coprology - Stool
Golimumab - serum	
Gonococcus	Neisseria gonorrhoeae - Detection by molecular amplification
Goodpasture's syndrome	Anti-glomerular basement membrane antibody - Serum
GOT2	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
GOT2	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
GP210	Anti-GP210 antibodies - serum
GPIIa antibodies	Anti-platelet antibodies bound or free - screening and identification
GPIbIX antibodies	Anti-platelet antibodies bound or free - screening and identification
GPIIbIIIa antibodies	Anti-platelet antibodies bound or free - screening and identification
GQ1b - anti-antibodies	Anti-glycolipid antibodies - CSF
GQ1b - anti-antibodies	Anti-glycolipid antibodies - serum
Granudoxy®	Doxycycline - Serum
GRE	Antibiotic multi-resistant bacteria (MRB) - screening
Growth hormone	GH - Growth hormone - Serum
Growth Hormone	GH - Growth hormone - Serum
Growth-Hormone-Releasing-Hormone - GH-RH	GHRH - Growth-Hormone-Releasing-Hormone - plasma
Guanidinoacetate and creatine - plasma	
Guanidinoacetate and creatine - urine	
Guthrie test	Full Neonatal screen - . - blotting paper

H63D mutation	Haemochromatosis - HFE gene - p.His63Asp mutation
HA test	Hyaluronic acid - pleural fluid
HA test	Hyaluronic acid - serum
HAD	ADH - Antidiuretic hormone
HAD	ADH - Antidiuretic hormone - urine
Haematological and molecular cytogenetics - FISH	
Haemochromatosis	Haemochromatosis - HFE gene - p.Cys282Tyr mutation
Haemochromatosis	Haemochromatosis - HFE gene - p.His63Asp mutation
Haemochromatosis	Haemochromatosis - HFE gene - p.Ser65Cys mutation
Haemochromatosis - HFE gene - p.Cys282Tyr mutation	
Haemochromatosis - HFE gene - p.His63Asp mutation	
Haemochromatosis - HFE gene - p.Ser65Cys mutation	
Haemoglobin	
Haemoglobin - electrophoresis - whole blood	
Haemoglobin - screening - Stool	
Haemoglobin anomaly	Haemoglobin - electrophoresis - whole blood
Haemoglobin electrophoresis	Haemoglobin - electrophoresis - whole blood
Haemoglobin glycosylated - HbA1c - EDTA whole blood	
Haemoglobin HbA1c fraction	Haemoglobin glycosylated - HbA1c - EDTA whole blood
Haemophilus ducreyi	Genital ulcérations panel - direct diagnosis - PCR
Haemophilus ducreyi - direct diagnosis - PCR	
Haemophilus ducreyi - screening	
Haemophilus influenzae	Haemophilus influenzae B - IgG serology - serum
Haemophilus influenzae B - IgG serology - serum	
Hageman - factor	Factor XII - Hageman - plasma
Hageman factor	Factor XII - Hageman - plasma
Halcion®	Triazolam - serum
Haldol®	Haloperidol
Haloperidol	
Hantan	Hantavirus - IgG/IgM serology - serum
Hantavirus - IgG/IgM serology - serum	
HAV - IgM antibodies	Hepatitis A - IgM antibody - serum
HAV - total antibodies	Hepatitis A - IgG immunity screening - serum
Havlane®	Loprazolam - Serum
HbA1c	Haemoglobin glycosylated - HbA1c - EDTA whole blood
HBc - IgM antibodies	Hepatitis B - anti-HBc antibodies - IgM - serum
HBc - total antibodies	Hepatitis B - total anti-HBc antibodies - serum
HBe - antigen	Hepatitis B - HBe antigen - serum
HBe - total antibodies	Hepatitis B - anti-HBe antibodies - serum
HBe - total antibodies - titration	Hepatitis B - anti-HBs antibodies - titration - serum
HBe antigen	Hepatitis B - HBe antigen - serum
HBOC	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
HBs - antigens	Hepatitis B - Qualitative HBs antigen - serum
HBs antigen	Hepatitis B - Qualitative HBs antigen - serum
HBs antigen quantification	Hepatitis B - HBs antigen - quantitative - serum
HBs antigen titration	Hepatitis B - HBs antigen - quantitative - serum
HBV - anti-viral resistance profile	Hepatitis B Genotyping and Antiviral Resistance Profile - serum
HBV - anti-HBc IgM antibodies	Hepatitis B - anti-HBc antibodies - IgM - serum
HBV - Anti-HBc IgM antibodies	Hepatitis B - anti-HBc antibodies - IgM - serum
HBV - HBe antigen	Hepatitis B - HBe antigen - serum
HBV - HBs antigen	Hepatitis B - Qualitative HBs antigen - serum
HBV - quantitative determination - viral load	Hepatitis B - viral load - PCR
HBV - total anti-HBc antibodies	Hepatitis B - total anti-HBc antibodies - serum
HBV - Total anti-HBc antibodies	Hepatitis B - total anti-HBc antibodies - serum
HBV - total anti-HBe antibodies	Hepatitis B - anti-HBe antibodies - serum
HBV - Total anti-HBe antibodies	Hepatitis B - anti-HBe antibodies - serum
HBV - total anti-HBs antibodies - titration	Hepatitis B - anti-HBs antibodies - titration - serum
HBV - Total anti-HBs antibodies - titration	Hepatitis B - anti-HBs antibodies - titration - serum

HBV genotyping	Hepatitis B Genotyping and Antiviral Resistance Profile - serum
HC protein	Alpha-1 Microglobulin - urine
HCB	Pesticides - screening and/or dosage - urine
hCG - free beta chain - subunit - serum	
hCG - free beta-subunit	hCG - free beta chain - subunit - serum
hcG dimeric molecule (alpha + beta) - biological fluid	
hCG dimers	Total HCG (HCG dimer + beta fraction) - serum
hCT	Calcitonin - aspiration biopsy
hCT	Calcitonin - Serum
HCV - genotyping	Hepatitis C - genotyping
HCV - quantitative determination - viral load	Hepatitis C - viral load - quantitative PCR
HCV - screening serology	Hepatitis C - screening serology - serum
HCV - viral RNA - PCR	Hepatitis C - viral RNA detection - PCR - serum
Hcy	Homocysteine - plasma
HDL cholesterol	HDL cholesterol - serum
HDL cholesterol - serum	
HDS	DNA fragmentation test - DNA decondensation - SCSA - Sperm
HDV	Hepatitis Delta - Viral load - PCR
HDV - antigen	Hepatitis Delta - antigen - serum
HDV - IgM antibodies	Hepatitis Delta - IgM antibody - serum
HDV - total antibodies	Hepatitis Delta - total antibodies - serum
HE4- ROMA calculation - serum	
Health Index	[-2]pro-PSA - Phi score calculation - serum
Helicobacter pylori - 13C urea breath test - expired gas	
Helicobacter pylori - antigen - Stools	
Helicobacter pylori - culture	
Helicobacter pylori - IgG serology - serum	
Helicobacter pylori and clarithromycin resistance gene - direct diagnosis - PCR	
Helminthes	Helminthes - Protozoaires - direct diagnosis - PCR - Stool
Helminthes - Protozoaires - direct diagnosis - PCR - Stool	
Hemigoxine®	Digoxine par immunoturbidimétrie (Abbott) - Sérum
Hemipralon® LP	Propranolol - Serum
Hemophilus	Haemophilus influenzae B - IgG serology - serum
Heparin - PF4 - anti-antibodies	Anti-PF4 antibodies - heparin - plasma
Heparin induced thrombocytopenia - functional test	
Heparin-induced thrombopaenia - screening	Anti-PF4 antibodies - heparin - plasma
Heparinaemia	Anti-Xa activity : Standard heparin - NFH - plasma
hépatite Delta	Hepatitis Delta - Viral load - PCR
Hepatite E	Hepatitis E - direct diagnosis - PCR
Hepatite E virus	Hepatitis E - direct diagnosis - PCR
Hepatitis A - IgG immunity screening - serum	
Hepatitis A - IgM antibody - serum	
Hepatitis B - anti-HBc antibodies - IgM - serum	
Hepatitis B - anti-HBe antibodies - serum	
Hepatitis B - anti-HBs antibodies - titration - serum	
Hepatitis B - HBe antigen - serum	
Hepatitis B - HBs antigen - quantitative - serum	
Hepatitis B - Qualitative HBs antigen - serum	
Hepatitis B - total anti-HBc antibodies - serum	
Hepatitis B - viral load - PCR	
Hepatitis B Genotyping and Antiviral Resistance Profile - serum	
Hepatitis C - Anti-viral resistance profile - serum	
Hepatitis C - confirmation - serum	
Hepatitis C - genotyping	
Hepatitis C - screening serology - serum	
Hepatitis C - viral load - quantitative PCR	
Hepatitis C - viral RNA detection - PCR - serum	
Hepatitis C confirmation	Hepatitis C - confirmation - serum

Hepatitis C resistance mutations screening	Hepatitis C - Anti-viral resistance profile - serum
Hepatitis Delta - antigen - serum	
Hepatitis Delta - IgM antibody - serum	
Hepatitis Delta - total antibodies - serum	
Hepatitis Delta - Viral load - PCR	
Hepatitis E - direct diagnosis - PCR	
Hepatitis E - IgG antibody - serum	
Hepatitis E - IgM antibody - serum	
Heptachlor	Pesticides - screening and/or dosage - urine
HER2 - Human Epidermal growth factor Receptor 2 - c-erbB-2 protein - serum	
HER2 ECD - extracellular domain	HER2 - Human Epidermal growth factor Receptor 2 - c-erbB-2 protein - serum
Hereditary amyloidosis	Amylose héréditaire à transthyréline - Etude moléculaire du gène TTR par séquençage
hereditary spherocytosis - screening (eosin-5'-maleimide (EMA) staining)	
Heroin	Opiates - dosage - plasma
Heroin	Opiates - dosage - urines
herpes gestationis factor	Anti-epidermal basement membrane and intracellular substance antibodies Typing
Herpes simplex 1/2	Genital ulcérations panel - direct diagnosis - PCR
Herpes simplex virus 1/2	Vesicular eruptions panel - direct diagnosis - PCR
Herpes simplex virus 1/2 - HSV - direct diagnosis - PCR	
Herpes simplex virus 1/2 - HSV - direct diagnosis - PCR - amniotic fluid	
Herpes simplex virus 1/2 - HSV - IgM serology - serum	
Herpes Simplex Virus-HSV1 IgG & HSV2 IgG - IgG serology - serum	
Herpes type 1 2	viral meningoencephalitis panel - direct diagnosis - PCR
Herpes Viridae	Herpes simplex virus 1/2 - HSV - IgM serology - serum
Herpes Viridae	Herpes Simplex Virus-HSV1 IgG & HSV2 IgG - IgG serology - serum
Herpes virus type 6 (HHV6) - direct diagnosis - PCR	
Herpes virus type 6 (HHV6) - IgG serology	
Herpes virus type 6 (HHV6) - IgM serology	
Herpes virus type 6 (HHV6) - viral load - PCR	
Herpes Virus type 7 - direct diagnosis - PCR	
Herpes virus type 8 (HHV8) - direct diagnosis - PCR	
Herpes virus type 8 (HHV8) - IgG serology - serum	
Herpes virus type 8 (HHV8) - viral load	
Herpes ZOSTER	Varicella zoster virus - IgM Serology - serum
Herpes zoster - IgM serology	Varicella zoster virus - IgM Serology - serum
Herpes zoster - immunity test - IgG antibodies	Varicella zoster virus - IgG serology - serum
Herpes zoster - viral DNA - PCR	Varicella zoster virus - direct diagnosis - PCR
HEV	Hepatitis E - direct diagnosis - PCR
HEV	Hepatitis E - IgG antibody - serum
HEV - IgG antibodies	Hepatitis E - IgG antibody - serum
HEV - IgM antibodies	Hepatitis E - IgM antibody - serum
HFSR hemorrhagic fever syndrome renal	Hantavirus - IgG/IgM serology - serum
Hg	Mercury
Hg	Mercury - Urine
hGH	GH - Growth hormone - Serum
HHV	Herpes virus type 6 (HHV6) - IgM serology
HHV-3 (human herpesvirus 3)	Varicella zoster virus - direct diagnosis - PCR - amniotic fluid
HHV-5	Cytomegalovirus - CMV - - direct diagnosis - PCR
HHV-5	Cytomegalovirus - CMV - - direct diagnosis - PCR - amniotic fluid
HHV-6 - molecular diagnosis	Herpes virus type 6 (HHV6) - direct diagnosis - PCR
HHV-6 - PCR	Herpes virus type 6 (HHV6) - direct diagnosis - PCR
HHV-6 - serology	Herpes virus type 6 (HHV6) - IgG serology
HHV-8 - molecular diagnosis	Herpes virus type 8 (HHV8) - direct diagnosis - PCR
HHV-8 - molecular diagnosis	Herpes virus type 8 (HHV8) - viral load
HHV-8 - PCR	Herpes virus type 8 (HHV8) - direct diagnosis - PCR
HHV-8 - serology	Herpes virus type 8 (HHV8) - IgG serology - serum
HHV6	Herpes virus type 6 (HHV6) - IgM serology
HHV6	Herpes virus type 6 (HHV6) - viral load - PCR

HHV6	viral meningoencephalitis panel - direct diagnosis - PCR
HHV7 - Herpes virus	Herpes Virus type 7 - direct diagnosis - PCR
HHV8 - PCR	Herpes virus type 8 (HHV8) - viral load
HIAA 5	5-HIAA - 5-hydroxyindoleacetic acid - Urine
High DNA Stainability	DNA fragmentation test - DNA decondensation - SCSA - Sperm
High¿grade serous ovarian carcinoma	Homologous recombination deficiency (HRD) and somatic mutations of the BRCA1 and
Hippurate	Hippuric acid - urine
Hippuric acid - urine	
Hirst serology	Influenza A and B - IgG serology - serum
Hirst test	Influenza A and B - IgG serology - serum
Histones - anti-antibodies	Anti-histone antibodies - serum
Histoplasma	Fungus - Yeasts - screening
HIT	Anti-PF4 antibodies - heparin - plasma
HIT	Heparin induced thrombocytopenia - functional test
HIT - screening	Anti-PF4 antibodies - heparin - plasma
HIT functional test	Heparin induced thrombocytopenia - functional test
HIT screening	Heparin induced thrombocytopenia - functional test
HIV	HIV2 - Human Immunodeficiency Virus - viral load - Plasma
HIV	HIV - Human Immunodeficiency Virus - genotype for antiretroviral resistance (integrase
HIV	HIV - Human Immunodeficiency Virus - genotyping (envelope, tropism): antiretroviral
HIV	HIV - Human Immunodeficiency Virus - viral load - CSF
HIV1 - Human Immunodeficiency Virus - viral load - Plasma	
HIV1/2 - Human Immunodeficiency Virus - confirmatory serology by immuno-transfer	
HIV1/2 - Human Immunodeficiency Virus - screening serology - serum	
HIV - Human Immunodeficiency Virus - genotype for antiretroviral resistance (integrase	
HIV - Human Immunodeficiency Virus - genotyping (envelope, tropism): antiretroviral	
HIV - Human Immunodeficiency Virus - genotyping (reverse transcriptase, protease and	
HIV - Human Immunodeficiency Virus - p24 antigen - serum	
HIV - Human Immunodeficiency Virus - viral load - CSF	
HIV - p24 Ag	HIV - Human Immunodeficiency Virus - p24 antigen - serum
HIV - viral load	HIV1 - Human Immunodeficiency Virus - viral load - Plasma
HIV-1 antigen	HIV - Human Immunodeficiency Virus - p24 antigen - serum
HIV-1 in CSF	HIV - Human Immunodeficiency Virus - viral load - CSF
HIV-1: Qualitative RNA detection, suspected primary infection - serum	
HIV-1/2 - confirmation serology	HIV1/2 - Human Immunodeficiency Virus - confirmatory serology by immuno-transfer
HIV-1/2 - screening control serology	HIV1/2 - Human Immunodeficiency Virus - screening serology - serum
HIV-1/2 - Western blot	HIV1/2 - Human Immunodeficiency Virus - confirmatory serology by immuno-transfer
HIV2 - Human Immunodeficiency Virus - viral load - Plasma	
HIV-1/2 immunoblot	HIV1/2 - Human Immunodeficiency Virus - confirmatory serology by immuno-transfer
HIV-2	HIV2 - Human Immunodeficiency Virus - viral load - Plasma
HIV-genotypic identification of ressitance to antiviral drugs	HIV - Human Immunodeficiency Virus - genotyping (reverse transcriptase, protease and
HIV-RNA viral load	HIV1 - Human Immunodeficiency Virus - viral load - Plasma
H+K+ ATPase - anti antibodies	Anti-proton pump antibodies - serum
HLA class 1	HLA-C genotyping (HLA de classe I) - Total EDTA blood
HLA Class 1	Class I HLA - HLA-A and HLA-B genotyping - EDTA whole blood
HLA class 2	Class II HLA - HLA-DQ and HLA-DR genotyping - EDTA whole blood
HLA class 2	HLA-C genotyping (HLA de classe I) - Total EDTA blood
HLA de classe 1 - génotypage HLA-C	HLA-C genotyping (HLA de classe I) - Total EDTA blood
HLA-A and HLA-B - genotyping	Class I HLA - HLA-A and HLA-B genotyping - EDTA whole blood
HLA-B27 - genotyping	
HLA-B27 antigen	HLA-B27 - genotyping
HLA-B27 genotyping	HLA-B27 - genotyping
HLA-B27 leukocyte and platelet antigens	HLA-B27 - genotyping
HLA-B27 typing	HLA-B27 - genotyping
HLA-C genotyping (HLA de classe I) - Total EDTA blood	
hMPV	Paediatric respiratory viruses (RSV / Metapneumovirus / Rhinovirus) - direct diagnosis -
HNF1B	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
HNF1B	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,

HOMA	Insulin - serum
HOMA + QUICKI - index - serum	
HOMA index	Insulin - serum
Homocysteinaemia	Homocysteine - plasma
Homocysteine - plasma	
Homogentisic acid	
Homologous recombination deficiency (HRD) and somatic mutations of the BRCA1 and	
Homologous recombination DNA repair defect	Homologous recombination deficiency (HRD) and somatic mutations of the BRCA1 and
Homovanilate	Homovanillic acid - Urine
Homovanillic acid - Urine	
HOXB13	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
HOXB13	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
HOXB13	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
Hp - 13C-urea breath tests	Helicobacter pylori - 13C urea breath test - expired gas
Hp - antigens	Helicobacter pylori - antigen - Stools
Hp - culture	Helicobacter pylori - culture
Hp - IgG serology	Helicobacter pylori - IgG serology - serum
HPV	Papillomavirus - HPV Détection and Typing of 19 high-risk HPV and 18 low-risk HPV -
HPV 16	Detection of the HPV16 genome in the context of anal cancer screening
HPV anal	Detection of the HPV16 genome in the context of anal cancer screening
HPV test for vaginal sampling or self-sampling	
H.pylori	Helicobacter pylori and clarithromycin resistance gene - direct diagnosis - PCR
HRAS	DNA NGS Panel for brain solid tumors
HRAS	DNA NGS Panel for thyroid cancer
HRD	DNA NGS Panel for pancreatic cancer
HRD - BRCA1 - BRCA2	Homologous recombination deficiency (HRD) and somatic mutations of the BRCA1 and
HRP - HRR	Homologous recombination deficiency (HRD) and somatic mutations of the BRCA1 and
HSV	Herpes simplex virus 1/2 - HSV - direct diagnosis - PCR
HSV	Herpes simplex virus 1/2 - HSV - direct diagnosis - PCR - amniotic fluid
HSV1	Genital ulcérations panel - direct diagnosis - PCR
HSV1	Vesicular eruptions panel - direct diagnosis - PCR
HSV 1	Herpes simplex virus 1/2 - HSV - direct diagnosis - PCR - amniotic fluid
HSV 1 and HSV 2	Herpes simplex virus 1/2 - HSV - direct diagnosis - PCR
HSV1 and HSV2	viral meningoencephalitis panel - direct diagnosis - PCR
HSV 2	Herpes simplex virus 1/2 - HSV - direct diagnosis - PCR - amniotic fluid
HSV 4	Epstein-Barr Virus - EBV - VCA IgG/IgM and EBNA IgG serology - serum
HSV2	Genital ulcérations panel - direct diagnosis - PCR
HSV2	Vesicular eruptions panel - direct diagnosis - PCR
HSV-1	Herpes simplex virus 1/2 - HSV - direct diagnosis - PCR
HSV-1 and -2 - IgG serology	Herpes Simplex Virus-HSV1 IgG & HSV2 IgG - IgG serology - serum
HSV-1 and -2 - IgM serology	Herpes simplex virus 1/2 - HSV - IgM serology - serum
HSV5	Cytomegalovirus - CMV - - IgG/IgM serology - serum
HSV6	Herpes virus type 6 (HHV6) - IgG serology
HT	Serotonin - whole blood
HT21	Trisomy 21 - 1st trimester combined screening
HT21	Trisomy 21 - second-trimester maternal serum markers - 2 parameters
HT21	Trisomy 21 - second-trimester maternal serum markers - 3 parameters
hTg	Thyroglobulin
HTLV 1 and 2 Anti antibodies	HTLV I/II - screening serology - serum
HTLV I/II - confirmation serology - serum	
HTLV I/II - screening serology - serum	
HTLV I/II Qualitative PCR - EDTA whole blood	
HTLV-1 and -2 - anti-antibodies	HTLV I/II - confirmation serology - serum
Hu - anti-antibodies	Anti-neuron antibodies - serum
Hukyndra	Adalimumab
Hulio	Adalimumab
Human babesiosis - serology - serum	
Human chorionic gonadotropin - whole molecule	Total HCG (HCG dimer + beta fraction) - serum

Human ehrlichiosis - IgG/IgM serology - serum	
Human Enterovirus/rhinovirus	Respiratory Infection Panel - direct diagnosis - PCR
Human Growth Hormone	GH - Growth hormone - Serum
Human Herpes Virus-5	Cytomegalovirus - CMV - - direct diagnosis - PCR
Human Herpes Virus-5	Cytomegalovirus - CMV - - direct diagnosis - PCR - amniotic fluid
Human immunodeficiency virus - profile of resistance to antiviral drug	HIV - Human Immunodeficiency Virus - genotyping (envelope, tropism): antiretroviral
Human immunodeficiency virus - antiviral resistance profile	HIV - Human Immunodeficiency Virus - genotyping (reverse transcriptase, protease and
Human Metapneumovirus	Respiratory Infection Panel - direct diagnosis - PCR
Human papillomavirus (HPV) primary screening	Detection of high-risk HPV nucleic acids using a molecular technique as part of primary
Human Parainfluenza Virus 1, 2, 3 - serology	Parainfluenzae 1, 2, 3 - IgG serology - serum
Human tissue transglutaminase - IgA - anti-antibodies	Human tissue transglutaminase antibody - IgA - serum
Human tissue transglutaminase - IgG - anti-antibodies	Human tissue transglutaminase antibody - IgG - serum
Human tissue transglutaminase antibody - IgA - serum	
Human tissue transglutaminase antibody - IgG - serum	
Humira	Adalimumab
HVA	Homovanillic acid - Urine
Hyaluronate	Hyaluronic acid - pleural fluid
Hyaluronate	Hyaluronic acid - serum
Hyaluronic acid - pleural fluid	
Hyaluronic acid - serum	
Hydatid cyst - echinococcus multilocularis	Echinococcosis - confirmation serology - serum
Hydatid cyst - echinococcus multilocularis	Echinococcosis - screening serology - serum
Hydatidosis	Echinococcosis - confirmation serology - serum
Hydatidosis	Echinococcosis - screening serology - serum
Hydrocortisone	Cortisol - Compound F - saliva
Hydrocortisone	Cortisol - Compound F - serum
Hydrocortisone	
Hydroquinidine - Serum	
Hydroxychloroquine - EDTA whole blood	
Hydroxyindoleacetic - 5	5-HIAA - 5-hydroxyindoleacetic acid - Urine
Hydroxyrisperidone	Risperidone
Hydroxytryptamine - whole blood	Serotonin - whole blood
Hypertrophic cardiomyopathy	Cardiogenetic panel or exome
Hyrmoz	Adalimumab
I Factor	Factor I - complement fraction - serum
IA-2 - anti-antibodies	Anti-tyrosine phosphatase antibodies - IA-2 - serum
IAA	Anti-insulin antibodies - serum
IAT - screening/identification	Indirect antiglobulin test - IAT - screening/identification
ICA	Anti-islet cell antibodies - ICA - serum
Idacio	Adalimumab
IDH1	DNA NGS Panel for brain solid tumors
IDH1	DNA NGS Panel for cholangiocarcinoma
IDH2	DNA NGS Panel for brain solid tumors
Idiopathic extramembranous glomerulonephritis	anti-PLA2R antibodies
IFCC	Carbohydrate-deficient transferrin - serum
IFN a- neutralizing antibody	Interferon alpha - neutralizing antibody - serum
IFN-a	Interferon alpha - biological fluid
IFN-a	Interferon alpha - CSF
IFN-a	Interferon alpha - serum
IFNL3	Pharmacogenetic Passport - PharmacoPass Expert
IFS	Immunobinding - serum
IFS	Protein electrophoresis and Immunobinding - serum
IgA - anti-antibodies	Anti-IgA antibodies - serum
IgA - Immunoglobulin A - IgA1 and IgA2 subclasses - serum	
IgA - Immunoglobulin A - serum	
IgA - Immunoglobulin A secretory - Stool	
IgA subclasses - IgA1/2	IgA - Immunoglobulin A - IgA1 and IgA2 subclasses - serum
IgA/G/M - Immunoglobulins A/G/M - serum	

IgAS	IgA - Immunoglobulin A secretory - Stool
IgD - immunoglobulins D - serum	
IgE against multiple individual allergens - food allergens - Serum	
IgE against multiple individual allergens - Food and Inhalant panels - Serum	
IgE against multiple individual allergens - inhalant allergens - Serum	
IgE (RAST) - medications - single allergen	Specific IgE medicines - single allergen - GSD Gold Standard Diagnostic - serum,
IgE (RAST) - medications - single allergen	Specific IgE medicines - single allergen-Phadia
IGF-1 - Somatomedin C - Serum	
IGF-2 - serum	
IGF-BP3	IGFBP-3 - Insulin-like Growth Factor Binding protein-3 - serum
IGFBP-3 - Insulin-like Growth Factor Binding protein-3 - serum	
IgG1 subclasses	IgG - immunoglobulins G - IgG1 subclasses - serum
IgG - immunoglobulins G - CSF	
IgG - immunoglobulins G - IgG1 subclasses - serum	
IgG - immunoglobulins G - IgG2 subclasses - serum	
IgG - immunoglobulins G - IgG3 subclasses - serum	
IgG - immunoglobulins G - IgG4 subclasses - serum	
IgG - immunoglobulins G - serum	
IgG Allergens	SPECIFIC IgG - serum
IgG antibody avidity - CMV	Cytomegalovirus - CMV - IgG avidity test - serum
IgG avidity index - Rubella	Rubella - antibody avidity - IgG - serum
IgG2 subclasses	IgG - immunoglobulins G - IgG2 subclasses - serum
IgG-RAST	SPECIFIC IgG - serum
IgG3 subclasses	IgG - immunoglobulins G - IgG3 subclasses - serum
IgG4	IgG - immunoglobulins G - IgG4 subclasses - serum
IgG4 Allergens	SPECIFIC IgG4 - serum
IgG4 subclasses	IgG - immunoglobulins G - IgG4 subclasses - serum
IgM - immunoglobulins M - serum	
IgM antibodies - HAV	Hepatitis A - IgM antibody - serum
IGRA	Interferon gamma production - Quantiferon® TB Gold Plus
IL-1 beta	Interleukin (IL)-1 beta
IL-1Beta	Interleukin (IL)-1 beta
IL-5	Interleukin (IL)-5
IL-6	Interleukin (IL)-6
IL-8	Interleukin (IL)-8
IL28B genotyping	Interleukin 28B (IL28B polymorphisme C/T rs 12979860) - genotyping - EDTA whole
Imatinib	
immu check	Juvenalis - ImmuCheck
Immunobinding - serum	
immunoblot	HTLV I/II - confirmation serology - serum
Immunoblot - HCV	Hepatitis C - confirmation - serum
Immunoelectrophoresis	Immunobinding - serum
Immunoelectrophoresis	Protein electrophoresis and Immunobinding - serum
Immunoglobulin A - anti-antibodies	Anti-IgA antibodies - serum
Immunoglobulins A	IgA - Immunoglobulin A - serum
Immunoglobulins A- IgA1/2 subclasses	IgA - Immunoglobulin A - IgA1 and IgA2 subclasses - serum
Immunoglobulins D	IgD - immunoglobulins D - serum
Immunoglobulins G	IgG - immunoglobulins G - CSF
Immunoglobulins G	IgG - immunoglobulins G - serum
Immunoglobulins G/A/M quantitation	IgA/G/M - Immunoglobulins A/G/M - serum
Immunoglobulins M	IgM - immunoglobulins M - serum
Immunological fibrinogen	Factor I - Fibrinogen - antigen - plasma
Immunophenotyping: Lymphocytes B - CD19 - EDTA whole blood	
Immunophenotyping: Lymphocytes CD3-CD4-CD8 - EDTA whole blood	
Immunophenotyping: Lymphocytes CD56+ / CD16+ - Natural Killer Cells - EDTA whole	
Immunophenotyping: Lymphocytes NK/CD57 - EDTA whole blood	
Immunophenotyping of malignant hemopaties	
Immunotherapy	DNA NGS Panel solid tumor Colon

Immunotherapy	DNA NGS Panel solid tumor Colon and MSI status determination
Immunotherapy	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
IMN	Epstein-Barr Virus - EBV - IgA EA serology - serum
IMN	Epstein-Barr Virus - EBV - IgG EA serology - serum
IMN	Epstein-Barr Virus - EBV - VCA IgG, EBNA, IgG serology - serum
IMN	Epstein-Barr Virus - EBV - VCA IgG/IgM and EBNA IgG serology - serum
IMN	Epstein-Barr Virus - EBV - VCA IgM serology - serum
Imraldi	Adalimumab
Imupro	Nutritol - IgG food panel 287 foods - serum
Index Kappa - CSF + serum	
Indirect antiglobulin test - IAT - screening/identification	
Indirect Coombs	Indirect antiglobulin test - IAT - screening/identification
INF-alpha	Interferon alpha - biological fluid
INF-alpha	Interferon alpha - CSF
INF-alpha	Interferon alpha - serum
INF-alpha - neutralizing antibody	Interferon alpha - neutralizing antibody - serum
INF-gamma	Interferon gamma - CSF
Infection tuberculeuse latente	Interferon gamma production - QuantiFERON® TB Gold Plus
Infectious mononucleosis	Epstein-Barr Virus - EBV - IgA EA serology - serum
Infectious mononucleosis	Epstein-Barr Virus - EBV - IgG EA serology - serum
Infectious mononucleosis	Epstein-Barr Virus - EBV - VCA IgG, EBNA, IgG serology - serum
Infectious mononucleosis	Epstein-Barr Virus - EBV - VCA IgG/IgM and EBNA IgG serology - serum
Infectious mononucleosis	Epstein-Barr Virus - EBV - VCA IgM serology - serum
Infertility exome	Exome sequencing - Male infertility - Screening for mutations involved in male infertility
Infertility gene panel	Exome sequencing - Male infertility - Screening for mutations involved in male infertility
Inflammation marker	Serum calprotectin - Serum
Inflixtra	Infliximab
Infliximab	Infliximab
Infliximab	
Influenza	Influenza A and B - IgG serology - serum
Influenza - direct diagnosis - PCR	
Influenza A	Respiratory Infection Panel - direct diagnosis - PCR
Influenza A - H1	Respiratory Infection Panel - direct diagnosis - PCR
Influenza A - H1-09	Respiratory Infection Panel - direct diagnosis - PCR
Influenza A - H3	Respiratory Infection Panel - direct diagnosis - PCR
Influenza A and B - IgG serology - serum	
Influenza A and B - serology	Influenza A and B - IgG serology - serum
Influenza B	Respiratory Infection Panel - direct diagnosis - PCR
INH	INH - Isoniazid - Serum
INH - Isoniazid - Serum	
INH B	Inhibin B - man
INH B	Inhibin B - woman
Inhibin A - serum	
Inhibin A - ultrasensitive - serum	
Inhibin B - Child - serum	
Inhibin B - man	
Inhibin B - woman	
Inhibiteurs de PARP	Séquencage des gènes BRCA1 et BRCA2
Inhibitor of type 1 of the Plasminogen activator	PAI1 - Plasminogen activator inhibitor-1 - plasma
INOVELON®	Rufinamide - Serum
Insulin	
Insulin 10 time	
Insulin 11 time	
Insulin 12 time	
Insulin 13 time	
Insulin - anti-antibodies	Anti-insulin antibodies - serum
Insulin - serum	
Insulin 9 time	

Insulin Growth Factor	IGF-1 - Somatomedin C - Serum
Insulin like Growth Factor	IGF-1 - Somatomedin C - Serum
Insulin resistance	HOMA + QUICKI - index - serum
Insulin-like Growth Factor Binding protein-3	IGFBP-3 - Insulin-like Growth Factor Binding protein-3 - serum
Insulinaemia	Insulin - serum
insuline	
Intact N-Terminal propeptide of type I procollagen	P1NP - Intact N-Terminal Propeptide of Type I Procollagen - serum
Intact procollagen type I (N-propeptide)	P1NP - Intact N-Terminal Propeptide of Type I Procollagen - serum
INTELLECTUAL DISABILITY	Exome Isolated or syndromic intellectual disability and syndromic neurodevelopmental
Intercellular substance + Epidermal basement membrane - anti-antibodies	Anti-epidermal basement membrane and intracellular substance antibodies - serum
Interferon alpha - biological fluid	
Interferon alpha - CSF	
Interferon alpha - neutralizing antibody - serum	
Interferon alpha - serum	
Interferon gamma - CSF	
Interferon gamma production - Quantiferon® TB Gold Plus	
Interferon gamma production blood test	Interferon gamma production - Quantiferon® TB Gold Plus
Interferon Gamma Release Assays	Interferon gamma production - Quantiferon® TB Gold Plus
Interleukin 28B (IL28B polymorphisme C/T rs 12979860) - genotyping - EDTA whole blood	
Interleukin (IL)-10	
Interleukin (IL)-10 - .	
Interleukin (IL)-1 beta	
Interleukin (IL)-2 - receptor	
Interleukin (IL)-5	
Interleukin (IL)-6	
Interleukin (IL)-6 - .	
Interleukin (IL)-8	
Interleukin-2 receptor	Interleukin (IL)-2 - receptor
Interleukine	TNF-alpha - Tumour Necrosis Factor
Interleukines	Interleukin (IL)-10
Interleukines	Interleukin (IL)-1 beta
Interleukines	Interleukin (IL)-6
Interleukines	Interleukin (IL)-8
Interleukines	TNF-alpha - Tumour Necrosis Factor
Intrathecal IgG synthesis	Isoelectric focusing - CSF + serum
Intrathecal synthesis of free light chains	Index Kappa - CSF + serum
Intrinsic factor - anti-antibodies	Anti-intrinsic factor antibodies - serum
Invasive aspergillosis	Beta-D-Glucan antigen - screening
Invasive fungal infection	Aspergillosis - soluble antigen - testing - serum
Invasive fungal infection	Beta-D-Glucan antigen - screening
Iodine - urine	
Iodine concentration	Iodine total - serum
Iodine overload	Iodine - urine
Iodine total - serum	
Ionised calcium	
ions	Electrolytes stool - Stool
Iron - saturation percentage - serum	
Iron - serum	
Iron - total binding capacity - serum	
Iron concentration	Iron - serum
Isentress®	Raltegravir - Serum
Isocoproporphyrins	Porphyrins - stool
Isocoproporphyrines	Porphyrins - Urines
Isoelectric focusing - CSF + serum	
Isoelectric focussing with IgG index	Isoelectric focusing - CSF + serum
Isoenzymes - alkaline phosphatase	Alkaline phosphatases - isoenzymes - serum
Isoniazid	INH - Isoniazid - Serum
Isoniazid resistance	Mycobacteria - PCR detection of mutations associated with resistance to isoniazid,

Isoporphyrins	Porphyryns - Urines
Isoprotoporphyryns	Porphyryns - stool
Isospora	Parasitology - microscopy
Isosporose	Parasitology - microscopy
IST	IST PMA Expert & Multiplex PCR detection & Urogenital mycoplasma, C. trachomatis, N.
IST PMA Expert & Multiplex PCR detection & Urogenital mycoplasma, C. trachomatis, N.	
itch mite	scabies - screening
ITK	DNA NGS Panel for cholangiocarcinoma
ITK	DNA NGS Panel for pancreatic cancer
ITK	DNA NGS Panel for prostate cancer
ITK	DNA NGS Panel for thyroid cancer
ITK	DNA NGS Panel solid tumor Breast
ITK	RNA NGS Panel for cholangiocarcinoma
ITK	RNA NGS Panel for lung cancers
ITK	RNA NGS Panel for pancreatic cancer
ITK	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Itraconazole and hydroxylated metabolite - Serum	
Ixel®	Milnacipran - Serum
Jaeken syndrom	Adenylosuccinase - deficiency syndrome screening - urine
JAK2 - gene - VAL617PHE (V617F) mutation	
Jansky-Bielschowsky disease	Lipofuscinosis Neuronal Ceroide enzymatic deficiency - EDTA whole blood
Janus kinase 2 - mutation V617F - c.1849G>T	JAK2 - gene - VAL617PHE (V617F) mutation
JCV	Polyomavirus JC - direct diagnosis - PCR
Jervell and Lange-Nielsen syndrome (JLNS)	Cardiogenetic panel or exome
Juvenalis - AgeCheck - man	
Juvenalis - AgeCheck - woman	
Juvenalis - DigestCheck - Stool	
Juvenalis - ImmuCheck	
Juvenalis - OxyCheck	
Juvenalis - Panel - FertiCheck - man	
Juvenalis - Panel - FertiCheck - woman	
Juvenalis - Panel - GlobalCheck	
Juvenalis - Panel - Neurocheck	
Juvenalis - Panel - ThyroCheck+	
Juvenalis - Panel - VeggieCheck	
K	Electrolytes stool - Stool
Kala-azar	Leishmaniasis - direct diagnosis - PCR
Kappa - Bence Jones	Kappa free light chains - serum
Kappa - Bence Jones	Kappa free light chains - urine
Kappa Bence Jones	Kappa free light chains - serum
Kappa Bence Jones	Kappa free light chains - urine
Kappa free light chains - serum	
Kappa free light chains - urine	
Kappa free-running chains in the LCR	Index Kappa - CSF + serum
Kappa index	Index Kappa - CSF + serum
KCNH2	Cardiogenetic panel or exome
KCNQ1	Cardiogenetic panel or exome
Keppra®	Levetiracetam - serum
Ketoconazole - Serum	
Ketone bodies - Deproteinized heparin whole blood sample or CSF	
KIAA1549	RNA NGS panel for brain solid tumors
KIAA1549	RNA NGS Panel solid tumors, Pan-organ : 65 genes
kidney	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
KIF5B	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Killer-Cell Immunoglobulin-like Receptors	KIR Génotypage KIR - Total EDTA blood
KIR	KIR Génotypage KIR - Total EDTA blood
KIR Génotypage KIR - Total EDTA blood	
KIT - PDGFRA - BRAF - TKI - ITK - GIST	DNA NGS Panel solid tumor GIST

Kivexa®	Abacavir - plasma
Kleihauer - test	Fetal-maternal haemorrhage - EDTA whole blood
KRAS	DNA NGS Panel for brain solid tumors
KRAS	DNA NGS Panel for cholangiocarcinoma
KRAS	DNA NGS Panel for pancreatic cancer
KRAS	DNA NGS Panel for thyroid cancer
KRAS - BRAF - NRAS - POLE	DNA NGS Panel solid tumor Colon and MSI status determination
KSHV - IgG serology	Herpes virus type 8 (HHV8) - IgG serology - serum
KSHV - PCR	Herpes virus type 8 (HHV8) - direct diagnosis - PCR
KSHV - PCR	Herpes virus type 8 (HHV8) - viral load
Ku - anti-antibodies	Myosite Markers - serum
Kufs disease	Lipofuscinosis Neuronal Ceroide enzymatic deficiency - EDTA whole blood
Lactate	Ketone bodies - Deproteinized heparin whole blood sample or CSF
Lactate	Lactic acid
Lactate	Lactic acid - CSF
Lactic acid	Ketone bodies - Deproteinized heparin whole blood sample or CSF
Lactic acid	
Lactic acid - CSF	
Lactic dehydrogenase	LDH - Lactate dehydrogenase
Lactic dehydrogenase	LDH - Lactate dehydrogenase - serum
Lactic dehydrogenase - Isoenzymes	LDH - Lactate dehydrogenase - isoenzymes
lactose primary intolerance - Genetic characterization	
Lambda - Bence Jones	Lambda free light chains - serum
Lambda - Bence Jones	Lambda free light chains - urine
Lambda Bence Jones	Lambda free light chains - serum
Lambda Bence Jones	Lambda free light chains - urine
Lambda free light chains - serum	
Lambda free light chains - urine	
Lamictal®	Lamotrigine - serum
Lamivudine - plasma	
Lamotrigine - serum	
Laroxyl®	Amitriptyline - Serum
Lasilix®	Furosemide
Lasilix®	Furosemide - urine
Latex IgE (RAST) - single allergen	Specific IgE : latex - single allergen
LATS	Anti-TSH receptor antibodies
LBI	Bartonellosis - B. henselae - B. quintana - IgG serology - serum
LBP - serum	
LC1 - anti-liver cytosol type 1 antibodies	Anti-liver cytosol type 1 antibodies - LC1 - confirmation - serum
LC1 anti cytosol Abs	Anti-liver cytosol type 1 antibodies - LC1 - screening - serum
LDH	LDH - Lactate dehydrogenase
LDH	LDH - Lactate dehydrogenase - serum
LDH - Isoenzymes	LDH - Lactate dehydrogenase - isoenzymes
LDH - Lactate dehydrogenase	
LDH - Lactate dehydrogenase - isoenzymes	
LDH - Lactate dehydrogenase - serum	
LDH isoenzymes	LDH - Lactate dehydrogenase - isoenzymes
LDL cholesterol	Direct LDL cholesterol - serum
Lead	
Lead - whole blood	
Lead concentration	Lead - whole blood
Lead overload	Lead
legionella	Legionellosis- IgG screening serology - serum
Legionella pneumophila	Respiratory Infection Panel - direct diagnosis - PCR
Legionella pneumophila - confirmation serology	Legionellosis- - serogroups 1 to 6 - IgG confirmation serology - serum
Legionella pneumophila - PCR	Legionellosis- - direct diagnosis - PCR
Legionella pneumophila - screening serology	Legionellosis- IgG screening serology - serum
Legionella pneumophila antigen	Legionellosis- antigenes LP1 - urines

Legionellosis- - direct diagnosis - PCR	
Legionellosis- - serogroups 1 to 6 - IgG confirmation serology - serum	
Legionellosis- antigenes LP1 - urines	
Legionellosis- IgG screening serology - serum	
Leishmaniasis	Leishmaniasis - direct diagnosis - PCR
Leishmaniasis - direct diagnosis - PCR	
Leishmania	Leishmaniasis - direct diagnosis - PCR
Leponex®	Clozapine
Lepticur - Serum	
Leptin - serum	
Leptospirosis	Leptospirosis - IgM serology - serum
Leptospirosis - IgM serology - serum	
Leukemia chronic myelomonocytic	Immunophenotyping of malignant hemopaties
Leukocidin	PVL toxin detection (Staphylococcus aureus) - PCR on strain
Leukocyte Count Detailed	
Levetiracetam - serum	
Levofloxacin - Serum	
Leydig cells - anti-antibodies	Anti-testicular antibodies - serum
LH - Luteinising hormone	
LH - Luteinising hormone - serum	
LH - Luteinising hormone - urines	
LH - Luteinising hormone 7 time	
LH - Luteinising hormone 8 time	
LH - Luteinising hormone 9 time	
Li	Lithium par spectrophométrie - serum
Libmyris	Adalimumab
Likozam	Clobazam - Serum
Lindane	Pesticides - screening and/or dosage - urine
linear perfluorooctanoïc acid (PFOA)	Per- and polyfluoroalkyl compounds - 7 compounds panel - Serum
Lipase - serum	
Lipase - urine	
Lipase activity	Lipase - serum
Lipase activity	Lipase - urine
Lipase concentration	Lipase - serum
Lipid dosage	Steatorrhoea - fat dosage - Stool
Lipofuscinosis Neuronal Ceroide enzymatic deficiency - EDTA whole blood	
Lipoprotein a	Lipoprotein Lp(a) - serum
Lipoprotein A1	Apolipoprotein A1 - serum
Lipoprotein B	Apolipoprotein B - serum
Lipoprotein C3	Apolipoprotein C3 - serum
Lipoprotein Lp-PLA2 - serum	
Lipoprotein Lp(a) - serum	
Lipoprotein phospho-lipase A2	Lipoprotein Lp-PLA2 - serum
Lipoprotein-associated phospholipase A2	Lipoprotein Lp-PLA2 - serum
Liquid biopsy	Circulating tumor DNA NGS Panel - Peripheral blood from Cell Free DNA tube : ambiant
Litatop	Specific IgE : food allergen testing - Trophatop® children - fx26/fx27/fx28
Litatop	Specific IgE : Inhalant allergen testing - Phadiatop®
Litatop	Specific IgE - Respiratory and food allergen screen -Phadiatop® infant
Litatop®,	Specific IgE : food allergen testing - Trophatop® adult - fx5/fx24/fx25
Lithium concentration	Lithium par spectrophométrie - serum
Lithium par spectrophométrie - serum	
Lithocholic acid	Bile acids - chromatography - serum
Lithocholic acid	Bile acids - serum
liver	Anti-liver cytosol type 1 antibodies - LC1 - screening - serum
LKM - anti-antibodies screening	Anti-LKM antibodies - endoplasmic reticulum - screening - serum
LKM - anti-antibodies typing	Anti-LKM antibodies - endoplasmic reticulum - typing - serum
LMMC	Immunophenotyping of malignant hemopaties
Loa loa filariasis	Filariasis - serology - serum

Loci A and B - Class I HLA	Class I HLA - HLA-A and HLA-B genotyping - EDTA whole blood
Locus C -HLA de classe 1	HLA-C genotyping (HLA de classe I) - Total EDTA blood
Locus D - Class II HLA	Class II HLA - HLA-DQ and HLA-DR genotyping - EDTA whole blood
Logryx®	Minocycline - Serum
Long QT syndrome	Cardiogenetic panel or exome
Long reads	Genome by long-read sequencing (Nanopore) - EDTA whole blood
Loprazolam - Serum	
Lopressor®	Metoprolol - Serum
Loxapac®	Loxapine - Serum
Loxapine - Serum	
Lp(a)	Lipoprotein Lp(a) - serum
LPA	Lipoprotein Lp(a) - serum
LPPLA2	Lipoprotein Lp-PLA2 - serum
LPS	LBP - serum
LRBI	Bartonellosis - B. henselae - B. quintana - IgG serology - serum
LRBI	Bartonellosis - direct diagnosis - PCR
LSD - Lysergic acid diethylamide	
LSD - Lysergic acid diethylamide - biological fluid	
LSD - Lysergic acid diethylamide - Urine	
LSTA	Anti-TSH receptor antibodies
LTH	Prolactin - serum
lung	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
Lung	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
Lung cancer	DNA NGS Panel solid tumor Lung
Lung cancer	DNA NGS Panel solid tumor Lung extended
Lung cancer	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Lung NGS panel	DNA NGS Panel solid tumor Lung
Lung NGS panel	DNA NGS Panel solid tumor Lung extended
Lung RNA-seq	RNA NGS Panel for lung cancers
Luteal hormone	Progesterone - serum
Luteinising hormone	LH - Luteinising hormone - serum
Luteinising hormone	LH - Luteinising hormone - urines
Luteinizing Hormone	LH - Luteinising hormone - serum
Luteinizing Hormone	LH - Luteinising hormone - urines
Lutropin	LH - Luteinising hormone - serum
Lutropin	LH - Luteinising hormone - urines
Lyme	Borreliosis - direct diagnosis - PCR
Lyme	Borreliosis - IgG confirmation serology
Lyme	Borreliosis - IgM confirmation serology - serum
Lyme	Lyme disease - IgG/IgM screening serology - serum
Lyme	Lyme disease - IgG intrathecal index - CSF + serum
Lyme	Lyme disease - screening serology - CSF
Lyme disease	Borreliosis - direct diagnosis - PCR
Lyme disease	Borreliosis - IgG confirmation serology
Lyme disease	Borreliosis - IgM confirmation serology - serum
Lyme disease	Lyme disease - IgG/IgM screening serology - serum
Lyme disease	Lyme disease - IgG intrathecal index - CSF + serum
Lyme disease	Lyme disease - screening serology - CSF
Lyme disease - IgG intrathecal index - CSF + serum	
Lyme disease - IgG/IgM screening serology - serum	
Lyme disease - screening serology - CSF	
Lymphocyte immunophenotyping	Immunophenotyping: Lymphocytes B - CD19 - EDTA whole blood
Lymphocyte immunophenotyping	Immunophenotyping: Lymphocytes CD3-CD4-CD8 - EDTA whole blood
Lymphocyte immunophenotyping	Immunophenotyping: Lymphocytes CD56+ / CD16+ - Natural Killer Cells - EDTA whole
Lymphocyte immunophenotyping	Immunophenotyping: Lymphocytes NK/CD57 - EDTA whole blood
Lymphocyte Immunophenotyping	Immunophenotyping of malignant hemopaties
Lymphocyte phenotyping	Immunophenotyping of malignant hemopaties
Lymphocyte typing	Immunophenotyping: Lymphocytes B - CD19 - EDTA whole blood

Lymphocyte typing	Immunophenotyping: Lymphocytes CD3-CD4-CD8 - EDTA whole blood
Lymphocyte typing	Immunophenotyping: Lymphocytes CD56+ / CD16+ - Natural Killer Cells - EDTA whole
Lymphogranuloma venereum	Chlamydia trachomatis - IgA serology - serum
Lymphogranuloma venereum	Chlamydia trachomatis - IgG serology - serum
Lymphogranuloma venereum	Chlamydia trachomatis - IgM serology - serum
Lymphoreticulosis-Benign Inoculation	Bartonellosis - B. henselae - B. quintana - IgG serology - serum
Lymphoreticulosis-Benign Inoculation	Bartonellosis - direct diagnosis - PCR
Lyrica	Pregabalin - serum
Lysanxia®	Prazepam - Serum
Lysergic acid diethylamide	LSD - Lysergic acid diethylamide - Urine
Lysergic Acid Diethylamide	LSD - Lysergic acid diethylamide - Urine
Lysergic Acid Diethylamide - serum	LSD - Lysergic acid diethylamide
Lysergic Acid Diethylamide - urine	LSD - Lysergic acid diethylamide - biological fluid
Lyso- globotriaosylcéramide	Lyso-Gb3
Lyso-Gb3	
Lysodren	Mitotane - plasma
Lysozyme - serum	
LZTR1	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
LZTR1	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
Mabthera ®	Rituximab - serum
Macro-CK	CPK - Creatine phosphokinase - isoenzymes - serum
MacroCK	CPK - Creatine phosphokinase - isoenzymes - serum
Macroglobulin - Alpha 2	Alpha-2 Macroglobulin - serum
Macrolides	Helicobacter pylori and clarithromycin resistance gene - direct diagnosis - PCR
Macroprolactin	Prolactin - molecular forms - serum
MAG - anti-antibodies	Anti-MAG antibodies - serum
Magnesium - serum	
Magnesium - urines	
Magnesium concentration	Magnesium - serum
Magnesium overload	Magnesium - urines
MAIPA	Anti-platelet antibodies bound or free - screening and identification
Maladie Vectorielle	Tularemia - Francisella tularensis - serology total antibodies - serum
Malaria - serology	Sérologie Palustre (IgM/IgG)
MALFORMATIONS	Exome Congenital heart defects
MALORMATIVE SYNDROME	Exome Isolated or syndromic intellectual disability and syndromic neurodevelopmental
Manganese - Sang total	
Manganese - Serum	
Manganese - urine	
Mansonella perstans	Filariasis - serology - serum
MAP2K1	DNA NGS Panel for brain solid tumors
Markers in progressive systemic sclerosis(Scleroderma) - serum	
Matrix	IgE against multiple individual allergens - inhalant allergens - Serum
Mature B lymphocytes	Immunophenotyping: Lymphocytes B - CD19 - EDTA whole blood
MAX	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
MAX	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
MC1R	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
MC1R	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
mCRPC	DNA NGS Panel for prostate cancer
MD5	Myosite Markers - serum
MDA	Amphetamines - dosage - plasma
MDA	Amphetamines - dosage - urines
MDEA	Amphetamines - dosage - plasma
MDEA	Amphetamines - dosage - urines
MDH2	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
MDH2	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
MDMA	Amphetamines - dosage - plasma
MDMA	Amphetamines - dosage - urines
Measles	Measles - direct diagnosis - PCR

Measles - direct diagnosis - PCR	
Measles - IgG/IgM serology - serum	
Measles - immunity test - IgG serology - serum	
Measles virus	Measles - direct diagnosis - PCR
Measles virus - IgG immunity test	Measles - immunity test - IgG serology - serum
Measurement of anti-Xa activity - low molecular weight heparin	Anti-Xa activity : low molecular weight heparin - LMWH - plasma
Mediterranean spotted fever - serology	Rickettsioses - R. conorii, R. typhi - serology - serum
Melanoma	DNA NGS Panel solid tumor Melanoma
Melanoma	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
Melatonin - 24-hour urine collection	
Melatonin - Urine	
membrane fatty acids	Fatty acids of erythrocyte membrane - chromatography - Total EDTA blood - Fatty free
MEN1	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
MEN1	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
Méningocoque	Designated bacterium - direct diagnosis - PCR
Mercury	
Mercury - Urine	
MERTK	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
MERTK	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
Mesothelioma	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
Mestacine®	Minocycline - Serum
MET	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
MET	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
MET	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
MET	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
MET	RNA NGS Panel solid tumors, Pan-organ : 65 genes
metabolites of DEHP, DnBP, BBzP, DEP, DiBP, DINCH, DiNP, DiDP,	Phthalates - extended panel - urines - Phthalates - extended panel
Metadrenaline	Methoxylated derivatives of catecholamines - Free metanephprines
Metadrenaline	Methoxylated derivatives of catecholamines - Free metanephprines - child - Urine
Metadrenaline	Methoxylated derivatives of catecholamines - Free metanephprines - Urine
Metanephprines	Methoxylated derivatives of catecholamines - Free metanephprines
Metanephprines	Methoxylated derivatives of catecholamines - Free metanephprines - child - Urine
Metanephprines	Methoxylated derivatives of catecholamines - Free metanephprines - Urine
Metapneumovirus	Paediatric respiratory viruses (RSV / Metapneumovirus / Rhinovirus) - direct diagnosis -
Metformin	
Methadone and EDDP - Serum	
methamphetamin	Amphetamines - dosage - urines
Methamphétamine	Amphetamines - dosage - plasma
Methanol	Methyl alcohol - Serum
Methanol	Methyl alcohol - urine
Methicillin-resistant Staphylococcus aureus (MRSA)	Antibiotic multi-resistant bacteria (MRB) - screening
Methotrexate	
Methotrexate Bellon®	Methotrexate
Methoxyamines	Methoxylated derivatives of catecholamines - Free metanephprines
Methoxyamines	Methoxylated derivatives of catecholamines - Free metanephprines - child - Urine
Methoxyamines	Methoxylated derivatives of catecholamines - Free metanephprines - Urine
Methoxychlor	Pesticides - screening and/or dosage - urine
Methoxylated derivatives of catecholamines - Free metanephprines	
Methoxylated derivatives of catecholamines - Free metanephprines - child - Urine	
Methoxylated derivatives of catecholamines - Free metanephprines - Urine	
Methyl alcohol - Serum	
Methyl alcohol - urine	
Methyl-, ethyl-, propyl-, butyl-paraben	Paraben - urines - Paraben
Methylbenzene	Toluene - EDTA whole blood or Heparin
Methylbenzene	Toluene - urine
Methylenetetrahydrofolate reductase - MTHFR gene - c.677C>T mutation	
Methylmalonic acid - urines	
Methylphenidate Hydrochloride - Serum	

Metoject®	Methotrexate
Metoprolol - Serum	
MEVE	Tick-borne encephalitis - TBE - IgG/IgM serology - serum
Mg	Magnesium - serum
Mg	Magnesium - urines
MGA	RNA NGS Panel solid tumors, Pan-organ : 65 genes
MGT30 - anti-antibodies	Anti-titin antibodies - serum
Mi-2 - anti-antibodies	Myosite Markers - serum
Mianserin - Serum	
MIC	Aerobic / anaerobic bacteria - MIC: Minimum Inhibitory Concentration
Microalbumin - urine	
Microdeletions in the Yq11 region, azoospermia factor (AZF)	Y chromosome - (regions AZFs, AZFb, AZFc)
Microglobulin - Alpha 1	Alpha-1 Microglobulin - urine
Microglobulin - Beta 2	Beta-2 microglobulin - CSF
Microglobulin - Beta 2	Beta-2 microglobulin - serum
Microglobulin - Beta 2	Beta-2 microglobulin - urine
Micropakine	Valproic acid - serum
Microsatellite instability	DNA NGS Panel solid tumor Colon and MSI status determination
Microsatellite instability	DNA NGS Panel solid tumor Endometrium and MSI status determination
Microsatellites	DNA NGS Panel solid tumor Colon and MSI status determination
Microsatellites	DNA NGS Panel solid tumor Endometrium and MSI status determination
Microsporidies	Cryptosporidies - Microsporidies - direct diagnosis - PCR
Microsporidiose	Cryptosporidies - Microsporidies - direct diagnosis - PCR
Milieu liquide	Anaerobic and aerobic bacteria - susceptibility testing
Milieu solide	Anaerobic and aerobic bacteria - susceptibility testing
Milnacipran - Serum	
Mineralocorticoid	Aldosterone
Mineralocorticoid	Aldosterone - urine
Minimum inhibitory concentrations - aerobic and anaerobic bacteria	Aerobic / anaerobic bacteria - MIC: Minimum Inhibitory Concentration
Minkowski - Chauffard disease	hereditary spherocytosis - screening (eosin-5'-maleimide (EMA) staining)
Minocycline - Serum	
Mirtazapine - serum	
MIS - Müllerian-Inhibiting Substance	AMH - Anti-Müllerian hormone
MIS - Müllerian-Inhibiting Substance	AMH - Anti-Müllerian hormone - Child
Mismatch repair deficiency	DNA NGS Panel solid tumor Colon and MSI status determination
MITF	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
MITF	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
MITF	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
Mitochondria - anti-antibodies	Anti-mitochondrial antibodies - screening and titration - serum
Mitotane - plasma	
MKP	Monkeypox virus - direct diagnosis - PCR
MLH1	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
MLH1	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
MLH1	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
MLH1	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
MMF	Mycophenolate - plasma
MMR	DNA NGS Panel solid tumor Colon and MSI status determination
MMR	DNA NGS Panel solid tumor Endometrium and MSI status determination
MMR	Oncogenetic exome
Mn	Manganese - Sang total
Mn	Manganese - Serum
Mn	Manganese - urine
Moclamine®	Moclobemide - Serum
Moclobemide - Serum	
modigraf	Tacrolimus - FK-506
Molecular forms of prolactin	Prolactin - molecular forms - serum
Molecular profile	DNA NGS Panel solid tumor Endometrium
Molecular profile	DNA NGS Panel solid tumor Endometrium and MSI status determination

Molecular profile	Homologous recombination deficiency (HRD) and somatic mutations of the BRCA1 and
Monkeypox virus - direct diagnosis - PCR	
Mono-2-ethyl-5-hydroxyhexyl phthalate - metabolite of Di-éthylhexyl phtalate (DEHP)	Phthalates - extended panel - urines - Phthalates - extended panel
Mono-2-ethyl-5-hydroxyhexyl phthalate - metabolite of Di-éthylhexyl phtalate (DEHP)	Phthalates - reduced panel - urines - Phthalates - reduced panel
Mono-2-ethyl-5-oxohexyl phthalate - metabolite of Di-éthylhexyl phtalate (DEHP)	Phthalates - extended panel - urines - Phthalates - extended panel
Mono-2-ethyl-5-oxohexyl phthalate - metabolite of Di-éthylhexyl phtalate (DEHP)	Phthalates - reduced panel - urines - Phthalates - reduced panel
Mono-ethylhexyl phtalate - metabolite of Di-éthylhexyl phtalate (DEHP)	Phthalates - reduced panel - urines - Phthalates - reduced panel
Mono-éthylhexyl phtalate - metabolite of Di-éthylhexyl phtalate (DEHP)	Phthalates - extended panel - urines - Phthalates - extended panel
Mono-iso-butyl phthalate - metabolite of Di-iso-butyl phthalate (DiBP)	Phthalates - extended panel - urines - Phthalates - extended panel
Mono-n-butyl phtalate - metabolite of Di-n-butyl phthalate (DnBP)	Phthalates - extended panel - urines - Phthalates - extended panel
Mono-n-butyl phtalate - metabolite of Di-n-butyl phthalate (DnBP)	Phthalates - reduced panel - urines - Phthalates - reduced panel
Monobenzylbutyl phtalate- metabolite of bBenzylbutylphtalate (BzBP)	Phthalates - extended panel - urines - Phthalates - extended panel
Monobenzylbutyl phtalate- metabolite of bBenzylbutylphtalate (BzBP)	Phthalates - reduced panel - urines - Phthalates - reduced panel
Monocytes	Immunophenotyping of malignant hemopaties
Monoethyl phtalate - metabolite of Diethyl phtalate (DEP)	Phthalates - extended panel - urines - Phthalates - extended panel
Monoethyl phtalate - metabolite of Diethyl phtalate (DEP)	Phthalates - reduced panel - urines - Phthalates - reduced panel
Monohydroxybenzene	Phenol - urine
Monomeric prolactin prolactin little	Prolactin - molecular forms - serum
Monomethyl arsenic acid	Arsenic - inorganic an metabolites derivatives - urine
mononucleosis	Epstein-Barr Virus - EBV - IgA EA serology - serum
mononucleosis	Epstein-Barr Virus - EBV - IgG EA serology - serum
Morbillivirus - IgG immunity test	Measles - immunity test - IgG serology - serum
Morphine	Opiates - dosage - plasma
Morphine	Opiates - dosage - urines
Motival®	Amitriptyline - Serum
MPA	Mycophenolate - plasma
MPO - anti-antibodies	Anti-MPO antibodies - myeloperoxidase - Serum
MRD LLC	Immunophenotyping of malignant hemopaties
MS	Anti-Natalizumab antibodies - serum
MSH2	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
MSH2	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
MSH2	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
MSH2	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
MSH6	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
MSH6	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
MSH6	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
MSH6	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
MSI	DNA NGS Panel solid tumor Colon and MSI status determination
MSI status	Circulating tumor DNA NGS Panel - Peripheral blood from Cell Free DNA tube : ambient
MSI status	DNA NGS Panel solid tumor Colon and MSI status determination
MSI status	DNA NGS Panel solid tumor Endometrium and MSI status determination
MSS	DNA NGS Panel solid tumor Colon and MSI status determination
MT-RNR1	Pharmacogenetic Passport - PharmacoPass Expert
MTX	Methotrexate
MUC1	MUC1 nanopore Nephropathie
MUC1 nanopore Nephropathie	
Muconic acid	Trans-muconic acid - ttMA - urine
Mucopeptide N-acetylmuramylhydrolase	Lysozyme - serum
Mucopolysaccharides - urines	
Mucorales	Fungus - Yeasts - screening
Mucormycosis	Fungus - Yeasts - screening
Multiple sclerosis	Anti-Natalizumab antibodies - serum
Mumps - IgG serology - serum	
Mumps - IgG/IgM serology - serum	
Mumps paramyxovirus - IgG/IgM serology	Mumps - IgG/IgM serology - serum
Mumps paramyxovirus (IgG serology)	Mumps - IgG serology - serum
Mumps virus - IgG/IgM serology	Mumps - IgG/IgM serology - serum
Mumps virus - (serodiagnosis: IgG)	Mumps - IgG serology - serum

Muramidase	Lysozyme - serum
murine typhus	Rickettsioses - R. conorii, R. typhi - serology - serum
Mushroom	Yeasts - antifungal susceptibility testing
Mutation Delta F508	Cystic fibrosis Detection of variations within the CFTR gene after massive
Mutation of the methylenetetrahydrofolate reductase gene	Methylenetetrahydrofolate reductase - MTHFR gene - c.677C>T mutation
mutation resistance to TKI	tyrosine kinase ABL mutation
MUTYH	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
MUTYH	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
Mvasi®)	Bevacizumab - serum
Myagawanellosis	Chlamydia trachomatis - IgG serology - serum
Myasthenia	Anti-acetylcholine receptor antibodies - serum
Myasthenia	Anti-MuSK antibodies - serum
MYBPC3	Cardiogenetic panel or exome
Mycobacteria - culture	
Mycobacteria - direct examination	
Mycobactéria - identification	
Mycobacteria - PCR detection of mutations associated with resistance to isoniazid,	
Mycobacteria - PCR detection of rifampicin and isoniazid resistance in M.tuberculosis complex	
Mycobacteria - PCR M. tuberculosis complex (BKBM)	
Mycobacteria - susceptibility testing	
Mycobacteria - tuberculosis complex	Mycobacteria - PCR detection of rifampicin and isoniazid resistance in M.tuberculosis
Mycobacteria - tuberculosis complex	Mycobacteria - PCR M. tuberculosis complex (BKBM)
Mycobacterium tuberculosis	Mycobacteria - PCR detection of mutations associated with resistance to isoniazid,
Mycology	Fungus - Yeasts - MIC: Minimum Inhibitory Concentration
Mycology	Fungus - Yeasts - screening
Mycology	Yeasts - antifungal susceptibility testing
Mycophenolate - plasma	
Mycophenolate Mofetil	Mycophenolate - plasma
Mycophenolic acid	Mycophenolate - plasma
Mycoplasma	Mycoplasma pneumoniae - direct diagnosis - PCR
Mycoplasma	Mycoplasma pneumoniae - IgG serology - serum
Mycoplasma	Mycoplasma pneumoniae - IgM serology - serum
Mycoplasma genitalium	Chlamydia trachomatis, Neisseria gonorrhoeae, Mycoplasma genitalium and
Mycoplasma genitalium	IST PMA Expert & Multiplex PCR detection & Urogenital mycoplasma, C. trachomatis, N.
Mycoplasma genitalium	Mycoplasma genitalium Search for mutations associated with macrolide resistance
Mycoplasma genitalium - Search by gene amplification	
Mycoplasma genitalium - Search by gene amplification - 1 sample	
Mycoplasma genitalium - Search by gene amplification - 2 sample	
Mycoplasma genitalium - Search by gene amplification - 3 sample	
Mycoplasma genitalium Search for mutations associated with macrolide resistance	
Mycoplasma hominis	IST PMA Expert & Multiplex PCR detection & Urogenital mycoplasma, C. trachomatis, N.
Mycoplasma hominis	urogenital (ureaplasma urealyticum) mycoplasma (mycoplasma hominis) - serology -
Mycoplasma pneumoniae	Respiratory Infection Panel - direct diagnosis - PCR
Mycoplasma pneumoniae - direct diagnosis - PCR	
Mycoplasma pneumoniae - IgG serology - serum	
Mycoplasma pneumoniae - IgM serology - serum	
Mycoplasme	Mycoplasma genitalium - Search by gene amplification
Mycoplasme	Mycoplasma genitalium - Search by gene amplification - 1 sample
Mycoplasme	Mycoplasma genitalium - Search by gene amplification - 2 sample
Mycoplasme	Mycoplasma genitalium - Search by gene amplification - 3 sample
Mycosis	Fungus - Yeasts - identification
Mycosis	Fungus - Yeasts - screening
Myelin - anti-antibodies	Anti-myelin antibodies - serum
Myelogram - Perls staining	
Myeloperoxidase - anti-antibodies	Anti-MPO antibodies - myeloperoxidase - Serum
Myfortic	Mycophenolate - plasma
MYH7	Cardiogenetic panel or exome
Myosin - anti-antibodies	Anti-striated muscle antibodies - serum

Myosite Markers - serum		
Myositis dot		Myosite Markers - serum
Myositis markers		Myosite Markers - serum
Myxovirus influenzae - serology;		Influenza A and B - IgG serology - serum
N-acetyl neuraminic acid		Sialic acid free - urines
N-telopeptides		NTx - N-telopeptides of type I collagen - urines
N-Terminal propeptide of type III procollagen		Type III procollagen - serum
Na		Electrolytes stool - Stool
NAB2		RNA NGS Panel for soft tissue tumors : 20 genes
NAB2		RNA NGS Panel solid tumors, Pan-organ : 65 genes
Nanopore		Genome by long-read sequencing (Nanopore) - EDTA whole blood
Nantucket fever		Human babesiosis - serology - serum
Naropeine - Serum		
NASH TEST		FibroMAX - . - serum
NASH-FibroTest - serum		
NashTest 2®		NASH-FibroTest - serum
Native DNA - anti-antibodies		Anti-DNA antibodies Double stranded DNA - Serum
Native DNA - anti-antibodies		Anti-DNA antibodies double stranded - Farr test - serum
Native PTH		PTH - Intact parathormone - serum
Natural killer		Immunophenotyping: Lymphocytes CD56+ / CD16+ - Natural Killer Cells - EDTA whole
NBN		Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
NBN		Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
NC16A - anti-antibodies		Anti-epidermal basement membrane and intracellular substance antibodies Typing
NCOA4		RNA NGS Panel solid tumors, Pan-organ : 65 genes
Nebcine®		Tobramycin - serum
Necrotising myositis		Myosite Markers - serum
NEFA		Fatty acids free non-esterified - serum
Neisseria gonorrhoeae		Chlamydia trachomatis, Neisseria gonorrhoeae and Trichomonas vaginalis - multiplex
Neisseria gonorrhoeae		Chlamydia trachomatis, Neisseria gonorrhoeae, Mycoplasma genitalium and
Neisseria gonorrhoeae		IST PMA Expert ̂ Multiplex PCR detection ̂ Urogenital mycoplasma, C. trachomatis, N.
Neisseria gonorrhoeae - Detection by molecular amplification		
Neisseria meningitidis		Designated bacterium - direct diagnosis - PCR
Neoral®		Cyclosporin
Nephritic factor		C3NeF - Nephritic factor - serum
nephroblastome		Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
Nephrology exome		
Nephropathy		Nephrology exome
Neuleptil®		Periciazine - Serum
Neuro Check		Juvenalis - Panel - Neurocheck
Neuro-endocrine		Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
Neuro-Endocrine		Neuroendocrine Oncogenetics Panel Target genes : DLST , DNMT3A, EGLN1, EGLN2,
Neuroborreliosis		Lyme disease - IgG intrathecal index - CSF + serum
Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,		
Neuromyelitis optica - anti-antibodies		Anti-neuromyelitis optica antibodies - serum
Neuron-Specific Enolase		NSE - Neuron-Specific Enolase - biological fluid
Neuron-Specific Enolase		NSE - Neuron-Specific Enolase - serum
Neurone		Anti-ZC4 antibodies - serum
Neurones		Anti-GAD antibodies - CSF
neurons		Anti-titin antibodies - serum
neurons		Anti-ZC4 antibodies - CSF
Neurons		Anti-retina antibodies - serum
Neurons		Anti- SOX1 antibodies - CSF
Neurons		Anti- SOX1 antibodies - serum
Neurons		Anti-titin antibodies - CSF
Neurons		Anti-Tr antibodies - CSF
Neurons		Anti-Tr antibodies - serum
Neurontin®		Gabapentin - serum
new index for prostate cancer		[-2]pro-PSA - Phi score calculation - serum

New marker of ovarian cancer	HE4- ROMA calculation - serum
New prostatic cancer marker	[-2]pro-PSA - Phi score calculation - serum
Next Generation Sequencing	Detection and identification bacterial DNA - 16S PCR by NGS sequencing
NF1	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
NF1	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
NF1	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
NF2	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
NF2	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
NGS	Detection and identification bacterial DNA - 16S PCR by NGS sequencing
NGS	DNA NGS Panel for brain solid tumors
NGS	DNA NGS Panel for cholangiocarcinoma
NGS	DNA NGS Panel for pancreatic cancer
NGS	DNA NGS Panel for prostate cancer
NGS	DNA NGS Panel for thyroid cancer
NGS	DNA NGS Panel solid tumor Breast
NGS	DNA NGS Panel solid tumor Colon
NGS	DNA NGS Panel solid tumor Colon and MSI status determination
NGS	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
NGS	DNA NGS Panel solid tumor Endometrium
NGS	DNA NGS Panel solid tumor Endometrium and MSI status determination
NGS	DNA NGS Panel solid tumor GIST
NGS	DNA NGS Panel solid tumor Lung
NGS	DNA NGS Panel solid tumor Lung extended
NGS	DNA NGS Panel solid tumor Melanoma
NGS	DNA NGS Panel solid tumor Pan-organ : 64 genes
NGS	Homologous recombination deficiency (HRD) and somatic mutations of the BRCA1 and
NGS	RNA NGS panel for brain solid tumors
NGS	RNA NGS Panel for cholangiocarcinoma
NGS	RNA NGS Panel for lung cancers
NGS	RNA NGS Panel for pancreatic cancer
NGS	RNA NGS Panel for soft tissue tumors : 20 genes
NGS	RNA NGS Panel for thyroid cancer
NGS	RNA NGS Panel solid tumors, Pan-organ : 65 genes
NGS panel	RNA NGS Panel for lung cancers
NH3	Ammoniaemia - plasma
Ni	Nickel - Serum
Ni	Nickel - Urine
Nickel - Serum	
Nickel - Urine	
Nicolas-Favre - disease	Chlamydia trachomatis - IgA serology - serum
Nicolas-Favre - disease	Chlamydia trachomatis - IgG serology - serum
Nicolas-Favre - disease	Chlamydia trachomatis - IgM serology - serum
Nicotine	Cotinine - urine
Nizoral®	Ketoconazole - Serum
NK	Immunophenotyping: Lymphocytes CD56+ / CD16+ - Natural Killer Cells - EDTA whole
NK cells	Immunophenotyping: Lymphocytes CD56+ / CD16+ - Natural Killer Cells - EDTA whole
NMO	Anti-neuromyelitis optica antibodies - CSF
NMO - anti-antibodies	Anti-neuromyelitis optica antibodies - serum
Noctran®	Clorazepate - Serum
non treponemal test CSF	Syphilis - titration - CSF
Non tuberculosis mycobacteria - PCR detection of macrolide and aminoglycoside resistance	
Non-Rhesus/Kell erythrocyte antigens	Extended phenotype
Non-Rhesus/Kell phenotype	Extended phenotype
Non-SHBG-bound testosterone	Testosterone bioavailability - serum
Noradrenaline	Catecholamines - free - Urine
Noradrenaline	Free catécholamines
Noradrenaline	Free catécholamines - children - Urine
Norclobazam	Clobazam - Serum

Norclomipramine	Clomipramine - Serum
Norclozapine	Clozapine
Nordiazepam	Diazepam + desmethyldiazepam - Serum
Norepinephrine	Catecholamines - free - Urine
Norepinephrine	Free catécholamines
Norepinephrine	Free catécholamines - children - Urine
Norfluoxetine	Fluoxetine - serum
Normetadrenaline	Methoxylated derivatives of catecholamines - Free metanephrines
Normetadrenaline	Methoxylated derivatives of catecholamines - Free metanephrines - child - Urine
Normetadrenaline	Methoxylated derivatives of catecholamines - Free metanephrines - Urine
Normetanephrines	Methoxylated derivatives of catecholamines - Free metanephrines
Normetanephrines	Methoxylated derivatives of catecholamines - Free metanephrines - child - Urine
Normetanephrines	Methoxylated derivatives of catecholamines - Free metanephrines - Urine
Norovirus - direct diagnosis - PCR - Stools	
Norovirus - screening - Stools	
Norovirus GI/GII	Gastrointestinal Panel - direct diagnosis - PCR - Stools
Norset®	Mirtazapine - serum
Nortriptyline	Amitriptyline - Serum
Norvir®	Ritonavir - Serum
Norwalk virus	Norovirus - direct diagnosis - PCR - Stools
Norwalk virus	Norovirus - screening - Stools
Novatrex®	Methotrexate
Novazam®	Diazepam + desmethyldiazepam - Serum
NPM1	RNA NGS Panel solid tumors, Pan-organ : 65 genes
NRAS	DNA NGS Panel for brain solid tumors
NRAS	DNA NGS Panel for thyroid cancer
NRAS	DNA NGS Panel solid tumor Colon
NRAS - KIT - BRAF	DNA NGS Panel solid tumor Melanoma
NRG1	RNA NGS Panel for pancreatic cancer
NRG1	RNA NGS Panel solid tumors, Pan-organ : 65 genes
NSCLC	DNA NGS Panel solid tumor Lung extended
NSD3	RNA NGS Panel solid tumors, Pan-organ : 65 genes
NSE - Neuron-Specific Enolase - biological fluid	
NSE - Neuron-Specific Enolase - serum	
NT-proBNP - N-Terminal Pro-Brain Natriuretic Peptide - serum	
NTM	Mycobactéria - identification
NTRK	RNA NGS panel for brain solid tumors
NTRK	RNA NGS Panel for cholangiocarcinoma
NTRK	RNA NGS Panel for lung cancers
NTRK	RNA NGS Panel for pancreatic cancer
NTRK	RNA NGS Panel for soft tissue tumors : 20 genes
NTRK1	RNA NGS Panel solid tumors, Pan-organ : 65 genes
NTRK1/2/3	RNA NGS Panel for lung cancers
NTRK1/2/3	RNA NGS Panel for pancreatic cancer
NTRK1/2/3	RNA NGS Panel for thyroid cancer
NTRK2	RNA NGS Panel solid tumors, Pan-organ : 65 genes
NTRK3	RNA NGS Panel solid tumors, Pan-organ : 65 genes
NTx	NTx - N-telopeptides of type I collagen - urines
NTx - N-telopeptides of type I collagen - urines	
Nucleosome - anti-antibodies	Anti-nucleosome antibodies - serum
Nuctalon®	Estazolam - Serum
NUDT15	Pharmacogenetic Passport - PharmacPass Expert
NUTM1	RNA NGS Panel for soft tissue tumors : 20 genes
Nutritol	Nutritol - IgG food panel 287 foods - serum
Nutritol - IgG food panel - 100 foods - serum	
Nutritol - IgG food panel - 25 foods - serum	
Nutritol - IgG food panel - 50 foods - serum	
Nutritol - IgG food panel - 50 foods VEGAN - serum	
Eurofins Biomnis – International Division – Test guide Specialised medical pathology – Valid from : 01/08/2026	

Nutritol - IgG food panel 287 foods - serum	
NXP2	Myosite Markers - serum
o,p DDD	Mitotane - plasma
O157	Escherichia coli - serotyping by agglutination (e.g. E. coli K1; EPEC; O157, etc.)
o-Cresol - urine	
Oalate content	Oxalic acid - serum
OAU	Organic acids - quantitative - urine
OB protein	Leptin - serum
(OCB) Oligoclonal Bands	Isoelectric focusing - CSF + serum
Oestradiol	Oestradiol - man/child - serum
Oestradiol	Oestradiol - woman - serum
Oestradiol	
Oestradiol - man/child - serum	
Oestradiol - woman - serum	
Oestradiol man/child - 2 time	
Oestradiol man/child - 3 time	
Oestradiol man/child - 4 time	
Oestrone	Oestrone - serum
Oestrone - serum	
OH-tryptamine (5-) - whole blood	Serotonin - whole blood
OJ - anti-antibodies	Myosite Markers - serum
Olanzapine	
Oligosaccharides - screening - urine	
Oligosaccharides - screening test	Oligosaccharides - screening - urine
Omega 3	Fatty acids of erythrocyte menbrane - chromatography - Total EDTA blood - Fatty free
Omega 6	Fatty acids of erythrocyte menbrane - chromatography - Total EDTA blood - Fatty free
OMIM code : #105830 / #176270	Angelman / Prader-Willi Syndrome - uniparental disomy - mechanism identification -
OMIM code : #105830 / #176270	Angelman / Prader-Willi Syndrome - SNRPN methylation
OMIM code : #105830/ #176270	Angelman / Prader-Willi Syndrome - SNRPN methylation
OMIM code : #105830/ #176270	Angelman / Prader-Willi Syndrome - uniparental disomy - mechanism identification -
OMIM code : *142800/+142830	Class I HLA - HLA-A and HLA-B genotyping - EDTA whole blood
OMIM code : *142857 / *604305	Class II HLA - HLA-DQ and HLA-DR genotyping - EDTA whole blood
OMIM code : *176930	Factor II - Prothrombin - G20210G>A mutation
OMIM code : *191740	UGT1A1 : screening for UGT1A1*28(rs8175347) polymorphism - Involved in irinotecan
OMIM code : *191740/ #143500	Gilbert Syndrome - Screening for UGT1A1*28 (rs8175347) polymorphism
OMIM code : #300624 / *309550	Fragile X - syndrome
OMIM code : #300624 / *309550	Fragile X - syndrome
OMIM code : *602421 / #219700	Cystic Fibrosis Detection of variations within the CFTR gene after massive
OMIM code : *602421/#219700	Cystic fibrosis Detection of variations within the CFTR gene after massive
OMIM code : *603202/ #223000	lactose primary intolerance - Genetic characterization
OMIM code : *607093 / #236250	Methylenetetrahydrofolate reductase - MTHFR gene - c.677C>T mutation
OMIM code : *608107 / #249100	Familial Mediterranean fever - MEFV gene - new generation sequencing
OMIM code : *612309	Factor V Leiden - p.Arg506Gln mutation
OMIM code : #613490 / *107400	Alpha-1 Antitrypsine - Testing for S and Z variants of the SERPINA1 gene
OMIM code : *613609 / #235200	Haemochromatosis - HFE gene - p.Cys282Tyr mutation
OMIM code : *613609 / #235200	Haemochromatosis - HFE gene - p.Ser65Cys mutation
Onchocerciasis	Filariasis - serology - serum
Oncogenetic	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
Oncogénétic	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
Oncogenetic exome	
Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM, MLH1,	
Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A, CHEK2,	
oncogenetics	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
Oncogenetics	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
Oncogenetics	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
Oncogenetics	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
Oncogenetics	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
Oncogenetics	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,

oncogenic HPV	Detection of the HPV16 genome in the context of anal cancer screening
Oncogenic HPV	Detection of high-risk HPV nucleic acids using a molecular technique as part of primary
Oncology	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
Oncology	DNA NGS Panel solid tumor Lung extended
Oncology	DNA NGS Panel solid tumor Pan-organ : 64 genes
Onconeuronal antibodies	Anti-neuron antibodies - CSF
Onconeuronal antibodies	Anti-neuron antibodies - serum
Onychomycosis	Fungus - Yeasts - screening
Opiates - dosage - plasma	
Opiates - dosage - urines	
Oral anticoagulants : Fluindione - Serum	
Orbenin®	Cloxacillin - Serum
Organic acids - quantitative - urine	
Orgaran® - anti-Xa	Anti-Xa activity : Danaparoid - Orgaran - plasma
Ornithosis	Chlamydia psittaci - IgA serology - serum
Ornithosis	Chlamydia psittaci - IgG serology (1st intention) - serum
oro-fecal transmission	Hepatitis E - IgM antibody - serum
Oro-fecal transmission	Hepatitis E - IgG antibody - serum
Orosomuroid - serum	
Orosomuroid (Alpha-1-acid glycoprotein)	Orosomuroid - serum
ortho-Cresol	o-Cresol - urine
Orthopoxvirus	Monkeypox virus - direct diagnosis - PCR
Osmolality	Osmolarity - serum
Osmolality	Osmolarity - urines
Osmolarity - serum	
Osmolarity - urines	
Osmolarity stool	Electrolytes stool - Stool
Ostase	Alkaline phosphatases bone - serum
Osteocalcin - Serum	
Osteomark	NTx - N-telopeptides of type I collagen - urines
Ovaire	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
Ovarian cancer	Homologous recombination deficiency (HRD) and somatic mutations of the BRCA1 and
Ovarian serous carcinoma	Homologous recombination deficiency (HRD) and somatic mutations of the BRCA1 and
Ovaries - anti-antibodies	Anti-ovarian antibodies - serum
Ovary	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
OVARY	Oncogenetic exome
Oxacillin - Serum	
Oxalate	Oxalic acid - serum
Oxalate	Oxalic acid - Urine
Oxalic acid - serum	
Oxalic acid - Urine	
Oxaluria	Oxalic acid - Urine
Oxazepam - Serum	
Oxcarbazepine	10-Hydroxy-carbazepine - serum
OxiCheck	Juvenalis - OxyCheck
Oxidant stress	Juvenalis - OxyCheck
Oxidative stress	Juvenalis - OxyCheck
Oxidised LDL	
Oxidized cholesterol	Oxidised LDL
OxyCheck	Juvenalis - OxyCheck
P	Progesterone - serum
P1NP - Intact N-Terminal Propeptide of Type I Procollagen - serum	
P24	HIV-1: Qualitative RNA detection, suspected primary infection - serum
P24 - HIV antigen	HIV - Human Immunodeficiency Virus - p24 antigen - serum
P25 antigen	HIV - Human Immunodeficiency Virus - p24 antigen - serum
P2PSA	[-2]pro-PSA - Phi score calculation - serum
Paediatric respiratory viruses (RSV / Metapneumovirus / Rhinovirus) - direct diagnosis - PCR	
PAI1 - Plasminogen activator inhibitor-1 - plasma	

p.Ala222Val mutation	Methylenetetrahydrofolate reductase - MTHFR gene - c.677C>T mutation
PALB2	DNA NGS Panel for pancreatic cancer
PALB2	DNA NGS Panel for prostate cancer
PALB2	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
PALB2	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
PALB2	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
Paliperidone	Risperidone
PAM50	Breast Cancer Prognostic Gene Signature Assay - PROSIGNA TEST
Pan-organ	DNA NGS Panel solid tumor Pan-organ : 64 genes
panbacterien	Detection and identification bacterial DNA - 16S PCR by NGS sequencing
Pancreas	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
Pancreas	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
Pancreas NGS panel	DNA NGS Panel for pancreatic cancer
Pancreas NGS panel	RNA NGS Panel for pancreatic cancer
Pancreas RNA-seq	RNA NGS Panel for pancreatic cancer
Pancreatic cancer	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4, CDKN2A, Pancreatic polypeptide - Serum	
Pancreatitis	Anti-lactoferrin antibodies and anti carbonic anhydrase antibodies - serum
Panel	Exome Panel for chronic and hereditary pancreatitis
Panel	Cardiogenetic panel or exome
Panel	exome
Panel	Exome Congenital heart defects
Panel	Exome Isolated or syndromic intellectual disability and syndromic neurodevelopmental
Panel	Exome with interpretation interface
Panel	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
Panel	Nephrology exome
Panel	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
Panel	Oncogenetic exome
Panel	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
Panel	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
Panel	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
Panel	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
Panel	Prenatal exome
Panel	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
Panel	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1, BRCA2,	
Panton Valentine toxin	PVL toxin detection (Staphylococcus aureus) - PCR on strain
Papillomavirus	Papillomavirus - HPV Détection and Typing of 19 high-risk HPV and 18 low-risk HPV -
Papillomavirus - HPV Détection and Typing of 19 high-risk HPV and 18 low-risk HPV - PCR	
Paraben	Paraben - urines - Paraben
Paraben - urines - Paraben	
Parabens	Paraben - urines - Paraben
Paraganglioma	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
Parainfluenza virus 1, 2, 3 - serology	Parainfluenzae 1, 2, 3 - IgG serology - serum
Parainfluenza virus types 1, 2, 3, 4	Respiratory Infection Panel - direct diagnosis - PCR
Parainfluenzae 1, 2, 3 - IgG serology - serum	
Paramyxovirus	Parainfluenzae 1, 2, 3 - IgG serology - serum
Paranodes antibodies (anti- NF155, NF186, CNTN1, CASPR1) - serum	
Paraprotein	Immunobinding - serum
Parasitological examination of stools	Parasitology - microscopy
Parasitology - microscopy	
Parastrongylus cantonensis	Angiostrongylus cantonensis - serology - serum
Parathormone	PTH - Intact parathormone - aspiration biopsy
Parathormone	PTH - Intact parathormone - serum
Parathormone-related protein	PTH-RP - Parathormone related protein - plasma
Parathyroid - anti-antibodies	Anti-Calcium Sensing Receptor antibodies - serum
Parathyroid hormone	PTH - Intact parathormone - aspiration biopsy

Parathyroid hormone	PTH - Intact parathormone - serum
Paratyphoid A/B/C - serology	Salmonellosis - serology - serum
Paroxetine - serum	
PARP inhibitor	DNA NGS Panel solid tumor Endometrium
PARP inhibitor	DNA NGS Panel solid tumor Endometrium and MSI status determination
PARP inhibitor	Homologous recombination deficiency (HRD) and somatic mutations of the BRCA1 and
PARP inhibitors	DNA NGS Panel solid tumor Endometrium
PARP inhibitors	DNA NGS Panel solid tumor Endometrium and MSI status determination
PARP inhibitors	Homologous recombination deficiency (HRD) and somatic mutations of the BRCA1 and
PARPi	DNA NGS Panel for pancreatic cancer
PARPi	DNA NGS Panel for prostate cancer
PARPi	DNA NGS Panel solid tumor Endometrium
PARPi	DNA NGS Panel solid tumor Endometrium and MSI status determination
PARPi	Homologous recombination deficiency (HRD) and somatic mutations of the BRCA1 and
Parvovirus B19 - direct diagnosis - PCR	
Parvovirus B19 - IgG/IgM serology - serum	
Parvovirus B19 - immunity test - serum	
PAT	Acid phosphatases total - serum
Paucialbumin	Microalbumin - urine
PAX3	RNA NGS Panel solid tumors, Pan-organ : 65 genes
PAX3/7	RNA NGS Panel for soft tissue tumors : 20 genes
PAX7	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Paxil®	Paroxetine - serum
Pb	Lead
Pb	Lead - whole blood
PBG	Porphobilinogen - urine
PBI	Iodine total - serum
PBKN	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
PBRM1	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
PBRM1	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
PC activity	Protein C - activity - plasma
PC Ag	Protein C - antigen - plasma
PCP - screening	Phencyclidine - screening - urine
PCR - proviral DNA - HTLV-I/II virus	HTLV I/II Qualitative PCR - EDTA whole blood
PCT	Procalcitonin
PDGFR alpha - exons 12, 14, 18	
Peflaccine®	Pefloxacin - Serum
Pefloxacin - Serum	
PEG prolactin	Prolactin - molecular forms - serum
pemphigoid	Anti-epidermal basement membrane and intracellular substance antibodies Typing
Pemphigoïde gravidique HGF (herpes Gestationis Factor)	Anti-epidermal basement membrane and intracellular substance antibodies Typing
pemphigus	Anti-epidermal basement membrane and intracellular substance antibodies Typing
Pemphigus	Anti-epidermal basement membrane and intracellular substance antibodies - serum
Peptide of type III procollagen	Type III procollagen - serum
Peptides from bone collagen degradation	CrossLaps - Serum
Peptides from bone collagen degradation	CrossLaps - urines
Peptides from NTx collagen degradation	NTx - N-telopeptides of type I collagen - urines
Per- and polyfluoroalkyl compounds - 7 compounds panel - Serum	
Per- and polyfluoroalkyl compounds - extended panel - Serum	
Perampanel - Serum - PERAMPANEL	
Perchloroethylene	Tetrachloroethylene - EDTA whole blood or Heparin
Perchloroethylene	Tetrachloroethylene - urine
Perfluoroalkyles et polyfluoroalkyles compound	Per- and polyfluoroalkyl compounds - 7 compounds panel - Serum
PerFluoroAlkylSubstances	Per- and polyfluoroalkyl compounds - 7 compounds panel - Serum
perfluorodecanoic acid (PFDA)	Per- and polyfluoroalkyl compounds - 7 compounds panel - Serum
perfluorohexanesulfonic acid (PFHxS)	Per- and polyfluoroalkyl compounds - 7 compounds panel - Serum
perfluorononanoic acid (PFNA)	Per- and polyfluoroalkyl compounds - 7 compounds panel - Serum
perfluoroundecanoic acid (PFUnDA)	Per- and polyfluoroalkyl compounds - 7 compounds panel - Serum

Periciazine - Serum	
Periodic disease - FMF	Familial Mediterranean fever - MEFV gene - new generation sequencing
Persantine®	Dipyridamole - Serum
Pertussis - serology	
Pertussis Disease	Pertussis - serology
Pertussis toxin	Pertussis - serology
Pesticides - screening and/or dosage - urine	
PEth	Phosphatidylethanol - Buvard ou sang total
PEth 16:0/18:1	Phosphatidylethanol - Buvard ou sang total
PF4 - heparin - anti-antibodies	Anti-PF4 antibodies - heparin - plasma
PFAS	Per- and polyfluoroalkyl compounds - 7 compounds panel - Serum
PFAS	Per- and polyfluoroalkyl compounds - extended panel - Serum
PFAS	trifluoroacetic acid TFA
p.Glu264Val mutation and p.Glu342Lys mutation	Alpha-1 Antitrypsine - Testing for S and Z variants of the SERPINA1 gene
Phadiatop	Specific IgE : inhalant allergen testing - Phadiatop®
Pharmacogenetic Passport - PharmacoPass Expert	
Pharmacogénétique	Pharmacogenetic Passport - PharmacoPass Expert
Phencyclidine - screening - urine	
Phenobarbital - serum	
Phenobarbitone levels in blood	Phenobarbital - serum
Phenol - urine	
Phenothiazine®	Cyamemazine - Serum
Phenotype (C, c, E, e, Kell)	Rhesus Kell
Phenotyping	Immunophenotyping: Lymphocytes B - CD19 - EDTA whole blood
Phenotyping	Immunophenotyping: Lymphocytes CD3-CD4-CD8 - EDTA whole blood
Phenotyping	Immunophenotyping: Lymphocytes CD56+ / CD16+ - Natural Killer Cells - EDTA whole
Phenotyping	Immunophenotyping: Lymphocytes NK/CD57 - EDTA whole blood
Phenotyping of dihydropyrimidine dehydrogenase	Uracile - dosage(search a risk of toxicity to fluoropyrimidines)
Phenylalanine - urine	
phenylketonuria	Phenylalanine - urine
Phenytoin - serum	
Phéochromocytoma	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
Phi	[-2]pro-PSA - Phi score calculation - serum
Phi index	[-2]pro-PSA - Phi score calculation - serum
Phosphate	Phosphorus - serum
Phosphatidylethanol - Buvard ou sang total	
Phosphatidylethanolamine - anti-IgG antibodies	Anti-phosphatidylethanolamine antibodies - IgG - serum
Phosphatidylethanolamine - anti-IgM antibodies	Anti-phosphatidylethanolamine antibodies - IgM - serum
Phosphaturia	Phosphorus - urine
phospholipase A2 receptor type M	anti-PLA2R antibodies
Phosphorus - serum	
Phosphorus - tubular reabsorption rate	
Phosphorus - urine	
Phosphorylated tau protein	TAU - PTAU - Amyloid proteins - LCR
Phtalates	Phtalates - extended panel - urines - Phtalates - extended panel
Phtalates	Phtalates - reduced panel - urines - Phtalates - reduced panel
Phtalates - extended panel - urines - Phtalates - extended panel	
Phtalates - reduced panel - urines - Phtalates - reduced panel	
Phylloquinone	Vitamin K1 - Phylloquinone
Phytomenadione	Vitamin K1 - Phylloquinone
Pi - Protease inhibitor	Alpha-1 Antitrypsin - dosage - Serum
Pi - Protease inhibitor - genotyping	Alpha-1 Antitrypsine - Testing for S and Z variants of the SERPINA1 gene
PIGF	Pre-eclampsia - risk estimation - serum
PIII-P	Type III procollagen - serum
PIK3CA	DNA NGS Panel for brain solid tumors
PIK3CA	DNA NGS Panel for cholangiocarcinoma
PIK3CA	DNA NGS Panel solid tumor Breast
Piperacillin - Serum	

Piroplasmosis	Human babesiosis - serology - serum
PITCH1	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
PITCH2	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
Pituitary hormones - alpha subunits	Alpha subunit - hCG, FSH, LH, TSH - serum
PKD1	Nephrology exome
PKD2	Nephrology exome
PL12 - anti-antibodies	Myosite Markers - serum
PL7 - anti-antibodies	Myosite Markers - serum
PLA	Lipoprotein Lp-PLA2 - serum
placental alkaline phosphatase	Alkaline phosphatases - isoenzymes - serum
Plaquenil®	Hydroxychloroquine - EDTA whole blood
Plasminogen - plasma	
Plasminogen activity	Plasminogen - plasma
Plasmocyte immunophenotyping	Immunophenotyping of malignant hemopaties
Plasmodium falciparum - serology	Sérologie Palustre (IgM/IgG)
Platelet Coombs	Anti-platelet antibodies bound or free - screening and identification
Platelet typing - EDTA whole blood	
Platelet VASP	VASP - VAsodilatator Stimulated Phosphoprotein - whole blood
plathelminthes et nemaltherminthes	Parasitology - microscopy
Plavix® - resistance test	VASP - VAsodilatator Stimulated Phosphoprotein - whole blood
Plesiomonas shigelloides	Gastrointestinal Panel - direct diagnosis - PCR - Stools
Pleura	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
PM100 - anti-antibodies	Myosite Markers - serum
PM1 - anti - antibodies	Myosite Markers - serum
PM-Scl - anti-antibodies	Myosite Markers - serum
PM75 - anti-antibodies	Myosite Markers - serum
PMS2	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
PMS2	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
PMS2	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
PMS2	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
Pneumococcus	Soluble bacterial antigens Streptococcus pneumoniae - CSF
Pneumococcus	Soluble bacterial antigens Streptococcus pneumoniae - urines
Pneumococcus - serology	Streptococcus pneumoniae - capsular polysaccharide - serology - serum
Pneumocoque	Designated bacterium - direct diagnosis - PCR
Pneumocystis	Beta-D-Glucan antigen - screening
Pneumocystis	Pneumocystis jirovecii - Quantitative PCR / fungal load
Pneumocystis carinii	Pneumocystis jirovecii - Quantitative PCR / fungal load
Pneumocystis jirovecii - Quantitative PCR / fungal load	
Pneumocystosis	Beta-D-Glucan antigen - screening
Pneumogéine®	Theophylline - serum
Pneumotropic viruses - adenovirus	Adenovirus - IgG serology - serum
Pneumotropic viruses - adenovirus	Adenovirus - screening - Stools
Pneumotropic viruses - influenza - serology	Influenza A and B - IgG serology - serum
POLD1	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
POLD1	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
POLE	DNA NGS Panel solid tumor Colon
POLE	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
POLE	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
Poliomyelitis	Poliovirus - serology - serum
Poliovirus - serology - serum	
Poliovirus type 1	Poliovirus - serology - serum
polycystic kidney disease	Nephrology exome
Polymyositis	Myosite Markers - serum
Polyomavirus BK - viral load	
Polyomavirus JC	Polyomavirus JC - direct diagnosis - PCR
Polyomavirus JC - direct diagnosis - PCR	
Ponderal Assay of Immune Globulin	IgA/G/M - Immunoglobulins A/G/M - serum
Pontiac fever	Legionellosis- IgG screening serology - serum

POR	Pharmacogenetic Passport - PharmacoPass Expert
Porphobilinogen - urine	
Porphyrin precursors	delta-Aminolevulinic acid - urine
Porphyrin precursors	Porphobilinogen - urine
Porphyryns - stool	
Porphyryns - Urines	
POT1	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
POT1	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
potassium	Electrolytes stool - Stool
Potassium channels - anti-antibodies	Anti-potassium channel antibodies / Lgi1 Caspr2 - serum
PP	Pancreatic polypeptide - Serum
PPARGC1A	RNA NGS Panel solid tumors, Pan-organ : 65 genes
PPP1CB	RNA NGS Panel solid tumors, Pan-organ : 65 genes
PR3 - anti-antibodies	Anti-PR3 antibodies - proteinase 3 - Serum
Pradaxa	Dabigatran - Measurement of the Dabigatran anti lia activity - plasma
Prasugrel	VASP - VAsodilator Stimulated Phosphoprotein - whole blood
Prazepam - Serum	
Pre-eclampsia	Pre-eclampsia - risk estimation - serum
Pre-eclampsia	Pre-eclampsia sFlt-1 et PlGF - Predictive test - sFlt-1/PlGF ratio - serum
Pre-eclampsia sFlt-1 et PlGF - Predictive test - sFlt-1/PlGF ratio - serum	
Pre-eclampsia - risk estimation - serum	
Pre-eclampsia risk calculation	Pre-eclampsia - risk estimation - serum
Prealbumin - serum	
precipitin (nature to be specified)	Serodiagnosis of cheesemonger lung Alveolitis Allergic Extrinsic) - screening - serum
precipitin (nature to be specified)	Serodiagnosis of Mushroom worker's lung (Alveolitis Allergic Extrinsic) - screening -
precipitins (nature to be specified)	Serodiagnosis of baker lung (Alveolitis Allergic Extrinsic) - screening - serum
precipitins (nature to be specified)	Serodiagnosis of Domestic lung (Alveolitis Allergic Extrinsic) - screening - serum
Predisposition	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
Predisposition	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
Predisposition	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
Predisposition	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
Predisposition	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
Predisposition	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
Predisposition	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
PREDISPOSITION	Oncogenetic exome
Prédisposition	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
Preeclampsia	Pre-eclampsia sFlt-1 et PlGF - Predictive test - sFlt-1/PlGF ratio - serum
Pregabalin - serum	
Pregnenolone	
Pregnenolone - 17-Hydroxy	17-Hydroxy-Pregnenolone - serum
Pregnenolone - 17-OH-	17-Hydroxy-Pregnenolone - serum
Pregnenolone - serum	
Pregnenolone sulfate - serum	
Prekallikrein - activity - plasma	
PRENATAL	Prenatal exome
Prenatal exome	
Presumed diagnosis of O27 types of Clostridium difficile	Clostridium difficile - toxins-screening by PCR
Previscan®	Oral anticoagulants : Fluindione - Serum
Prezista®	Darunavir - Serum
PRIST	Total IgE - serum
PRKACA	RNA NGS Panel solid tumors, Pan-organ : 65 genes
PRKAR1A	RNA NGS Panel solid tumors, Pan-organ : 65 genes
PRKN	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
PRL	Prolactin - serum
Pro-Brain Natriuretic Peptide	NT-proBNP - N-Terminal Pro-Brain Natriuretic Peptide - serum
Pro-gastrin-releasing-peptide (ProGRP)	
Proaccelerin	Factor V - Proaccelerin - plasma
Procalcitonin	

Proconvertin	Factor VII - Proconvertin - plasma
Prodilantin	Phenytoin - serum
Progesterone	
Progesterone, 17-OH-	17-Hydroxyprogesterone
Progesterone - serum	
Progesterone hormone	Progesterone - serum
Prognosis	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
Prograf®	Tacrolimus - FK-506
Progression	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
Proinsulin	
Proinsulin 10 time	
Proinsulin 11 time	
Proinsulin 12 time	
Proinsulin 13 time	
Proinsulin - serum	
Proinsulin 7 time	
Proinsulin 8 time	
Proinsulin 9 time	
proinsuline	
Prolactin	
Prolactin - molecular forms - serum	
Prolactin - serum	
Prolactin 7 time	
Prolactin 8 time	
Prolactin 9 time	
Prolans A	FSH - Follicle-stimulating hormone - serum
Prolans A	FSH - Follicle-stimulating hormone - urines
Prolans A	
PROLANS B	Total HCG (HCG dimer + beta fraction) - serum
Propafenone - Serum	
propériciazine	Periciazine - Serum
Propofan®	Dextropropoxyphene - urine
Propoxyphène	Dextropropoxyphene - urine
Propranolol - Serum	
proPSA	[-2]pro-PSA - Phi score calculation - serum
Prostate	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
Prostate	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
Prostate cancer	DNA NGS Panel for prostate cancer
Prostate cancer	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Prostate Health Index	[-2]pro-PSA - Phi score calculation - serum
Prostate NGS panel	DNA NGS Panel for prostate cancer
Protease inhibitor	Alpha-1 Antitrypsin - dosage - Serum
Protein 14.3.3	
Protein 4	HE4- ROMA calculation - serum
protein C	Protein C - activity - plasma
Protein C - activity - plasma	
Protein C - antigen - plasma	
Protein C activity	Protein C - activity - plasma
Protein C anticoagulant activity	Protein C - activity - plasma
Protein C antigen	Protein C - antigen - plasma
Protein electrophoresis	
Protein electrophoresis - serum	
Protein electrophoresis and Immunobinding - serum	
protein S	Protein S - activity - plasma
Protein S100	Protein S100 - serum
Protein S100 - serum	
Protein S100 beta	Protein S100 - serum
Protein S - activity - plasma	

Protein S activity	Protein S - activity - plasma
Protein S anticoagulant activity	Protein S - activity - plasma
Protein-bound iodine	Iodine total - serum
Proteinase 3 - anti-antibodies	Anti-PR3 antibodies - proteinase 3 - Serum
Proteinemia	Proteins - dosage - serum
Proteinogram	Protein electrophoresis
Proteinogram	Protein electrophoresis and Immunobinding - serum
Proteinogram	Protein electrophoresis - serum
Proteins - biological fluid	
Proteins - dosage - serum	
Proteins - dosage - urine	
Proteins - electrophoresis	Protein electrophoresis
Proteins - electrophoresis	Protein electrophoresis and Immunobinding - serum
Proteins - electrophoresis	Protein electrophoresis - serum
Proteins - urine	Protein electrophoresis
Proteinuria	Proteins - dosage - urine
Prothrombin - activity	Factor II - Prothrombin - activity - plasma
Prothrombin - Anti-IgG antibodies G	Anti-prothrombin antibodies - IgG - serum
Prothrombin - fragment 1+2	Fragment 1+2 - prothrombin - plasma
Prothrombin mutation	Factor II - Prothrombin - G20210G>A mutation
Prothrombin time	Prothrombine Ratio - plasma
Prothrombine Ratio - plasma	
Protide	Proteins - biological fluid
Protidemia	Proteins - dosage - serum
Proton pump - anti antibodies	Anti-proton pump antibodies - serum
Protopic®	Tacrolimus - FK-506
Protoporphyrins	Erythrocyte porphyrins
Protoporphyrins	Porphyrins - stool
Protozoaires	Helminthes - Protozoaires - direct diagnosis - PCR - Stool
Provitamin A	Beta-carotene
Prozac®	Fluoxetine - serum
PRSS1	Exome Panel for chronic and hereditary pancreatitis
PS activity	Protein S - activity - plasma
PS-100	Protein S100 - serum
PSA - Prostate-specific antigen free - serum	
PSA - Prostate-specific antigen total - Aspiration biopsy	
PSA - Prostate-specific antigen total - serum	
PSA ultra sensible	PSA - Prostate-specific antigen total - serum
PSA2	[-2]pro-PSA - Phi score calculation - serum
PSAL	PSA - Prostate-specific antigen free - serum
PSAT	PSA - Prostate-specific antigen total - Aspiration biopsy
PSAT	PSA - Prostate-specific antigen total - serum
Pseudocholinesterase	Cholinesterase - Enzymatic activity - serum
Psittacosis	Chlamydia psittaci - IgA serology - serum
Psittacosis	Chlamydia psittaci - IgG serology (1st intention) - serum
PSV1	Poliovirus - serology - serum
PTCH1	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
PTCH2	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
PTEN	DNA NGS Panel for brain solid tumors
PTEN	DNA NGS Panel for prostate cancer
PTEN	DNA NGS Panel solid tumor Breast
PTEN	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
PTEN	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
PTEN	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
PTEN	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
PTEN	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
pteroyl-monoglutamic acid	Erythrocyte folic acid
pteroyl-monoglutamic acid	Folic acid - serum

pteroylglutamic acid	Erythrocyte folic acid
pteroylglutamic acid	Folic acid - serum
PTH 1-84	PTH - Intact parathormone - aspiration biopsy
PTH 1-84	PTH - Intact parathormone - serum
PTH - intact	PTH - Intact parathormone - aspiration biopsy
PTH - intact	PTH - Intact parathormone - serum
PTH - Intact parathormone - aspiration biopsy	
PTH - Intact parathormone - serum	
PTH-like	PTH-RP - Parathormone related protein - plasma
PTH-related peptide	PTH-RP - Parathormone related protein - plasma
PTH-RP - Parathormone related protein - plasma	
PTHrP	PTH-RP - Parathormone related protein - plasma
PTT	Factor Willebrand protease activity - ADAMTS 13
Puce à ADN	Constitutional Molecular Karyotype - SNP ARRAY - Product of conception
Pulmonary	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
Pulmonary adenocarcinoma	DNA NGS Panel solid tumor Lung
Pulmonary adenocarcinoma	DNA NGS Panel solid tumor Lung extended
Pulmonary RNA-seq	RNA NGS Panel for lung cancers
Purpura thrombotique thrombocytopénique	Factor Willebrand protease activity - ADAMTS 13
Puumala	Hantavirus - IgG/IgM serology - serum
PV1	Poliovirus - serology - serum
PVL toxin detection (Staphylococcus aureus) - PCR on strain	
Pyridoxal	Vitamin B6 - Pyridoxal phosphate
Pyridoxal phosphate	Vitamin B6 - Pyridoxal phosphate
Pyridoxamine	Vitamin B6 - Pyridoxal phosphate
Pyridoxine	Vitamin B6 - Pyridoxal phosphate
Pyruvate	Ketone bodies - Deproteinized heparin whole blood sample or CSF
Pyruvate	Pyruvic acid - Supernatant or CSF
Pyruvic acid	Ketone bodies - Deproteinized heparin whole blood sample or CSF
Pyruvic acid - Supernatant or CSF	
Pyzchiva	Ustekinumab - serum
Q fever	Coxiella burnetii (Q fever) - IgG and IgM serology, Phases I and II - serum
Q fever	Coxiella burnetii (Q fever) - Screening serology Total Ac Phase II - serum
QKI	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Quantiferon® TB Gold Plus	Interferon gamma production - Quantiferon® TB Gold Plus
Quantiferon test	Interferon gamma production - Quantiferon® TB Gold Plus
Quantitative RPR	Syphilis - VDRL or quantitative RPR
Quantitative VDRL	Syphilis - VDRL or quantitative RPR
Quetiapine	
R506Q mutation	Factor V Leiden - p.Arg506Gln mutation
Rabies	Rabies - IgG serology - serum
Rabies - anti-antibodies	Rabies - IgG serology - serum
Rabies - IgG serology - serum	
RAC	Anti-acetylcholine receptor antibodies - serum
RAD51B	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
RAD51C	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
RAD51C	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
RAD51C	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
RAD51D	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
RAD51D	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
RAF1	DNA NGS Panel for brain solid tumors
Raltegravir - Serum	
RAP	Renin active
Rapamune®	Sirolimus
Rapid Plasma Reagin	Syphilis - VDRL or quantitative RPR
RARE DISEASES	exome
RARE DISEASES	Exome with interpretation interface
RARE DISEASES	Nephrology exome

RAS status	DNA NGS Panel solid tumor Colon
RAS status	DNA NGS Panel solid tumor Colon and MSI status determination
RAS status	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
Rat flea	Rickettsioses - R. conorii, R. typhi - serology - serum
Rat louse	Rickettsioses - R. conorii, R. typhi - serology - serum
Ratio cuivre échangeable / total	Copper - exchangeable / free
RB1	DNA NGS Panel for brain solid tumors
RB1	Retinoblastoma oncogenetic panel Target gene: RB1
RBP	Retinol-Binding Protein - RBP
Rearrangement	RNA NGS panel for brain solid tumors
Rearrangement	RNA NGS Panel for cholangiocarcinoma
Rearrangement	RNA NGS Panel for pancreatic cancer
Rearrangement	RNA NGS Panel for soft tissue tumors : 20 genes
Rearrangement	RNA NGS Panel for thyroid cancer
Rearrangement	RNA NGS Panel solid tumors, Pan-organ : 65 genes
REC relative exchangeable copper	Copper - exchangeable / free
rectal	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
Rectal	DNA NGS Panel solid tumor Colon
Rectal	DNA NGS Panel solid tumor Colon and MSI status determination
Rectal	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
REDOX	Juvenalis - OxyCheck
Reducing sugars	Sugars - screening - Stool
réinterprétation d'exome	
Remicade	Infliximab
Remsima	Infliximab
Renal	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B, MET,	
Renin	
Renin - Standing or Sitting position	
Renin active	
Renin Active - lying	
Renin activity	Renin active
Renin-angiotensin system activity	Renin active
Renome	Nephrology exome
Repaglinide	Sulphonylureas - serum
residual disease LLC	Immunophenotyping of malignant hemopaties
Resistance mutation	DNA NGS Panel solid tumor Pan-organ : 64 genes
Resistance to macrolides	Mycoplasma genitalium Search for mutations associated with macrolide resistance
Resistance to second-line injectable antibiotics (aminoglycosides)	Mycobacteria - PCR detection of mutations associated with resistance to isoniazid,
Respiratory Infection Panel - direct diagnosis - PCR	
Respiratory Syncytial Virus	Respiratory syncytial virus - RSV - IgG serology - serum
Respiratory syncytial virus - RSV - IgG serology - serum	
RET	DNA NGS Panel for thyroid cancer
RET	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
RET	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
RET	RNA NGS Panel for lung cancers
RET	RNA NGS Panel for thyroid cancer
RET	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Retine	Retinoblastoma oncogenetic panel Target gene: RB1
Retinoblastoma oncogenetic panel Target gene: RB1	
Rétinoblastome	Retinoblastoma oncogenetic panel Target gene: RB1
Retinol	Vitamin A - Retinol - Plasma
Retinol-Binding Protein - RBP	
Reverse T3	rT3 - Reverse triiodothyronine
Reverse transcriptase - sequencing of the gene	HIV - Human Immunodeficiency Virus - genotyping (reverse transcriptase, protease and
Reverse triiodothyronine	rT3 - Reverse triiodothyronine
RF	Rheumatoid factors - IgA - serum
RF	Rheumatoid factors - serum
Eurofins Biomnis – International Division – Test guide Specialised medical pathology – Valid from : 01/08/2026	

RH3 Genotyping	Fetal RH3(E) - genotyping
RH3 Genotyping	Fetal RHc - genotyping
RHD	
Rhesus D	blood grouping - genotyping - EDTA whole blood
Rhesus D	
Rhesus D - foetal genotype	
Rhesus Kell	
Rhesus/Kell	Rhesus Kell
Rhesus/Kell antigens	Rhesus Kell
Rheumatoid factor - IgM	Rheumatoid factors - serum
Rheumatoid factors - IgA - serum	
Rheumatoid factors - serum	
Rhinovirus	Paediatric respiratory viruses (RSV / Metapneumovirus / Rhinovirus) - direct diagnosis -
Ri - anti-antibodies	Anti-neuron antibodies - serum
Riboflavin	Vitamin B2 - Riboflavin
Ribosomes - anti-antibodies	Anti-ribosome antibodies - serum
Rickettsia	Rickettsioses - R. conorii, R. typhi - serology - serum
Rickettsia mooseri	Rickettsioses - R. conorii, R. typhi - serology - serum
Rickettsia prowazekii	Rickettsioses - R. conorii, R. typhi - serology - serum
Rickettsia typhi	Rickettsioses - R. conorii, R. typhi - serology - serum
Rickettsioses - R. conorii, R. typhi - serology - serum	
Rifadine®	Rifampicin - Serum
Rifampicin - Serum	
R/IL-2	Interleukin (IL)-2 - receptor
Rilpivirine - Serum	
RIMIFON	INH - Isoniazid - Serum
Risk of Ovarian Malignancy Algorithm	HE4- ROMA calculation - serum
Risperdal®	Risperidone
Risperidone	
RIST	Total IgE - serum
Ristocetin cofactor	Von Willebrand factor - plasma
Ritalin	Methylphenidate Hydrochloride - Serum
Ritemvia®	Rituximab - serum
Ritonavir - Serum	
Rituxan®	Rituximab - serum
Rituximab	Immunophenotyping: Lymphocytes B - CD19 - EDTA whole blood
Rituximab - serum	
Rituzena®	Rituximab - serum
Rivaroxaban - Anti Xa activity measurement - plasma	
Rivotril®	Clonazepam - Serum
Rixathon®	Rituximab - serum
Riximyo®	Rituximab - serum
RNA	Targeted RNA sequencing
RNA NGS panel for brain solid tumors	
RNA NGS Panel for cholangiocarcinoma	
RNA NGS Panel for lung cancers	
RNA NGS Panel for pancreatic cancer	
RNA NGS Panel for soft tissue tumors : 20 genes	
RNA NGS Panel for thyroid cancer	
RNA NGS Panel solid tumors, Pan-organ : 65 genes	
RNA panel	RNA NGS panel for brain solid tumors
RNA panel	RNA NGS Panel for cholangiocarcinoma
RNA panel	RNA NGS Panel for pancreatic cancer
RNA panel	RNA NGS Panel for soft tissue tumors : 20 genes
RNA panel	RNA NGS Panel for thyroid cancer
RNA polymerase	Markers in progressive systemic sclerosis(Scleroderma) - serum
RNA seq	RNA NGS panel for brain solid tumors
RNA seq	RNA NGS Panel for cholangiocarcinoma

RNA seq	RNA NGS Panel for soft tissue tumors : 20 genes
RNA seq	RNA NGS Panel for thyroid cancer
RNA seq	RNA NGS Panel solid tumors, Pan-organ : 65 genes
RNA-seq	RNA NGS Panel for cholangiocarcinoma
RNAseq	Targeted RNA sequencing
RO52	Anti-ENA antibodies (SSA, SSB, Sm, Rnp, Jo1, Scl-70) - screening and titration - Serum
RoActemra	Tocilizumab - serum
ROCA	HE4- ROMA calculation - serum
Rocephine®	Ceftriaxone - Serum
ROMA	HE4- ROMA calculation - serum
ROMA algorithm	HE4- ROMA calculation - serum
ROMA index	HE4- ROMA calculation - serum
ROMA Index	HE4- ROMA calculation - serum
Ropivacaine®	Naropeine - Serum
ROS1	RNA NGS panel for brain solid tumors
ROS1	RNA NGS Panel for lung cancers
ROS1	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Rose-Bengal	Brucellosis - Serology (Rose bengale + brucellacapt) - serum
Rosenthal - factor	Factor XI - Rosenthal - plasma
Rosenthal factor	Factor XI - Rosenthal - plasma
Roseola	Herpes virus type 6 (HHV6) - IgG serology
Roseola	Herpes virus type 6 (HHV6) - IgM serology
Rosner index	Circulating anticoagulant - screening - plasma
Rotavirus - rapid screening - Stools	
Rotavirus A	Gastrointestinal Panel - direct diagnosis - PCR - Stools
Roalcyte	Ganciclovir - plasma
RP	Prothrombine Ratio - plasma
RPCa	Activated protein C resistance - plasma
RPR	Syphilis - VDRL or quantitative RPR
rfl	Non tuberculosis mycobacteria - PCR detection of macrolide and aminoglycoside
rrs	Non tuberculosis mycobacteria - PCR detection of macrolide and aminoglycoside
RSSE	Tick-borne encephalitis - TBE - IgG/IgM serology - serum
RST	Soluble transferrin receptor - serum
RsTf	Soluble transferrin receptor - serum
rT3	rT3 - Reverse triiodothyronine
rT3 - Reverse triiodothyronine	
RTEL1	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
RTEL1	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
Rubella - antibody avidity - IgG - serum	
Rubella - Western blot - serum	
Rubella virus - antibody avidity	Rubella - antibody avidity - IgG - serum
Rufinamide - Serum	
RVIH3	HIV - Human Immunodeficiency Virus - genotyping (reverse transcriptase, protease and
RYR1	Pharmacogenetic Passport - PharmacoPass Expert
Rythmodan®	Disopyramide - serum
Rythmol®	Propafenone - Serum
S100 proteins	Serum calprotectin - Serum
S100A8/S100A9 heterodimer	Serum calprotectin - Serum
S65C mutation	Haemochromatosis - HFE gene - p.Ser65Cys mutation
Sabril®	Vigabatrin - Serum
Saccharomyces cerevisiae - anti-antibodies	Anti-Saccharomyces cerevisiae antibodies - IgG/IgA - serum
SAE	Myosite Markers - serum
SAICAR	Adenylosuccinase - deficiency syndrome screening - urine
salivary lithiasis	Gallstones/Saliva/Other - morphologic and infrared analysis
salivary stone	Gallstones/Saliva/Other - morphologic and infrared analysis
Salmonella sp	Gastrointestinal Panel - direct diagnosis - PCR - Stools
Salmonellosis - serology - serum	
Sandell-Kolthoff reaction	Iodine total - serum

Sandell-Kolthoff reaction	Iodine - urine
Sandimun®	Cyclosporin
Sapovirus GI/GII/GIV/GV	Gastrointestinal Panel - direct diagnosis - PCR - Stools
Sarcoma	RNA NGS Panel for soft tissue tumors : 20 genes
Sarcoma RNA-seq	RNA NGS Panel for soft tissue tumors : 20 genes
Sarcomas	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Sarcoptes scabiei	scabies - screening
SARS-CoV-2	COVID 19 / Influenza virus A and B - panel - direct diagnosis - PCR
SARS-CoV-2	COVID-19 (SARS-CoV-2) IgG serology anti-S
SARS-CoV-2	Respiratory Infection Panel - direct diagnosis - PCR
saturnism	Lead - whole blood
SBG	Sex-Binding Globulin - SHBG
SBP	Sex-Binding Globulin - SHBG
SCA	Anti-adrenal gland antibodies - serum
scabies	scabies - screening
scabies - screening	
scabies mite	scabies - screening
SCC	SCC-TA4 antigen - aspiration biopsy
SCC	SCC-TA4 antigen - serum
SCC-TA4 antigen - aspiration biopsy	
SCC-TA4 antigen - serum	
schistocyte	Erythrocytes morphological aspect
Schistosoma haematobium	Schistosomiasis - direct diagnosis - PCR
Schistosoma mansoni	Schistosomiasis - direct diagnosis - PCR
Schistosomas	Schistosomiasis - confirmation serology - serum
Schistosomas	Schistosomiasis - direct diagnosis - PCR
Schistosomas	Schistosomiasis - screening serology - serum
Schistosomiasis	Schistosomiasis - direct diagnosis - PCR
Schistosomiasis - confirmation serology - serum	
Schistosomiasis - direct diagnosis - PCR	
Schistosomiasis - screening serology - serum	
Schwannoma	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
SCN-	Thiocyanates - ion - Serum
SCN-	Thiocyanates - ion - urine
SCN5A	Cardiogenetic panel or exome
Score matutes	Immunophenotyping of malignant hemopaties
Screening for heparin-induced thrombopaenia	Anti-PF4 antibodies - heparin - plasma
SCSA	DNA fragmentation test - DNA decondensation - SCSA - Sperm
SDC4	RNA NGS Panel solid tumors, Pan-organ : 65 genes
SDHA	DHEAS - Dehydroepiandrosterone sulfate
SDHA	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
SDHA	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
SDHAF2	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
SDHAF2	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
SDHAF2	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
SDHB	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
SDHB	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
SDHB	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
SDHC	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
SDHC	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
SDHC	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
SDHD	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
SDHD	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
SDHD	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
SDHEA	DHEAS - Dehydroepiandrosterone sulfate
Se	Selenium
Se	Selenium - urine
Search for B and binary toxins of C. difficile.	Clostridium difficile - toxins-screening by PCR

SEBG	Sex-Binding Globulin - SHBG
Selectivity index	Protein electrophoresis
Selenium	
Selenium - urine	
Seloken®	Metoprolol - Serum
Sensibilité aux antibiotiques	Anaerobic and aerobic bacteria - susceptibility testing
Sensitivity to antifungal agents	Fungus - Yeasts - MIC: Minimum Inhibitory Concentration
Sensitivity to antifungal agents	Yeasts - antifungal susceptibility testing
Seoul	Hantavirus - IgG/IgM serology - serum
SEQONE	Exome with interpretation interface
Séquençage de 3e génération	Séquencage des gènes BRCA1 et BRCA2
Séquençage des gènes BRCA1 et BRCA2	
Séquençage long read	Séquencage des gènes BRCA1 et BRCA2
Sequencing	DNA NGS Panel for brain solid tumors
Sequencing	DNA NGS Panel for cholangiocarcinoma
Sequencing	DNA NGS Panel for pancreatic cancer
Sequencing	DNA NGS Panel for prostate cancer
Sequencing	DNA NGS Panel for thyroid cancer
Sequencing	DNA NGS Panel solid tumor Breast
Sequencing	DNA NGS Panel solid tumor Colon
Sequencing	DNA NGS Panel solid tumor Colon and MSI status determination
Sequencing	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
Sequencing	DNA NGS Panel solid tumor Endometrium
Sequencing	DNA NGS Panel solid tumor Endometrium and MSI status determination
Sequencing	DNA NGS Panel solid tumor GIST
Sequencing	DNA NGS Panel solid tumor Lung
Sequencing	DNA NGS Panel solid tumor Lung extended
Sequencing	DNA NGS Panel solid tumor Melanoma
Sequencing	Homologous recombination deficiency (HRD) and somatic mutations of the BRCA1 and
Sequencing	RNA NGS panel for brain solid tumors
Sequencing	RNA NGS Panel for cholangiocarcinoma
Sequencing	RNA NGS Panel for lung cancers
Sequencing	RNA NGS Panel for pancreatic cancer
Sequencing	RNA NGS Panel for soft tissue tumors : 20 genes
Sequencing	RNA NGS Panel for thyroid cancer
Sequencing	RNA NGS Panel solid tumors, Pan-organ : 65 genes
SEQUENCING	Prenatal exome
Sequencing of the gene encoding the envelope protein gp120	HIV - Human Immunodeficiency Virus - genotyping (envelope, tropism): antiretroviral
Serecor®	Hydroquinidine - Serum
Seresta®	Oxazepam - Serum
Serodiagnosis of baker lung (Alveolitis Allergic Extrinsic) - screening - serum	
Serodiagnosis of cheesemonger lung Alveolitis Allergic Extrinsic) - screening - serum	
Serodiagnosis of Domestic lung (Alveolitis Allergic Extrinsic) - screening - serum	
Serodiagnosis of Mushroom worker's lung (Alveolitis Allergic Extrinsic) - screening - serum	
Sérologie Palustre (IgM/IgG)	
Seroplex	Escitalopram - Serum
Serotonin - whole blood	
Serotonin métabolite	5-HIAA - 5-hydroxyindoleacetic acid - Urine
Sertraline - serum	
sérum and plasma EDTA	
Serum calprotectin - Serum	
Serum markers for Trisomy 21	Trisomy 21 - 1st trimester combined screening
Serum markers for Trisomy 21	Trisomy 21 - second-trimester maternal serum markers - 2 parameters
Serum markers for Trisomy 21	Trisomy 21 - second-trimester maternal serum markers - 3 parameters
Serum proteins - immunofixation	Immunobinding - serum
Serum proteins - immunofixation	Protein electrophoresis and Immunobinding - serum
Serum screening for Trisomy 21 risk	Trisomy 21 - 1st trimester combined screening
Serum screening for Trisomy 21 risk	Trisomy 21 - second-trimester maternal serum markers - 2 parameters

Serum screening for Trisomy 21 risk	Trisomy 21 - second-trimester maternal serum markers - 3 parameters
SETD2	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
SETD2	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
Sex Hormone-Binding Globulin	Sex-Binding Globulin - SHBG
Sex-Binding Globulin	Sex-Binding Globulin - SHBG
Sex-Binding Globulin - SHBG	
Sézary - cells	Sézary cells - screening - smear
Sézary cells - screening - smear	
sFlt-1	Pre-eclampsia sFlt-1 et PIGF - Predictive test - sFlt-1/PIGF ratio - serum
Shiga-like toxin-producing E. coli (STEC) stx1/stx2	Gastrointestinal Panel - direct diagnosis - PCR - Stools
Shigella/Enteroinvasive E. coli (EIEC)	Gastrointestinal Panel - direct diagnosis - PCR - Stools
Shingles - immunity test - IgG antibodies	Varicella zoster virus - IgG serology - serum
Short QT syndrome	Cardiogenetic panel or exome
Si	Silicon - Plasma
Sialic acid free - urines	
Sialic acid-deficient transferrin	Carbohydrate-deficient transferrin - serum
Sickle cell disease	Haemoglobin - electrophoresis - whole blood
Sickle cell disease	
sickle cells	Erythrocytes morphological aspect
Sickness disease	Leptospirosis - IgM serology - serum
Siderophilin	Transferrin - serum
slgA	IgA - Immunoglobulin A secretory - Stool
Silicon - Plasma	
Simponi®	Golimumab - serum
Sin	Hantavirus - IgG/IgM serology - serum
Sirolimus	
Skeletal muscle	Anti-striated muscle antibodies - serum
Skin	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
Skin	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
Skin - anti-antibodies	Anti-epidermal basement membrane and intracellular substance antibodies - serum
Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R, MITF,	
SLA/LP - anti-antibodies	Anti-SLA antibodies - Soluble Liver Antigen - serum
SLC25A11	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
SLC25A11	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
SLC34A2	RNA NGS Panel solid tumors, Pan-organ : 65 genes
SLCO1B1	Pharmacogenetic Passport - PharmacoPass Expert
Sleeping sickness - screening for parasite	Trypanosoma gambiense - screening - CSF
Sleeping sickness - serology	Trypanosoma gambiense - serology - CSF
Sleeping sickness - serology	Trypanosoma gambiense - serology - serum
SLO syndrome	7-Dehydrocholesterol
SMAD4	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
SMAD4	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
SMARCB1	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
SMARCB1	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
SMC	IGF-1 - Somatomedin C - Serum
Smith-Lemli-Opitz - syndrome	7-Dehydrocholesterol
Smith-Lemli-Opitz syndrome	7-Dehydrocholesterol
Smooth muscles - anti-antibodies	Anti-smooth muscle antibodies - screening and titration - serum
SMT	Melatonin - 24-hour urine collection
SMT	Melatonin - Urine
SND1	RNA NGS Panel solid tumors, Pan-organ : 65 genes
snp	Constitutional Molecular Karyotype-SNP ARRAY
SNP	Constitutional Molecular Karyotype-SNP ARRAY
SNP	Constitutional Molecular Karyotype - SNP ARRAY - Product of conception
SNP -array	Constitutional Molecular Karyotype - SNP ARRAY - Product of conception
SNP array	Constitutional Molecular Karyotype-SNP ARRAY
SNV	Cardiogenetic panel or exome
SNV	exome

SNV	Exome Congenital heart defects
SNV	Exome Isolated or syndromic intellectual disability and syndromic neurodevelopmental
SNV	Exome Panel for chronic and hereditary pancreatitis
SNV	Exome with interpretation interface
SNV	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
SNV	Hantavirus - IgG/IgM serology - serum
SNV	Nephrology exome
SNV	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
SNV	Oncogenetic exome
SNV	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
SNV	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
SNV	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
SNV	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
SNV	Prenatal exome
SNV	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
SNV	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
SNV, CNV, Indels, variants de structure	Séquencage des gènes BRCA1 et BRCA2
SNVs / Indels: AKT1; ALK; BRAF; BRCA1; BRCA2; EGFR; ERBB2; ESR1; FGFR2; FGFR3;	Circulating tumor DNA NGS Panel - Peripheral blood from Cell Free DNA tube : ambiant
SOD	Superoxide dismutase - SOD
Sodium	Electrolytes stool - Stool
Sodium valproate	Valproic acid - serum
Soft tissue	RNA NGS Panel for soft tissue tumors : 20 genes
Soft tissue	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Solian®	Amisulpride - Serum
Solid tumor	Circulating tumor DNA NGS Panel - Peripheral blood from Cell Free DNA tube : ambiant
Solid tumor	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
Solid tumor	DNA NGS Panel solid tumor Lung extended
Solid tumor	DNA NGS Panel solid tumor Pan-organ : 64 genes
Solid tumors RNA-seq	RNA NGS Panel solid tumors, Pan-organ : 65 genes
soluble antigen	Candida - Mannane antigen - serum
Soluble antigen - Aspergillus fumigatus	Aspergillosis - soluble antigen - testing - BAL
Soluble antigen - Aspergillus fumigatus	Aspergillosis - soluble antigen - testing - serum
Soluble antigen of Legionella pneumophila	Legionellosis- antigenes LP1 - urines
Soluble bacterial antigens Streptococcus pneumoniae - CSF	
Soluble bacterial antigens Streptococcus pneumoniae - urines	
Soluble complexes - plasma	
Soluble cryptococcal antigen	Cryptococcus neoformans - soluble antigen - testing
Soluble Interleukin-2 receptor	Interleukin (IL)-2 - receptor
Soluble Liver Antigen - anti-antibodies Soluble Liver Antigen - anti-antibodies	Anti-SLA antibodies - Soluble Liver Antigen - serum
Soluble nuclear anti-antigen antibodies	Anti-ENA antibodies (SSA, SSB, Sm, Rnp, Jo1, Scl-70) - screening and titration - Serum
Soluble transferrin receptor	Soluble transferrin receptor - serum
Soluble transferrin receptor - serum	
Somathormone	GH - Growth hormone - Serum
Somatic	Circulating tumor DNA NGS Panel - Peripheral blood from Cell Free DNA tube : ambiant
Somatic	DNA NGS Panel for brain solid tumors
Somatic	DNA NGS Panel for cholangiocarcinoma
Somatic	DNA NGS Panel for pancreatic cancer
Somatic	DNA NGS Panel for prostate cancer
Somatic	DNA NGS Panel for thyroid cancer
Somatic	DNA NGS Panel solid tumor Breast
Somatic	DNA NGS Panel solid tumor Colon
Somatic	DNA NGS Panel solid tumor Colon and MSI status determination
Somatic	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
Somatic	DNA NGS Panel solid tumor Endometrium
Somatic	DNA NGS Panel solid tumor Endometrium and MSI status determination
Somatic	DNA NGS Panel solid tumor GIST
Somatic	DNA NGS Panel solid tumor Lung
Somatic	DNA NGS Panel solid tumor Lung extended

Somatic	DNA NGS Panel solid tumor Melanoma
Somatic	DNA NGS Panel solid tumor Pan-organ : 64 genes
Somatic	Homologous recombination deficiency (HRD) and somatic mutations of the BRCA1 and
Somatic	RNA NGS panel for brain solid tumors
Somatic	RNA NGS Panel for cholangiocarcinoma
Somatic	RNA NGS Panel for lung cancers
Somatic	RNA NGS Panel for pancreatic cancer
Somatic	RNA NGS Panel for soft tissue tumors : 20 genes
Somatic	RNA NGS Panel for thyroid cancer
Somatic	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Somatocrinin	GHRH - Growth-Hormone-Releasing-Hormone - plasma
Somatomedin C	IGF-1 - Somatomedin C - Serum
Somatostatin - plasma	
Somatotrop Hormone	GH - Growth hormone - Serum
Somatotrophic Hormone	GH - Growth hormone - Serum
Somatotrophin	GH - Growth hormone - Serum
Somatotropin	GH - Growth hormone - Serum
Somatotropin Release Inhibiting Factor	Somatostatin - plasma
SOPK	Steroids panel - Search polycystic ovary syndrome
Sotalex®	Sotalol - serum
Sotalol - serum	
South American trypanosomiasis	Trypanosoma cruzi - serology - serum
South American trypanosomiasis - confirmation	Trypanosoma cruzi - confirmation - serum
Spanor®	Doxycycline - Serum
Specific fibrin degradation product	D-Dimers - plasma
Specific IgE : food allergen testing - Trophatop® adult - fx5/fx24/fx25	
Specific IgE : food allergen testing - Trophatop® children - fx26/fx27/fx28	
Specific IgE : food allergens - allergen mix - serum	
Specific IgE : hymenoptera venom - single allergen	
Specific IgE : inhalant allergen testing - Phadiatop®	
Specific IgE : inhalant allergens - allergen mix	
Specific IgE : latex - single allergen	
Specific IgE : Phadia - food allergens - single allergen	
Specific IgE : Phadia - inhalant allergens - single allergen	
Specific IgE - inhalant - single allergen	Specific IgE inhalant allergens - single allergen - GSD Gold Standard Diagnostic -
Specific IgE - inhalant - single allergen	Specific IgE : Phadia - inhalant allergens - single allergen
Specific IgE - Respiratory and food allergen screen -Phadiatop® infant	
Specific IgE (CAP-RAST) - inhalant allergens - single allergen	Specific IgE inhalant allergens - single allergen - GSD Gold Standard Diagnostic -
Specific IgE (CAP-RAST) - inhalant allergens - single allergen	Specific IgE : Phadia - inhalant allergens - single allergen
Specific IgE inhalant allergens - single allergen - GSD Gold Standard Diagnostic - serum,	
Specific IgE medicines - single allergen - GSD Gold Standard Diagnostic - serum, plasma	
Specific IgE medicines - single allergen-Phadia	
Specific IgE (RAST) - single allergen	Specific IgE inhalant allergens - single allergen - GSD Gold Standard Diagnostic -
Specific IgE (RAST) - single allergen	Specific IgE : Phadia - inhalant allergens - single allergen
Specific IgEs - CLA30 - food allergens	IgE against multiple individual allergens - food allergens - Serum
Specific IgEs - CLA30 - inhalant allergens	IgE against multiple individual allergens - inhalant allergens - Serum
Specific IgEs - CLA30 - mixed allergens	IgE against multiple individual allergens - Food and Inhalant panels - Serum
Specific IgEs - food - single allergen	Specific IgE : Phadia - food allergens - single allergen
Specific IgEs - food mixes	Specific IgE : food allergens - allergen mix - serum
Specific IgEs - inhalant mixes	Specific IgE : inhalant allergens - allergen mix
Specific IgEs (CAP-RAST) - food allergen combination	Specific IgE : food allergens - allergen mix - serum
Specific IgEs (CAP-RAST) - food allergens - single allergen	Specific IgE : Phadia - food allergens - single allergen
Specific IgEs (CAP-RAST) - hymenoptera venom	Specific IgE : hymenoptera venom - single allergen
Specific IgEs (CAP-RAST) - inhalant allergen mix	Specific IgE : inhalant allergens - allergen mix
Specific IgEs: inhalant mixes	Specific IgE : inhalant allergen testing - Phadiatop®
Specific IgEs (RAST) - food allergens - allergen mix	Specific IgE : food allergens - allergen mix - serum
Specific IgEs (RAST) - food allergens - single allergen	Specific IgE : Phadia - food allergens - single allergen
Specific IgEs (RAST) - hymenoptera venom	Specific IgE : hymenoptera venom - single allergen

Specific IgEs (RAST) - inhalant allergen mix	Specific IgE : inhalant allergens - allergen mix
SPECIFIC IgG - serum	
SPECIFIC IgG4 - serum	
Sperm - anti-antibodies	Anti-sperm antibodies - indirect screening
Sperm Chromatin Structure Assay	DNA fragmentation test - DNA decondensation - SCSA - Sperm
spherocytes	Erythrocytes morphological aspect
Spingolipid	Anti-glycolipid antibodies - serum
SPINK1	Exome Panel for chronic and hereditary pancreatitis
Sporanox®	Itraconazole and hydroxylated metabolite - Serum
Squamous Cell Carcinoma	SCC-TA4 antigen - aspiration biopsy
Squamous Cell Carcinoma	SCC-TA4 antigen - serum
SRIF	Somatostatin - plasma
SRP - anti-antibodies	Myosite Markers - serum
SRV A/B	Paediatric respiratory viruses (RSV / Metapneumovirus / Rhinovirus) - direct diagnosis -
SS18	RNA NGS Panel for soft tissue tumors : 20 genes
SS18	RNA NGS Panel solid tumors, Pan-organ : 65 genes
SSA	Anti-ENA antibodies (SSA, SSB, Sm, Rnp, Jo1, Scl-70) - screening and titration - Serum
SSX1	RNA NGS Panel for soft tissue tumors : 20 genes
SSX1	RNA NGS Panel solid tumors, Pan-organ : 65 genes
SSX2	RNA NGS Panel for soft tissue tumors : 20 genes
SSX2	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Stablon®	Tianeptine
Stallertest	Specific IgE : inhalant allergen testing - Phadiatop®
STAT6	RNA NGS Panel for soft tissue tumors : 20 genes
STAT6	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Steatorrhoea - fat dosage - Stool	
SteatoTest	FibroMAX - . - serum
SteatoTest 2®.	NASH-FibroTest - serum
Stelara®	Ustekinumab - serum
Steqeyma	Ustekinumab - serum
Steroid cell antibodies	Anti-adrenal gland antibodies - serum
Steroid profile - urine	
Steroids panel - Examination of gonad function	
Steroids panel - Exploration of hirsutism	
Steroids panel - Search polycystic ovary syndrome	
Steroids panel - Testing for a disorder in steroid biosynthesis	
STH	GH - Growth hormone - Serum
STI screening - multiplex gene amplification	
Stiripentol - Serum	
STK11	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
STK11	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
STN1	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
STN1	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
Stomach	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
Stomach - anti-antibodies	Anti-parietal cell antibodies - stomach - serum
Stone analysis	Urinary stone - morphologic and infrared analysis
streptococcic serology	Streptococcus A - Serology- Ac anti streptolysin O (ASLO) - serum
Streptococcus A - Serology- Ac anti streptolysin O (ASLO) - serum	
Streptococcus A- Serology-Abs to streptodornase B (ASD) - serum	
Streptococcus pneumoniae	Designated bacterium - direct diagnosis - PCR
Streptococcus pneumoniae	Soluble bacterial antigens Streptococcus pneumoniae - CSF
Streptococcus pneumoniae	Soluble bacterial antigens Streptococcus pneumoniae - urines
Streptococcus pneumoniae - capsular polysaccharide - serology - serum	
Streptococcus pneumoniae - serology	Streptococcus pneumoniae - capsular polysaccharide - serology - serum
stretodornase	Streptococcus A- Serology-Abs to streptodornase B (ASD) - serum
Striated muscles - anti-antibodies	Anti-striated muscle antibodies - serum
Stribild	Tenofovir - plasma
Strongyloidiasis - serology	Strongyloidiasis - serology - serum

Strongyloidiasis - serology - serum	
Stuart - factor	Factor X - Stuart-Prower - plasma
Stuart factor	Factor X - Stuart-Prower - plasma
Subutex	Buprenorphine - Serum
Subutex®	Buprenorphine and metabolites - urine
Succinyl AICAR	Adenylosuccinase - deficiency syndrome screening - urine
Sudden cardiac death	Cardiogenetic panel or exome
Sudden death	Cardiogenetic panel or exome
SUFU	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
SUFU	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
Sugars - screening - Stool	
Sulfatides - urine	
Sulfatoxymelatonin	Melatonin - 24-hour urine collection
Sulfatoxymelatonin	Melatonin - Urine
Sulphonylureas - serum	
Superoxide dismutase - SOD	
Susceptibility testing - MIC	Helicobacter pylori - culture
Syndrome Upshaw-Schulman	Factor Willebrand protease activity - ADAMTS 13
Syphilis	Genital ulcérations panel - direct diagnosis - PCR
Syphilis	Syphilis - titration - CSF
Syphilis	Treponema pallidum - Detection via PCR
Syphilis - IgG confirmation serology - serum	
Syphilis - IgM confirmation serology - serum	
Syphilis - IgM serology - serum	
Syphilis - titration - CSF	
Syphilis - Treponemique test - screening serology - serum	
Syphilis - VDRL or quantitative RPR	
T lymphocytes	Immunophenotyping: Lymphocytes CD3-CD4-CD8 - EDTA whole blood
T21 - serum markers	Trisomy 21 - 1st trimester combined screening
T21 - serum markers	Trisomy 21 - second-trimester maternal serum markers - 2 parameters
T21 - serum markers	Trisomy 21 - second-trimester maternal serum markers - 3 parameters
T21 MSM	Trisomy 21 - 1st trimester combined screening
T21 MSM	Trisomy 21 - second-trimester maternal serum markers - 2 parameters
T21 MSM	Trisomy 21 - second-trimester maternal serum markers - 3 parameters
t-PA	Tissue plasminogen activator - tPA - antigen - plasma
T3, T4, T8	Immunophenotyping: Lymphocytes CD3-CD4-CD8 - EDTA whole blood
T3 - anti-antibodies	Anti-T3 antibodies - triiodothyronine - serum
T3 - Triiodothyronine - urine	
T4 - anti-antibodies	Anti-T4 antibodies - thyroxine - serum
T4 - Thyroxine - urine	
TA 4	SCC-TA4 antigen - aspiration biopsy
TA 4	SCC-TA4 antigen - serum
TA4 antigen	SCC-TA4 antigen - aspiration biopsy
TA4 antigen	SCC-TA4 antigen - serum
TACC1	RNA NGS Panel solid tumors, Pan-organ : 65 genes
TACC3	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Tacrolimus	Tacrolimus - FK-506
Tacrolimus - FK-506	
Taenia	Cysticercosis - Taenia Solium - IgG serology - serum
Taenia solium	Cysticercosis - confirmation serology
Taenia Solium	Cysticercosis - Taenia Solium - IgG serology - serum
TAG-72	CA 72-4 antigen - aspiration biopsy
TAG-72	CA 72-4 antigen - serum
TAG-72 antigen	CA 72-4 antigen - aspiration biopsy
TAG-72 antigen	CA 72-4 antigen - serum
Takrozem	Tacrolimus - FK-506
Taloxa®	Felbamate - Serum
Targeted RNA sequencing	

Targeted Sequencing - Postnatal	
Targeted Sequencing - Prenatal	
Targeted therapy	Circulating tumor DNA NGS Panel - Peripheral blood from Cell Free DNA tube : ambiant
Targeted therapy	DNA NGS Panel for brain solid tumors
Targeted therapy	DNA NGS Panel for cholangiocarcinoma
Targeted therapy	DNA NGS Panel for pancreatic cancer
Targeted therapy	DNA NGS Panel for prostate cancer
Targeted therapy	DNA NGS Panel for thyroid cancer
Targeted therapy	DNA NGS Panel solid tumor Breast
Targeted therapy	DNA NGS Panel solid tumor Colon
Targeted therapy	DNA NGS Panel solid tumor Colon and MSI status determination
Targeted therapy	DNA NGS Panel solid tumor colo-rectal extended : 22 genes
Targeted therapy	DNA NGS Panel solid tumor Endometrium
Targeted therapy	DNA NGS Panel solid tumor Endometrium and MSI status determination
Targeted therapy	DNA NGS Panel solid tumor GIST
Targeted therapy	DNA NGS Panel solid tumor Lung
Targeted therapy	DNA NGS Panel solid tumor Lung extended
Targeted therapy	DNA NGS Panel solid tumor Melanoma
Targeted therapy	DNA NGS Panel solid tumor Pan-organ : 64 genes
Targeted therapy	Homologous recombination deficiency (HRD) and somatic mutations of the BRCA1 and
Targeted therapy	RNA NGS panel for brain solid tumors
Targeted therapy	RNA NGS Panel for cholangiocarcinoma
Targeted therapy	RNA NGS Panel for lung cancers
Targeted therapy	RNA NGS Panel for pancreatic cancer
Targeted therapy	RNA NGS Panel for soft tissue tumors : 20 genes
Targeted therapy	RNA NGS Panel for thyroid cancer
Targeted therapy	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Targocid®	Teicoplanine par immunoturbidimétrie (Thermo Scientific) - Sérum
TAU - PTAU - Amyloid proteins - LCR	
Tavanic®	Levofloxacin - Serum
Tazocilline®	Piperacillin - Serum
TBE virus - serology	Tick-borne encephalitis - TBE - IgG/IgM serology - serum
TBG	Thyroxine-binding globulin - TBG - serum
TBII	Anti-TSH receptor antibodies
TCE + TCA	Trichloroacetic acid + Trichloroethanol - TCE + TCA - urine
TCK	Activated Partial Thromboplastin Time - aPTT - plasma
TCT	Calcitonin - aspiration biopsy
TCT	Calcitonin - Serum
TDA	Erythrocyte direct Coombs
TeBG	Sex-Binding Globulin - SHBG
Tegretol®	Carbamazepine - serum
Teicoplanine par immunoturbidimétrie (Thermo Scientific) - Sérum	
Temgesic®	Buprenorphine and metabolites - urine
Temgesic®	Buprenorphine - Serum
Tenofovir - plasma	
Teralithe	Lithium par spectrophométrie - serum
Tercian®	Cyamemazine - Serum
TERF2IP	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
TERF2IP	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
TERT	DNA NGS Panel for thyroid cancer
TERT	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
TERT	RNA NGS panel for brain solid tumors
TERT	RNA NGS Panel solid tumors, Pan-organ : 65 genes
TERT	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
TERT promoter	DNA NGS Panel for brain solid tumors
Test d'Itano	Haemoglobin - electrophoresis - whole blood
Testicular - anti-antibodies	Anti-testicular antibodies - serum
Testing for steatorrhea	Steatorrhea - fat dosage - Stool

Testosterone	
Testosterone - man	
Testosterone - woman/child - serum	
Testosterone bioavailability - serum	
Testosterone Bioavailable - woman/ child - serum	
Testosterone free - Serum	
Testosterone oestradiol-binding globulin	Sex-Binding Globulin - SHBG
Testosterone transport protein	Sex-Binding Globulin - SHBG
Testosterone-Binding Globulin	Sex-Binding Globulin - SHBG
Tetanic	Tetanus - immunity test - IgG serology - serum
Tetanus	Tetanus - immunity test - IgG serology - serum
Tetanus - immunity test - IgG serology - serum	
Tetra Hydro aldosteron	Aldosterone - urine
Tetrachloroethylene - EDTA whole blood or Heparin	
Tetrachloroethylene - urine	
Tetrahydrocannabinol	Cannabis
Tétrahydrocannabinol	Cannabidoïdes - dosage - plasma
Tetraiodothyronine	T4 - Thyroxine - urine
Tétralogie de Fallot	Exome Congenital heart defects
TFA	ttrifluoroacetic acid TFA
ttrifluoroacetic acid TFA	
Tg	Thyroglobulin
TH50a	50% haemolysis time - alternative pathway complement - serum
Thalassemia	Haemoglobin - electrophoresis - whole blood
Thalassemia	
THC	Cannabidoïdes - dosage - plasma
THC-COOH	Cannabidoïdes - dosage - plasma
THC-COOH	Cannabidoïdes - dosage - urines
Theophylline - serum	
Theranostique	Séquencage des gènes BRCA1 et BRCA2
Thermal soluble protein	Bence Jones - screening
Thiamin	Vitamin B1 - Thiamin
Thienopyridine - resistance test	VASP - VAsodilator Stimulated Phosphoprotein - whole blood
Thiocyanates - ion - Serum	
Thiocyanates - ion - urine	
Thiopurine S-méthyltransférase phenotyping	TPMT Activity or Thiopurine S-Methyltransferase Phenotyping - activity
Third-generation cephalosporin-resistant Enterobacteriaceae	Antibiotic multi-resistant bacteria (MRB) - screening
Thrombin clotting time - plasma	
Thrombocystin	Serotonin - whole blood
Thrombotonin	Serotonin - whole blood
thrush	Candida - Mannane antigen - serum
Thyro Check	Juvenalis - Panel - ThyroCheck+
Thyrocalcitonin	Calcitonin - aspiration biopsy
Thyrocalcitonin	Calcitonin - Serum
Thyroglobulin	
Thyroglobulin - anti-antibodies	Anti-thyroglobulin antibodies
thyroid biomarkers	DNA NGS Panel for thyroid cancer
Thyroid cancer	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Thyroid microsomal - anti-antibodies	Anti-thyroid peroxidase antibodies
Thyroid NGS panel - Thyroid cancer	DNA NGS Panel for thyroid cancer
Thyroid NGS panel - Thyroid cancer	RNA NGS Panel for thyroid cancer
Thyroid peroxidase - anti-antibodies	Anti-thyroid peroxidase antibodies
thyroid RNA-seq	RNA NGS Panel for thyroid cancer
Thyroid-stimulating hormone - third generation	TSH - Thyroid-stimulating hormone - Third generation - serum
Thyroid-stimulating hormone - Third generation	TSH - Thyroid-stimulating hormone - Third generation - serum
Thyrotropin - ultrasensitive - third generation	TSH - Thyroid-stimulating hormone - Third generation - serum
Thyrotropin-releasing hormone - ultrasensitive third generation	TSH - Thyroid-stimulating hormone - Third generation - serum
Thyroxine	T4 - Thyroxine - urine

Thyroxine - anti-antibodies	Anti-T4 antibodies - thyroxine - serum
Thyroxine-binding globulin - TBG - serum	
Ti	Titanium
Tianeptine	
TIBC	Iron - total binding capacity - serum
Tick	Bartonellosis - B. henselae - B. quintana - IgG serology - serum
Tick	Coxiella burnetii (Q fever) - IgG and IgM serology, Phases I and II - serum
Tick	Coxiella burnetii (Q fever) - Screening serology Total Ac Phase II - serum
Tick	Human babesiosis - serology - serum
Tick	Human ehrlichiosis - IgG/IgM serology - serum
Tick	Tick-borne encephalitis - TBE - IgG/IgM serology - serum
Tick	Tularemia - Francisella tularensis - serology total antibodies - serum
Tick bite	Bartonellosis - B. henselae - B. quintana - IgG serology - serum
Tick bite	Coxiella burnetii (Q fever) - IgG and IgM serology, Phases I and II - serum
Tick bite	Coxiella burnetii (Q fever) - Screening serology Total Ac Phase II - serum
Tick bite	Human babesiosis - serology - serum
Tick bite	Human ehrlichiosis - IgG/IgM serology - serum
Tick bite	Tick-borne encephalitis - TBE - IgG/IgM serology - serum
Tick bite	Tularemia - Francisella tularensis - serology total antibodies - serum
Tick Born Encephalitis	Tick-borne encephalitis - TBE - IgG/IgM serology - serum
Tick-borne encephalitis (TBE)	Tick-borne encephalitis - TBE - IgG/IgM serology - serum
Tick-borne disease	Bartonellosis - B. henselae - B. quintana - IgG serology - serum
Tick-borne disease	Coxiella burnetii (Q fever) - IgG and IgM serology, Phases I and II - serum
Tick-borne disease	Coxiella burnetii (Q fever) - Screening serology Total Ac Phase II - serum
Tick-borne disease	Human babesiosis - serology - serum
Tick-borne disease	Human ehrlichiosis - IgG/IgM serology - serum
Tick-borne disease	Tick-borne encephalitis - TBE - IgG/IgM serology - serum
Tick-borne disease	Tularemia - Francisella tularensis - serology total antibodies - serum
Tick-borne encephalitis - TBE - IgG/IgM serology - serum	
TIF	Myosite Markers - serum
TIF 1	Myosite Markers - serum
Timothy syndrome (TS)	Cardiogenetic panel or exome
Tissue plasminogen activator - tPA - antigen - plasma	
Tissue Polypeptide Antigen	TPA - Tissue Polypeptide Antigen
Titanium	
Titin - anti-antibodies	Anti-titin antibodies - serum
TIVICAY	Dolutegravir - Serum
TKI	DNA NGS Panel for cholangiocarcinoma
TKI	DNA NGS Panel for pancreatic cancer
TKI	DNA NGS Panel for prostate cancer
TKI	DNA NGS Panel for thyroid cancer
TKI	DNA NGS Panel solid tumor Breast
TKI	DNA NGS Panel solid tumor Lung
TKI	DNA NGS Panel solid tumor Lung extended
TKI	RNA NGS Panel for cholangiocarcinoma
TKI	RNA NGS Panel for lung cancers
TKI	RNA NGS Panel for pancreatic cancer
TKI	RNA NGS Panel for thyroid cancer
TKI	RNA NGS Panel solid tumors, Pan-organ : 65 genes
TKI - ITK	DNA NGS Panel solid tumor Melanoma
TMAM127	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
TMC114	Darunavir - Serum
TMEM127	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
TMEM127	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
TMPRSS2	RNA NGS Panel solid tumors, Pan-organ : 65 genes
TNF-alpha - Tumour Necrosis Factor	
[To be translated]	ABO system immune antibodies - anti-A anti-B - screening and titration
[To be translated]	Alkaline phosphatases - fluid

[To be translated]	Alpha-1 Antitrypsin - PI phenotype
[To be translated]	Alpha-2 Antiplasmin - activity - plasma
[To be translated]	Anti-acetylcholine receptor antibodies - serum
[To be translated]	Anti-beta 2 glycoprotein 1 antibodies - B2GP1 - IgG - Serum
[To be translated]	Anti-beta 2 glycoprotein 1 antibodies - B2GP1 - IgM - Serum
[To be translated]	Anti-glomerular basement membrane antibody - Serum
[To be translated]	Anti-PR3 antibodies - proteinase 3 - Serum
[To be translated]	Bile acids - serum
[To be translated]	Class I HLA - HLA-A and HLA-B genotyping - EDTA whole blood
[To be translated]	Cryoglobulins - identification - serum
[To be translated]	Dabigatran - Measurement of the Dabigatran anti lia activity - plasma
[To be translated]	Electrolytes stool - Stool
[To be translated]	Extended phenotype
[To be translated]	Factor I - Fibrinogen - activity - plasma
[To be translated]	Genome by long-read sequencing (Nanopore) - EDTA whole blood
[To be translated]	Genotyping of dihydropyrimidine dehydrogenase
[To be translated]	Gilbert Syndrome - Screening for UGT1A1*28 (rs8175347) polymorphism
[To be translated]	Glutathione peroxidase - plasma
[To be translated]	Heparin induced thrombocytopenia - functional test
[To be translated]	hereditary spherocytosis - screening (eosin-5'-maleimide (EMA) staining)
[To be translated]	Immunophenotyping of malignant hemopaties
[To be translated]	Methylmalonic acid - urines
[To be translated]	Myelogram - Perls staining
[To be translated]	Platelet typing - EDTA whole blood
[To be translated]	Rheumatoid factors - serum
[To be translated]	Rickettsioses - R. conorii, R. typhi - serology - serum
[To be translated]	Sulfatides - urine
[To be translated]	Syphilis - IgG confirmation serology - serum
[To be translated]	Syphilis - IgM confirmation serology - serum
[To be translated]	Targeted Sequencing - Postnatal
[To be translated]	Targeted Sequencing - Prenatal
[To be translated]	TPMT Genotyping or testing for major polymorphisms in the TPMT gene: Alleles
[To be translated]	Uric acid
[To be translated]	
Tobacco test	Cotinine - urine
Tobi®	Tobramycin - serum
Tobradax®	Tobramycin - serum
Tobramycin - serum	
Tobrex®	Tobramycin - serum
Tocilizumab - serum	
Tocopherol	Vitamin E - Tocopherol - Plasma
Tolbutamide	Sulphonylureas - serum
Tolexine®	Doxycycline - Serum
Toluene - EDTA whole blood or Heparin	
Toluene - urine	
Topalgic®	Tramadol - Serum
Topiramate - serum	
Total anti-HBc antibodies	Hepatitis B - total anti-HBc antibodies - serum
Total anti-HBe antibodies	Hepatitis B - anti-HBe antibodies - serum
Total anti-HBs antibodies - titration	Hepatitis B - anti-HBs antibodies - titration - serum
Total anti-HDV antibodies	Hepatitis Delta - total antibodies - serum
Total antibodies - HAV	Hepatitis A - IgG immunity screening - serum
total antibodies IgG + IgM	Strongyloidiasis - serology - serum
Total B lymphocytes	Immunophenotyping: Lymphocytes B - CD19 - EDTA whole blood
Total Bile acids	Bile acids - serum
Total CK	CPK - Creatine phosphokinase total - serum
Total creatine phosphokinase	CPK - Creatine phosphokinase total - serum
Total haemolytic complement	CH50 - Total haemolytic complement

Total HCG (HCG dimer + beta fraction) - serum	
Total human chorionic gonadotropin - bound and unbound fractions	Total HCG (HCG dimer + beta fraction) - serum
Total human chorionic gonadotropin - bound and unbound fractions - liquid	hcG dimeric molecule (alpha + beta) - biological fluid
Total IgE - serum	
Total iron-binding capacity	Iron - total binding capacity - serum
total perfluorooctanesulfonic acid (PFOS)	Per- and polyfluoroalkyl compounds - 7 compounds panel - Serum
Total prostate-specific antigen	PSA - Prostate-specific antigen total - Aspiration biopsy
Total prostate-specific antigen	PSA - Prostate-specific antigen total - serum
Total protein S - antigen - plasma	
Total protein S antigen	Total protein S - antigen - plasma
Total protein S Antigen	Total protein S - antigen - plasma
Total PS Ag	Total protein S - antigen - plasma
Total T3 - Total triiodothyronine - serum	
Total T4 - Total thyroxine - serum	
total testosterone	
Total testosterone	Testosterone - man
Total testosterone	Testosterone - woman/child - serum
Total tetraiodothyronine	Total T4 - Total thyroxine - serum
Total thyroxine	Total T4 - Total thyroxine - serum
Total triiodothyronine	Total T3 - Total triiodothyronine - serum
toxicitY irinotecan ou Campto - UGT1A1	UGT1A1 : screening for UGT1A1*28(rs8175347) polymorphism - Involved in irinotecan
toxin	Clostridium difficile - Glutamate dehydrogenase (GDH) screening (+toxins)-stool
Toxocara canis	Toxocariasis - confirmation serology - serum
Toxocara canis	Toxocariasis - serology - serum
Toxocariasis - confirmation serology - serum	
Toxocariasis - serology - serum	
Toxoplasmosis - antibody avidity - IgG - serum	
Toxoplasmosis - IgG/IgM serology - serum	
TP53	DNA NGS Panel for brain solid tumors
TP53	DNA NGS Panel for cholangiocarcinoma
TP53	DNA NGS Panel for prostate cancer
TP53	DNA NGS Panel for thyroid cancer
TP53	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
TP53	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
TP53	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
TP53	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
TP53	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
TPA	TPA - Tissue Polypeptide Antigen
TPA - Tissue Polypeptide Antigen	
TPHA EIA	Syphilis - Treponemique test - screening serology - serum
TPHA ELISA	Syphilis - Treponemique test - screening serology - serum
TPM3	RNA NGS Panel solid tumors, Pan-organ : 65 genes
TPMT	Pharmacogenetic Passport - PharmacoPass Expert
TPMT	TPMT Activity or Thiopurine S-Methyltransferase Phenotyping - activity
TPMT	TPMT Genotyping or testing for major polymorphisms in the TPMT gene: Alleles
TPMT Activity or Thiopurine S-Methyltransferase Phenotyping - activity	
TPMT Genotyping or testing for major polymorphisms in the TPMT gene: Alleles TPMT*2,	
TPO - anti-antibodies	Anti-thyroid peroxidase antibodies
TPR	RNA NGS Panel solid tumors, Pan-organ : 65 genes
TRAB	Anti-TSH receptor antibodies
TRAK	Anti-TSH receptor antibodies
Tramadol - Serum	
Trans,trans-muconic acid	Trans-muconic acid - ttMA - urine
Trans-Chlordane	Pesticides - screening and/or dosage - urine
Trans-muconic acid - ttMA - urine	
Transcortin	CBG - Cortisol-Binding Globulin - Transcortin - serum
transcript study.	Targeted RNA sequencing
Transferrin - serum	
Eurofins Biomnis – International Division – Test guide Specialised medical pathology – Valid from : 01/08/2026	

Transferrin saturation percentage	Iron - saturation percentage - serum
Transposition of the Large Vessels	Exome Congenital heart defects
Transthyretin	Prealbumin - serum
Tranxene®	Clorazepate - Serum
trench fever	Bartonellosis - B. henselae - B. quintana - IgG serology - serum
Treponema	Syphilis - IgG confirmation serology - serum
Treponema	Syphilis - IgM confirmation serology - serum
Treponema	Syphilis - IgM serology - serum
Treponema	Syphilis - titration - CSF
Treponema	Syphilis - Treponemique test - screening serology - serum
Treponema pallidum	Genital ulcérations panel - direct diagnosis - PCR
Treponema pallidum	Syphilis - Treponemique test - screening serology - serum
Treponema pallidum - Detection via PCR	
treponemal test CSF	Syphilis - titration - CSF
treponematosi	Syphilis - IgG confirmation serology - serum
treponematosi	Syphilis - IgM confirmation serology - serum
treponematosi	Syphilis - IgM serology - serum
treponematosi	Syphilis - titration - CSF
treponematosi	Syphilis - Treponemique test - screening serology - serum
TRF	Transferrin - serum
Triazolam - serum	
Trichinosis	Trichinosis - confirmation serology - serum
Trichinosis	Trichinosis - IgG serology - serum
Trichinosis - confirmation serology - serum	
Trichinosis - IgG serology - serum	
Trichloracetate	Trichloroacetic acid + Trichloroethanol - TCE + TCA - urine
Trichloroacetic acid + Trichloroethanol - TCE + TCA - urine	
Trichloroethanol	Trichloroacetic acid + Trichloroethanol - TCE + TCA - urine
Trichomonas vaginalis	Chlamydia trachomatis, Neisseria gonorrhoeae and Trichomonas vaginalis - multiplex
Trichomonas vaginalis	Chlamydia trachomatis, Neisseria gonorrhoeae, Mycoplasma genitalium and
Trichomonas vaginalis	IST PMA Expert 2 Multiplex PCR detection 2 Urogenital mycoplasma, C. trachomatis, N.
Trichomonas vaginalis	Trichomonas vaginalis - Search by gene amplification
Trichomonas vaginalis - Search by gene amplification	
Tricyclic antidepressants	Antidepressants - screening - identification - dosage - urine
Tricyclic antidepressants	Antidepressants screening- identification - without dosage
Triflucon®	Fluconazole - Serum
Triiodothyronine - anti-antibodies	Anti-T3 antibodies - triiodothyronine - serum
Trileptal®	10-Hydroxy-carbazepine - serum
TRIM21	Anti-ENA antibodies (SSA, SSB, Sm, Rnp, Jo1, Scl-70) - screening and titration - Serum
TRIM24	RNA NGS Panel solid tumors, Pan-organ : 65 genes
TRIM33	RNA NGS Panel solid tumors, Pan-organ : 65 genes
TRIO	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Triplex testing for Chlamydia trachomatis and neisseria gonorrhoeae and mycoplasma	STI screening - multiplex gene amplification
Trisomy 21 - 1st trimester combined screening	
Trisomy 21 - second-trimester maternal serum markers - 2 parameters	
Trisomy 21 - second-trimester maternal serum markers - 3 parameters	
Triumeq®	Abacavir - plasma
Triumeq®	Lamivudine - plasma
Trizivir®	Abacavir - plasma
tRNA synthetases - anti-antibodies	Myosite Markers - serum
Tropatepine®	Lepticur - Serum
Trophatop adult - fx5/fx24/fx25	Specific IgE : food allergen testing - Trophatop® adult - fx5/fx24/fx25
Trophatop child	Specific IgE : food allergen testing - Trophatop® children - fx26/fx27/fx28
Tropheryma whipplei - direct diagnosis - PCR	
TRP	Phosphorus - tubular reabsorption rate
Truvada® (emtricitabine + tenofovir)	Emtricitabine - plasma
Truvada® (emtricitabine + tenofovir)	Tenofovir - plasma
Truxima®	Rituximab - serum

Trypanosoma	Trypanosoma cruzi - serology - serum
Trypanosoma	Trypanosoma gambiense - serology - CSF
Trypanosoma	Trypanosoma gambiense - serology - serum
Trypanosoma cruzi - confirmation - serum	
Trypanosoma cruzi - serology - serum	
Trypanosoma gambiense - screening - CSF	
Trypanosoma gambiense - serology - CSF	
Trypanosoma gambiense - serology - serum	
Trypanosomiasis - serology	Trypanosoma cruzi - serology - serum
Trypanosomiasis - serology	Trypanosoma gambiense - serology - CSF
Trypanosomiasis - serology	Trypanosoma gambiense - serology - serum
TSC1	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
TSC1	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
TSC1	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
TSC2	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
TSC2	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
TSC2	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
TSH - Thyroid-stimulating hormone	
TSH - Thyroid-stimulating hormone - Third generation - serum	
TSH receptor - anti-antibodies	Anti-TSH receptor antibodies
TSI	Anti-TSH receptor antibodies
TSI : THYROIDE STIMULATIN IMMUNOGLOBULIN	Anti-TSH receptor antibodies
TT	Thrombin clotting time - plasma
tt-MA	Trans-muconic acid - ttMA - urine
TTG - IgA - anti-antibodies	Human tissue transglutaminase antibody - IgA - serum
tTG - IgG - anti-antibodies	Human tissue transglutaminase antibody - IgG - serum
TTN	Cardiogenetic panel or exome
TTR	Amylose héréditaire à transthyréline - Etude moléculaire du gène TTR par séquençage
TTR	Prealbumin - serum
Tuberculosis - culture	Mycobacteria - culture
Tuberculosis - direct exam	Mycobacteria - direct examination
Tuberculosis - identification	Mycobactéria - identification
Tuberculosis - susceptibility testing	Mycobacteria - susceptibility testing
Tubular reabsorption rate of phosphorus	Phosphorus - tubular reabsorption rate
Tularemia - Francisella tularensis - serology total antibodies - serum	
Tumeur	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
Tumeur	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
Tumor	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
Tumor	Oncogenetic panel digestive tract Targeted genes: APC, BMPR1A, CDH1, EPCAM,
Tumor	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
Tumor	Pancreatic oncogenetic panel Target genes : APC, ATM, BRCA1, BRCA2, CDK4,
Tumor	Panel Hereditary breast ovarian cancer and prostate Gènes ciblés: ATM, BRCA1,
Tumor	Skin oncogenetic panel Gènes ciblés : ACD, BAP1, CDK4, CDKN2A, FLCN, MC1R,
Tumor Necrosis Factor	TNF-alpha - Tumour Necrosis Factor
TWAR	Chlamydia pneumoniae - IgG/IgM serology - serum
Type II collagen - anti-antibodies	Anti-type II collagen antibodies - serum
Type III procollagen - serum	
Typhoid fever - serology	Salmonellosis - serology - serum
Typing of the Lymphoproliferative Disorders	Immunophenotyping of malignant hemopaties
tyrosine kinase ABL mutation	
Tyrosine kinase inhibitor	DNA NGS Panel for cholangiocarcinoma
Tyrosine kinase inhibitor	DNA NGS Panel for pancreatic cancer
Tyrosine kinase inhibitor	DNA NGS Panel for prostate cancer
Tyrosine kinase inhibitor	DNA NGS Panel for thyroid cancer
Tyrosine kinase inhibitor	DNA NGS Panel solid tumor Breast
Tyrosine kinase inhibitor	DNA NGS Panel solid tumor GIST
Tyrosine kinase inhibitor	DNA NGS Panel solid tumor Lung
Tyrosine kinase inhibitor	DNA NGS Panel solid tumor Lung extended

Tyrosine kinase inhibitor	DNA NGS Panel solid tumor Melanoma
Tyrosine kinase inhibitor	RNA NGS Panel for cholangiocarcinoma
Tyrosine kinase inhibitor	RNA NGS Panel for lung cancers
Tyrosine kinase inhibitor	RNA NGS Panel for pancreatic cancer
Tyrosine kinase inhibitor	RNA NGS Panel for soft tissue tumors : 20 genes
Tyrosine kinase inhibitor	RNA NGS Panel for thyroid cancer
Tyrosine phosphatase - anti-antibodies	Anti-tyrosine phosphatase antibodies - IA-2 - serum
Tysabri®	Anti-Natalizumab antibodies - serum
Ubiquinone	Coenzyme Q10 - CoQ10 - Serum
UGT1A1	Pharmacogenetic Passport - PharmacoPass Expert
UGT1A1 : screening for UGT1A1*28(rs8175347) polymorphism - Involved in irinotecan	
Ultra sensitive thyroglobulin	Thyroglobulin
universal PCR	Detection and identification bacterial DNA - 16S PCR by NGS sequencing
Upshaw-Schulman syndrome	Factor Willebrand protease activity - ADAMTS 13
Uracil level plasma	Uracile - dosage(search a risk of toxicity to fluoropyrimidines)
Uracile - dosage(search a risk of toxicity to fluoropyrimidines)	
Urates	Uric acid - serum
Uraturia	Uric acid
Urbanyl®	Clobazam - Serum
Ureaplasma parvum	IST PMA Expert ¿ Multiplex PCR detection ¿ Urogenital mycoplasma, C. trachomatis, N.
Ureaplasma urealyticum	IST PMA Expert ¿ Multiplex PCR detection ¿ Urogenital mycoplasma, C. trachomatis, N.
Ureaplasma urealyticum	urogenital (ureaplasma urealyticum) mycoplasma (mycoplasma hominis) - serology -
Uric acid	
Uric acid - serum	
Uricacidemia	Uric acid - serum
Uricaciduria	Uric acid
Urinary calculi	Urinary stone - morphologic and infrared analysis
urinary lithiasis	Urinary stone - morphologic and infrared analysis
urinary parasite	Schistosomiasis - direct diagnosis - PCR
Urinary stone - morphologic and infrared analysis	
Urine proteins immunoelectrophoresis	Bence Jones - screening
urogenital (ureaplasma urealyticum) mycoplasma (mycoplasma hominis) - serology - serum	
Uroporphyrins	Porphyrins - stool
Uroporphyrins	Porphyrins - Urines
Ursodeoxycholic acid	Bile acids - chromatography - serum
Ursodeoxycholic acid	Bile acids - serum
Ustekinumab	Ustekinumab - serum
Ustekinumab - serum	
Uzpruvo	Ustekinumab - serum
V Leiden mutation	Factor V Leiden - p.Arg506Gln mutation
vaccination	Diphtheria - immunity test - IgG serology - serum
vaccination	Haemophilus influenzae B - IgG serology - serum
vaccination	Tetanus - immunity test - IgG serology - serum
Vaginal sampling and self-sampling	HPV test for vaginal sampling or self-sampling
Valaciclovir - Plasma	
Valganciclovir	Ganciclovir - plasma
Valium®	Diazepam + desmethyldiazepam - Serum
Valproic acid - serum	
Valpromide	Valproic acid - serum
Vancocin®	Vancomycin - serum
Vancomycin - serum	
Vancomycin-resistant enterococci (VRE)	Antibiotic multi-resistant bacteria (MRB) - screening
Vanilmandelic acid - Urine	
Vanilylmandelic acid	Vanilmandelic acid - Urine
Vanylmandelic acid	Vanilmandelic acid - Urine
Varicella zoster	Vesicular eruptions panel - direct diagnosis - PCR
Varicella zoster virus - direct diagnosis - PCR	
Varicella zoster virus - direct diagnosis - PCR - amniotic fluid	

Varicella zoster virus - IgG serology - serum	
Varicella zoster virus - IgM Serology - serum	
varicella-zoster virus	Varicella zoster virus - direct diagnosis - PCR - amniotic fluid
Variole du singe	Monkeypox virus - direct diagnosis - PCR
Vasoactive intestinal peptide	VIP - Vasoactive Intestinal Polypeptide - plasma
VAsodilator Stimulated Phosphoprotein	VASP - VAsodilator Stimulated Phosphoprotein - whole blood
Vasopressin	ADH - Antidiuretic hormone
Vasopressin	ADH - Antidiuretic hormone - urine
VASP - VAsodilator Stimulated Phosphoprotein - whole blood	
VCA - IgG serology	Epstein-Barr Virus - EBV - VCA IgG, EBNA, IgG serology - serum
VCA - IgM serology	Epstein-Barr Virus - EBV - VCA IgM serology - serum
VCA - VCA IgG/IgM serology	Epstein-Barr Virus - EBV - VCA IgG/IgM and EBNA IgG serology - serum
VDRL	Syphilis - VDRL or quantitative RPR
Vector-borne disease	Bartonellosis - B. henselae - B. quintana - IgG serology - serum
Vector-borne disease	Coxiella burnetii (Q fever) - IgG and IgM serology, Phases I and II - serum
Vector-borne disease	Coxiella burnetii (Q fever) - Screening serology Total Ac Phase II - serum
Vector-borne disease	Human babesiosis - serology - serum
Vector-borne disease	Human ehrlichiosis - IgG/IgM serology - serum
Vector-borne disease	Rickettsioses - R. conorii, R. typhi - serology - serum
Vector-borne disease	Tick-borne encephalitis - TBE - IgG/IgM serology - serum
Védolizumab	Vedolizumab - serum
Vedolizumab - serum	
Veggie Check	Juvenalis - Panel - VeggieCheck
Veneral Diseases Research Laboratory	Syphilis - VDRL or quantitative RPR
Venlafaxine - serum	
Verno-estival encephalitis	Tick-borne encephalitis - TBE - IgG/IgM serology - serum
Vesicular eruptions panel - direct diagnosis - PCR	
VFEND	Voriconazole - Serum
VHD RNA quantification	Hepatitis Delta - Viral load - PCR
VHE	Hepatitis E - direct diagnosis - PCR
VHL	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
VHL	Neuroendocrine Oncogenetics Panel Target genes : DLST, DNMT3A, EGLN1, EGLN2,
VHL	Oncogenetic panel Lung Target genes : ATM, BAP1, BRCA1, BRCA2, CDKN2A,
VHL	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
Vibramycine®	Doxycycline - Serum
Vibraveineuse®	Doxycycline - Serum
Vibrio cholerae	Gastrointestinal Panel - direct diagnosis - PCR - Stools
Vibrio (V. parahaemolyticus/V. vulnificus/ V. cholerae)	Gastrointestinal Panel - direct diagnosis - PCR - Stools
Victan®	Ethyl loflazepate - serum
Vigabatrin - Serum	
VIH	HIV - Human Immunodeficiency Virus - viral load - CSF
VIH 1	HIV - Human Immunodeficiency Virus - viral load - CSF
VIH-1	HIV-1: Qualitative RNA detection, suspected primary infection - serum
VIII R (related)-Ag	Von Willebrand factor - plasma
VIP - Vasoactive Intestinal Polypeptide - plasma	
viral meningoencephalitis panel - direct diagnosis - PCR	
VIREMIE HBV	Hepatitis B - viral load - PCR
VIREMIE HCV	Hepatitis C - viral load - quantitative PCR
VIREMIE HIV	HIV1 - Human Immunodeficiency Virus - viral load - Plasma
Virus FibroMeter	FibroMeter
virus influenza type A	Influenza - direct diagnosis - PCR
virus influenza type B	Influenza - direct diagnosis - PCR
Visceral larva migrans - serology	Toxocariasis - confirmation serology - serum
Visceral larva migrans - serology	Toxocariasis - serology - serum
Vitamin A - Retinol - Plasma	
Vitamin B1 - Thiamin	
Vitamin B12 - serum	
Vitamin B2 - Riboflavin	
Eurofins Biomnis – International Division – Test guide Specialised medical pathology – Valid from : 01/08/2026	

Vitamin B6 - Pyridoxal phosphate	
Vitamin B8 or H - Biotin	
Vitamin B9	Folic acid - serum
Vitamin D - 1,25-dihydroxyvitamin D - serum	
Vitamin D3 1,25-hydroxy	Vitamin D - 1,25-dihydroxyvitamin D - serum
Vitamin E - Tocopherol - Plasma	
Vitamin H	Vitamin B8 or H - Biotin
Vitamin K1 - Phylloquinone	
VITT	Anti-PF4 antibodies - heparin - plasma
VKORC1	Pharmacogenetic Passport - PharmacoPass Expert
VLM - serology	Toxocariasis - confirmation serology - serum
VLM - serology	Toxocariasis - serology - serum
VMA	Vanilmandelic acid - Urine
von Willebrand - factor	Von Willebrand factor - plasma
Von Willebrand antigen factor	Von Willebrand factor - antigen - plasma
von Willebrand factor	Von Willebrand factor - antigen - plasma
Von Willebrand factor	Von Willebrand factor - plasma
Von Willebrand factor - antigen - plasma	
Von Willebrand factor - plasma	
Voriconazole - Serum	
Vortioxétine - Serum	
Votubia	Everolimus
VRS	Respiratory Infection Panel - direct diagnosis - PCR
VWF:Ag	Von Willebrand factor - antigen - plasma
vWFCP	Factor Willebrand protease activity - ADAMTS 13
VWF:Rco	Von Willebrand factor - plasma
VZV	Varicella zoster virus - direct diagnosis - PCR - amniotic fluid
VZV	Vesicular eruptions panel - direct diagnosis - PCR
VZV	viral meningoencephalitis panel - direct diagnosis - PCR
VZV - IgM serology	Varicella zoster virus - IgM Serology - serum
VZV - immunity test - IgG antibody	Varicella zoster virus - IgG serology - serum
VZV - immunity test - IgG antibody	Varicella zoster virus - IgG serology - serum
VZV - viral DNA - PCR	Varicella zoster virus - direct diagnosis - PCR
Water - free - clearance	
West Nile serology	West Nile virus - IgG/IgM serology - serum
West Nile virus	West Nile virus - IgG/IgM serology - serum
West Nile virus - IgG/IgM serology - serum	
Western blot HTLV	HTLV I/II - confirmation serology - serum
Wezenla	Ustekinumab - serum
WGS	Genome by long-read sequencing (Nanopore) - EDTA whole blood
Whipple - disease	Tropheryma whipplei - direct diagnosis - PCR
Whipple's disease	Tropheryma whipplei - direct diagnosis - PCR
WHOLE EXOME SEQUENCING	Prenatal exome
whooping cough	Pertussis - serology
Whooping cough	Respiratory Infection Panel - direct diagnosis - PCR
Whooping cough - Bordetella pertussis and parapertussis - direct diagnosis - PCR	
Widal and Félix serodiagnosis	Salmonellosis - serology - serum
Willebrand factor cleaving protease	Factor Willebrand protease activity - ADAMTS 13
Willebrand factor protease	Factor Willebrand protease activity - ADAMTS 13
Wilms	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
WNV	West Nile virus - IgG/IgM serology - serum
WT1	Extended oncogenetic panel sein, ovaire, prostate, tube digestif, pancréas, rein, neuro-
WT1	Renal oncogenetic panel Gènes ciblés: BAP1, BRK1, CDKN2B, FH, FLNC, HNF1B,
WT1	RNA NGS Panel solid tumors, Pan-organ : 65 genes
Xarelto®	Rivaroxaban - Anti Xa activity measurement - plasma
Xeplion	Risperidone
Xeroquel	Quetiapine
Y chromosome - (regions AZFs, AZFb, AZFc)	

Yeasts - antifungal susceptibility testing		
Yellow Fever - immunity test - serum		
Yersinia enterocolitica		Gastrointestinal Panel - direct diagnosis - PCR - Stools
YFV		Yellow Fever - immunity test - serum
Yo - anti-antibodies		Anti-neuron antibodies - serum
Yuflyma		Adalimumab
Zarontin®		Ethosuximide - serum
Zessly		Infliximab
Ziagen®		Abacavir - plasma
Zinc		
Zinc - seminal plasma		
Zinc - urines		
Zincaemia		Zinc
Zn		Zinc
Zn		Zinc - seminal plasma
Zn		Zinc - urines
Zoloft®		Sertraline - serum
Zonagran		Zonisamide - Serum
Zonisamide - Serum		
Zuclopenthixol - Serum		
Zyprexa®		Olanzapine

www.eurofins-biomnis.com/en/

International Division | 17/19 av. Tony Garnier | 69007 LYON | FRANCE
Tel.: +33 (0)4 72 80 23 85 | Fax: +33 (0)4 72 80 73 56 | E-mail: International@biomnis.eurofinseu.com